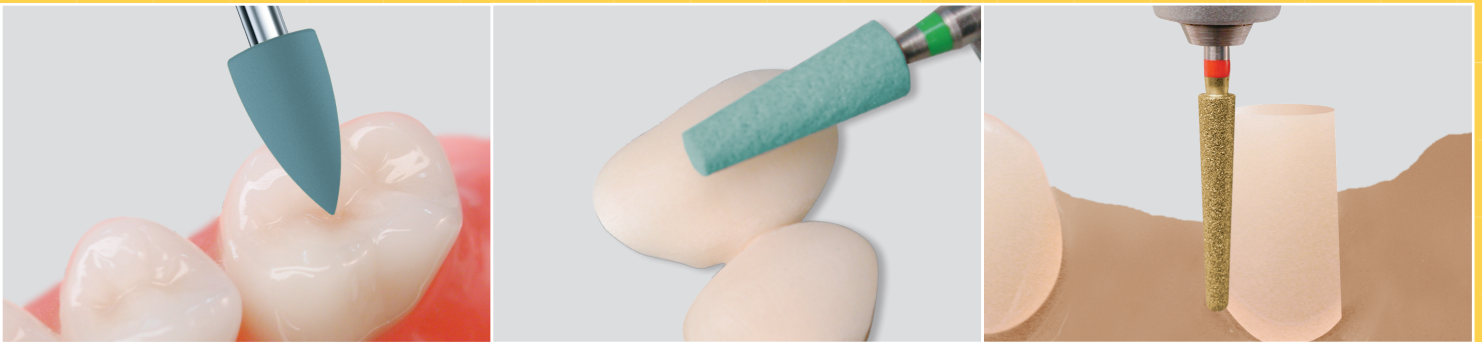
Keramik - Restaurationen
Porcelain restorations
Restaurations en céramique

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	Seite
	Page
	Page
Kronenstumpfpräparation	
Preparing teeth for crowns	
Préparation de moignons de couronnes	4
CeraPREP Set	
CeraPREP Set	
Set CeraPREP	4
WhiteTIGER Diamanten	
WhiteTIGER Diamonds	
Instruments diamantés WhiteTIGER	5
Okklusale Adjustierung/Trepanation/Politur/Keramik Kits	
Occlusal Adjustment, Trepanation, Polishing and Ceramic Kits	
Ajustage occlusal/trépanation/polissage/Kits pour céramique	6
Politur - CeraGloss/CeraPol Plus/Exa CeraPol/CeraPol Super	
Polishing - CeraGloss/CeraPol Plus/Exa CeraPol/CeraPol Super	
Polissage - CeraGloss/CeraPol Plus/Exa CeraPol/CeraPol Super	7
Entfernen einer Vollkeramik - Restauration (Kronentrenner) 5114RA/C34	
Removing an all-porcelain restoration (crown cutter) 5114RA/C34	
Dépose d'une restauration tout céramique (coupe-couronne) 5114RA/C34	8
Entfernen einer Niedertemperatur Keramik - Restauration C4AK	
Removing a low-fusing porcelain restoration C4AK	
Dépose d'une restauration en céramique à basse fusion C4AK	8
Nachbearbeitung des ungesinterten Brückengerüsts	
Finishing an unsintered bridge framework	
Usinage de l'armature de bridge encore non frittée	9
Aufpassen des gesinterten Zirkonoxid Gerüsts/Hartmetall - Fräser für die Keramikbearbeitung	
Fitting the sintered zircon oxide framework/TC Cutter for use with porcelain restorations	
Ajustage de l'armature en zircone frittée/Fraise en carbure pour les restaurations céramique.....	9
Grobschliff - SuperMax/Vorkonturierung - CeraPro	
Rough trimming - SuperMax/Initial contouring - CeraPro	
Abrasion puissante - SuperMax/l'usinage des contours - CeraPro	10
Politur - CeraGloss/Exa CeraPol/CeraPol Super/Occlupol	
Polish - CeraGloss/Exa CeraPol/CeraPol Super/Occlupol	
Polissage - CeraGloss/Exa CeraPol/CeraPol Super/Occlupol	11/12/13
Bearbeitung vollkeramischer Primärteile - Diamantinstrumente MT	
Preparation of all-porcelain primary units - Diamond Instruments MT	
Usinage de pièces primaires tout céramique - Instruments diamantés MT	14/15

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Porcelain and all - porcelain restorations

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Rotary instruments as integrated components in the dental practice for maximum safety during preparation and optimum aesthetics. Also specially designed for all-porcelain restorations with zircon oxide.

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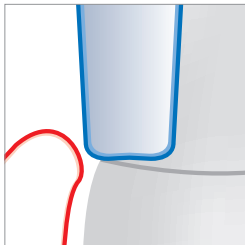
Preparing teeth for crowns

The tooth is generally prepared according to the usual principles for shoulder and deep chamfer preparations with a preparation depth of 1 mm at the preparation margin.

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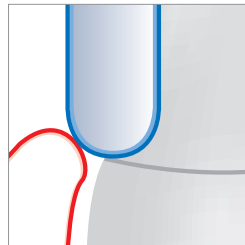
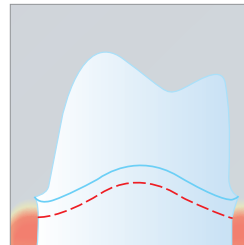
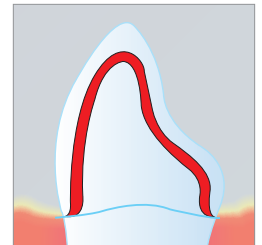
CeraPREP Set

Specially developed and patented WhiteTiger diamonds, enable a controlled and minimally invasive reduction of the tooth structure. For efficient preparation of porcelain restorations with either a shoulder or deep chamfer preparation in the anterior and posterior regions.

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CeraPREPSet Art.No. 40010SO Kronen & Brücken-Präparationsset

CeraPREP Set Order no. 40010SO Crown & Bridge Preparation Set

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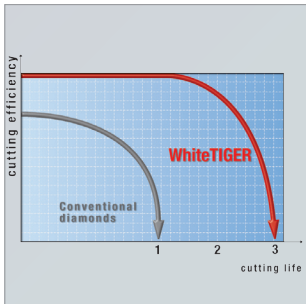
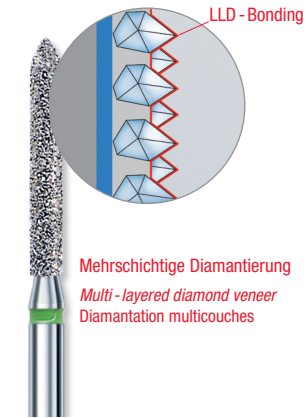
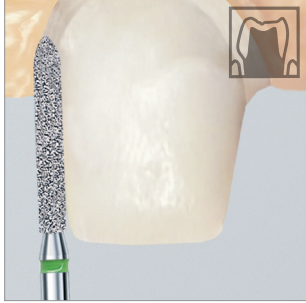
opt. 160.000

3

WhiteTIGER Diamonds for crown preparation

The multilayered diamond coating and patented LLD bonder ensure increased material reduction with excellent chip flow. There is greater reduction of tooth structure compared with standard diamonds, saving time during treatment as well as reducing patient discomfort.

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Occlusal Adjustment and Polishing

Fine-grit White Tiger diamond instruments are used for adjusting occlusal contact points. Ceramic polishers are used for polishing.

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Trepanation with WhiteTiger diamonds

An access cavity is prepared in two stages. ① The facing porcelain is first removed with a diamond instrument without perforating the porcelain framework. ② The porcelain framework is then perforated with a diamond instrument leaving a gap of 0.5 mm from the bonded porcelain (to prevent bonded porcelain breaking off).

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CeraGloss Ceramic Kit 30040SO

Unique diamond-impregnated surface treatment system for high-gloss polishing of surgery ceramics. Time-saving, convenient single step process. Natural, high temperature-resistant rubber with top-grade natural diamond particles which ensure rapid surface reduction combined with a high-gloss result of unsurpassed excellence. Set contains 6 CeraGloss RA polishers.

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Porcelain Veneer Kit 0390SO

Kit consists of 9 assorted instruments in RA and FG Shank, specifically designed for the Porcelain Veneer Technique. The fine diamond (red ring) is used in preparation of the tooth and the special silicone CeraPol Plus diamond impregnated points are used to obtain a smooth and satin finish. Its application can be effected dry or by using water-spray.

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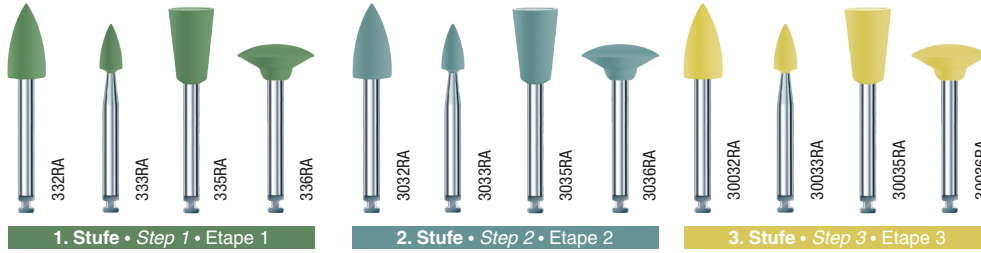
10.000 - 12.000, 6/100

CeraGloss

3 - stage polishing system with natural diamond grit for an optimum polish on all porcelain materials.

Stage 1, preparation (green)

Stage 2, smoothing (blue)

Stage 3, high-lustre polish (yellow) - without polishing paste.

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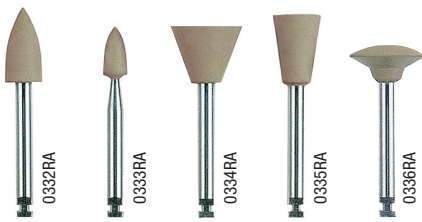
CeraGloss Kit

3 - stage diamond polishing system for all types of porcelain fillings, partial and full crown restorations. Polishers with natural diamond grit for finishing, polishing and high - lustre polishing of porcelain surfaces. Further glaze firing is unnecessary after adjustment.

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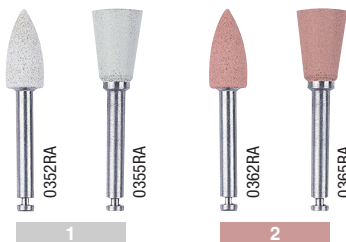
CeraPol Plus

1 - stage diamond polishing system for all types of porcelain fillings, partial and full crown restorations. Polishers with natural diamond grit for high - lustre polishing of porcelain surfaces. Further glaze firing is unnecessary after adjustment. Easy to use, saves time and produces an optimum polish.

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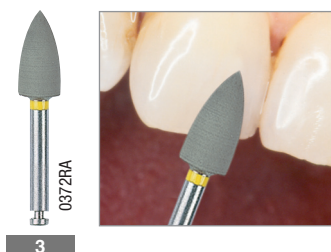
Exa Cerapol

2 - stage finishing and polishing system for all types of porcelain fillings, partial and full crown restorations. Polishers for finishing and polishing porcelain surfaces. Further glaze firing is unnecessary after adjustment. Porcelain polisher **light grey** = abrasive prepolish, smoothes surfaces ground with diamonds. Porcelain polisher **pink** = slightly abrasive polish, maintains the anatomical structure.

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CeraPol Super

Polisher for all types of porcelain fillings, partial and full crown restorations. For high - lustre polishing of porcelain surfaces, produces a natural glaze without polishing paste. Further glaze firing is unnecessary after adjustment.

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opt. 20.000 1

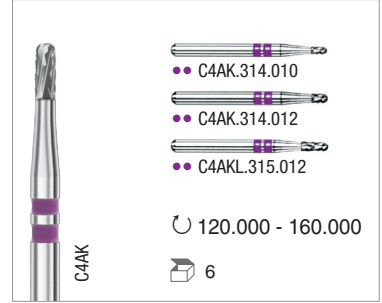
Removing an all-porcelain restoration (crown cutter)

To remove an all-porcelain restoration, cut the restoration open along the length of the axial wall to the centre of the occlusal surface or incisal edge using the sinter diamond disc order no.5114RA. Then bend the restoration open with a suitable instrument to make it fracture. No. 9920 Cleaning stone

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Removing a low-fusing porcelain restoration.

Crown cutter C4AK for all types of metal alloys, titanium and low-fusing porcelain and composite facings: **C4AK Crown cutter with a new MultiCut design.** Separates crowns and bridges easily and quickly in one working stage. These crown cutters are not suited for cutting all - ceramics restorations of extremely hard ceramics, such as zirconium - oxide ceramics.

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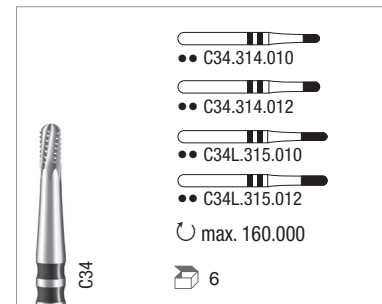
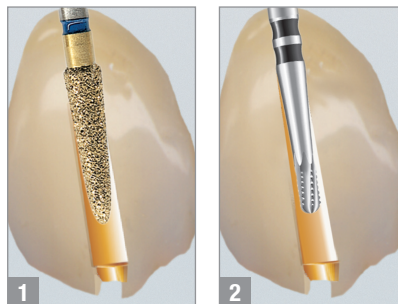
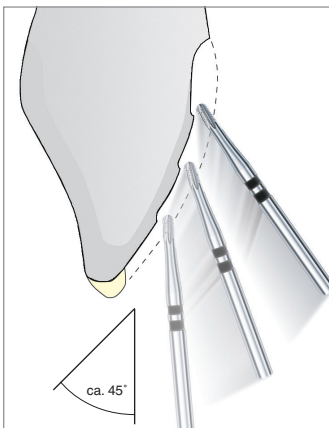
Crown cutter C34

User recommendations: When cutting crowns and bridges the following should be observed:
1. The veneer should be first separated with a diamond instrument.
2. Crown material reduction of the crown material layer, which means starting the procedure from top to bottom end several times.
Max. speed: 120.000 - 160.000
Contact pressure: 0.5 N

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C34 TC Crown Separator (Fig. 2) for: Crown - & Bridge - Metal separation.
 ■ optimal cutting geometry
 ■ high material reduction
 ■ virtually vibration-free
 ■ longterm performance

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Rotary instruments as integrated components in the dental laboratory for maximum safety during preparation and optimum aesthetics. Specially designed for use with **all-porcelain restorations with zircon oxide.**

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Finishing an unsintered bridge framework

Smoothing the connectors after milling the bridge framework in a CAD milling unit with tungsten carbide cutters and polishers.

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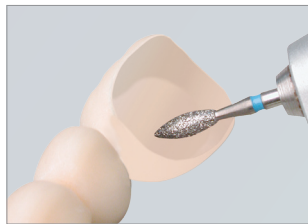
Fitting the sintered zircon oxide framework




By trimming the high spots with diamond instruments. Caution: always use water coolant and minimum pressure when preparing.

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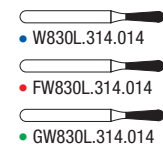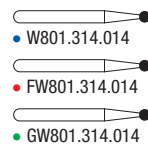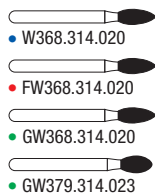The porcelain is trimmed under water coolant using diamonds in the turbine.

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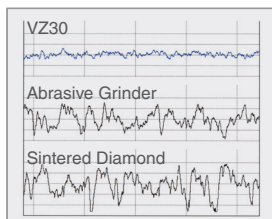
Optimum material reduction and excellent surface quality

when preparing porcelain and composite facings. Minimum vibration when preparing results in optimum surface quality, superior to preparing with diamond instruments. Controlled material reduction and increased cost-effectiveness.

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Rough trimming ①

SuperMax

Sintered diamond with organic bonder, enables rough trimming without heat build-up in the restoration. Suitable for rough trimming and pressable porcelain sprues.

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Initial contouring ②

CeraPro

Abrasive trimmer with natural diamond grit. Ideal for initial contouring and preparing without heat-build in the restoration. Suitable for preparing larger vestibular and oral surfaces.

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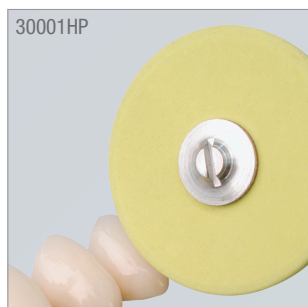
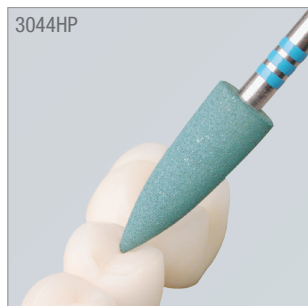
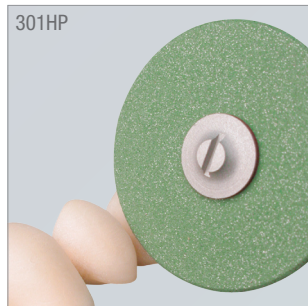
CeraGloss

Diamond Porcelain Polishers

3-step diamond - polishing system for working on all types of ceramic without heat generation. A new binding component as well as the natural diamonds allow for optimum polishing results, achieved without any polishing paste.

1. step: Green = coarse grit
for pre - grinding, for quick and bulk material reduction.
2. step: Blue = medium-coarse grit
for smoothing the surfaces and preparation for final polishing.
3. step: Yellow = super - fine grit
for high - shine polishing without additional final glaze.

User Information:

Light working pressure.

In order to achieve optimal polishing results, it is important to use the three polishers in the sequence as mentioned.

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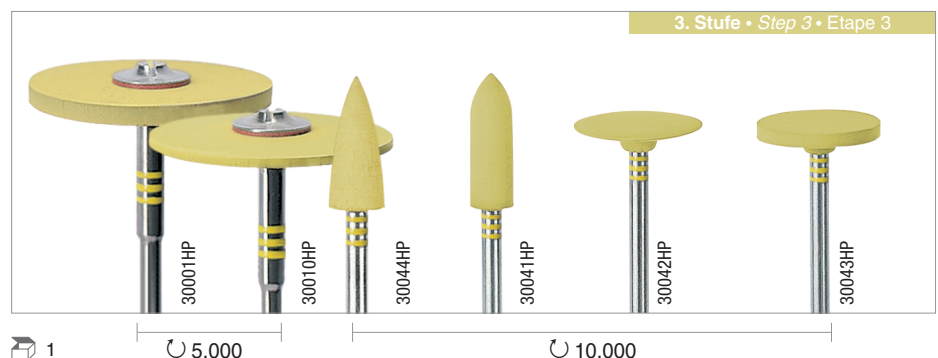
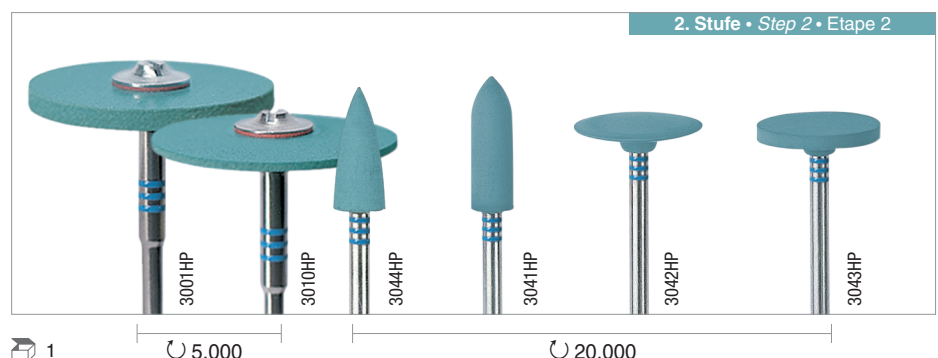
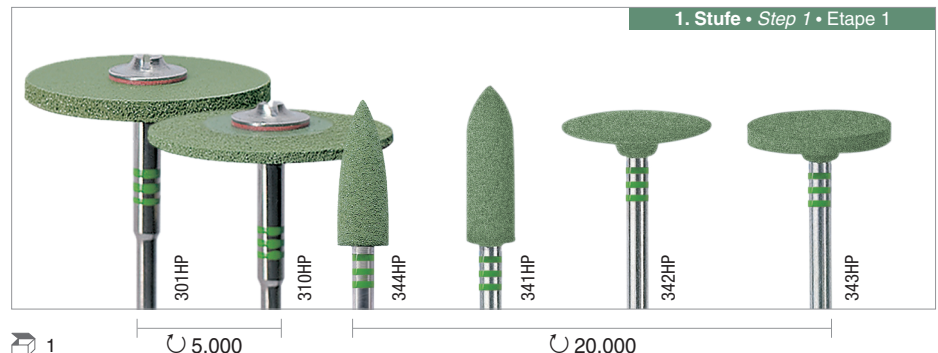
🔄 20.000
📅 12/100

Exa CeraPol

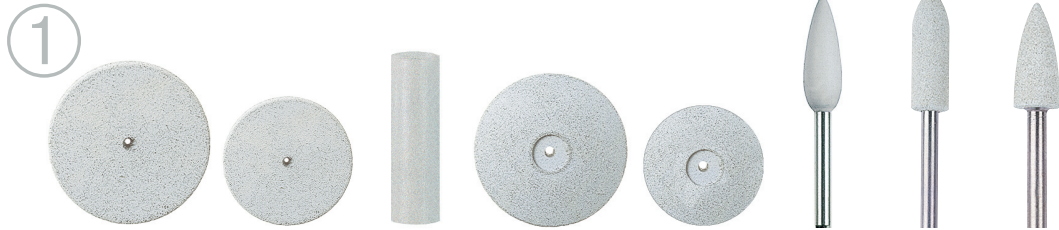
for porcelain/ceramics, grey-white, abrasive, eliminates scratches and smoothes the surface.

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🔄 10.000
📅 12/100

Exa CeraPol

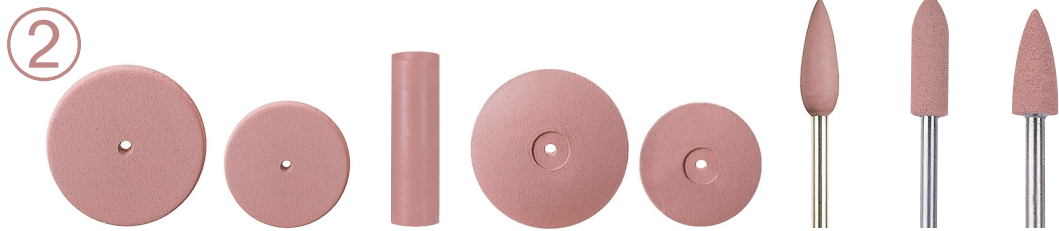
for porcelain/ceramics pink, retains the structure and provides a final shine

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🔄 5.000
📅 12/100

CeraPol Super

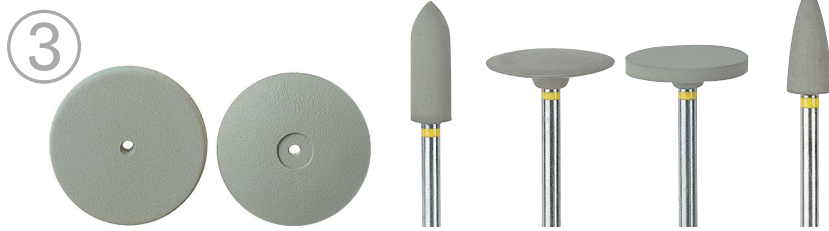
for porcelain/ceramics grey, with extra fine grain provides a lustre high-gloss finish without polishing paste.

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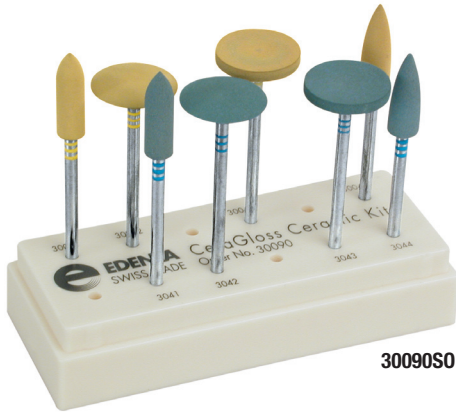
30090SO

CeraGloss HP Ceramic Kit 30090SO

Unique diamond-impregnated surface treatment system for **contouring & high-gloss polishing of laboratory ceramics**. Time-saving, convenient two-step process. Synthetic, high temperature-resistant rubber with a unique concentration of top-grade natural diamond particles which ensure rapid surface reduction combined with a high-gloss result of unsurpassed excellence.

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Cerapol Porcelain Kit 0391SO

Set consists of 1 dozen white wheels for pre-polishing. 1 dozen pink wheels for high-gloss polishing and 1 dozen grey wheels for final polishing and 1 dozen TC cutters, cross cut superfine and 3 mandrels. Important note: rpm recommendations i.e. for grinding and abrasive work \curvearrowright 20.000 for final polish – below \curvearrowright 10.000

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\curvearrowright 20.000
 \square 25

Occlupol

Polisher with diamond impregnation for preparation of occlusal surfaces on ceramics.

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\square 6/100

Occlupol Mandrel

Mandrel HP, especially designed to guarantee a firm hold on all Occlupol polishers and of the steel brush in any position.

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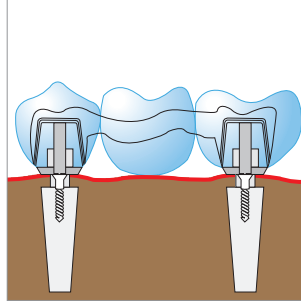
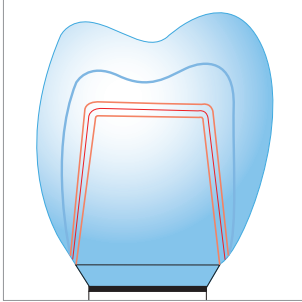
Preparation of all-porcelain primary units (implant prosthetics)

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Rotary instruments for preparing all-porcelain primary units in telescopic technique, telescopic-borne implants all-porcelain abutments.

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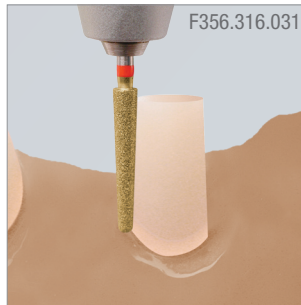
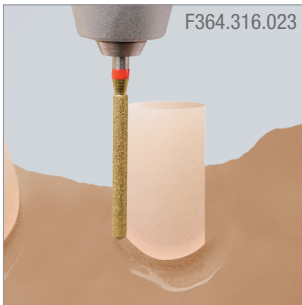
Diamond Instruments MT

For use in milling units with air turbine and water coolant.
For ideal surface contouring.

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2° diamonds available in standard and fine grit sizes.
0° cylindrical diamonds available in standard and fine grit sizes.

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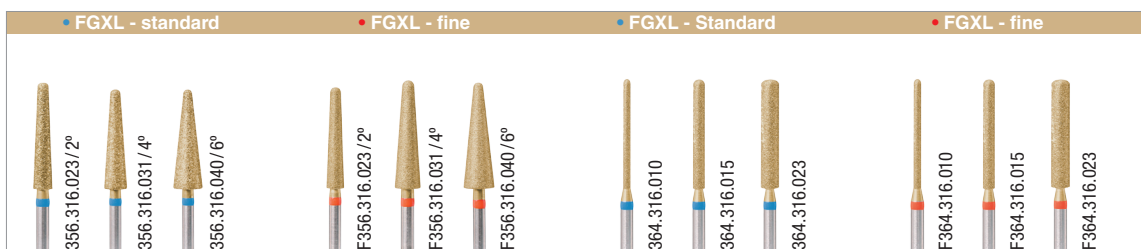
Caution: always prepare using water coolant and minimum pressure.

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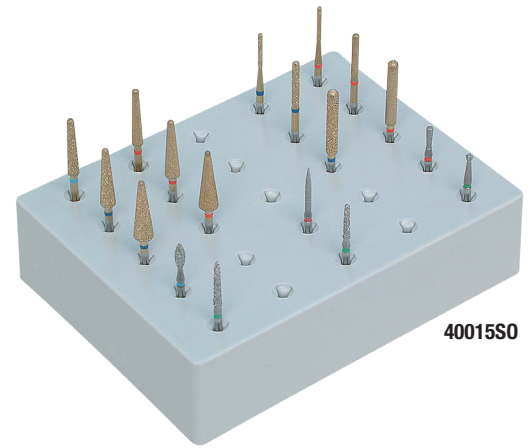
Special instruments for preparing all porcelain primary units without heat build-up to prevent microcracks in the telescopic technique, telescopic-borne implantology or all-porcelain abutments.

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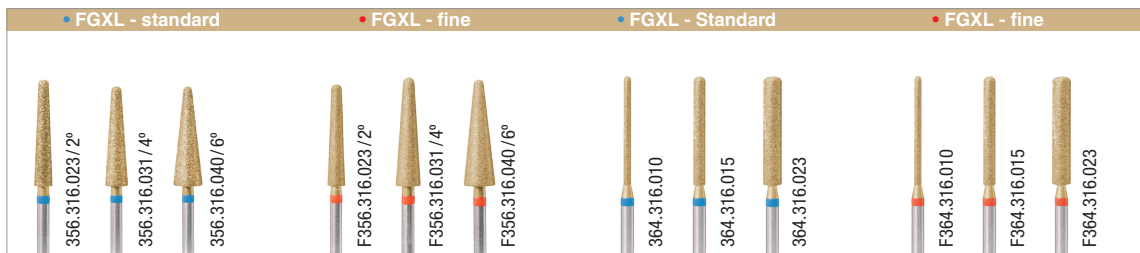
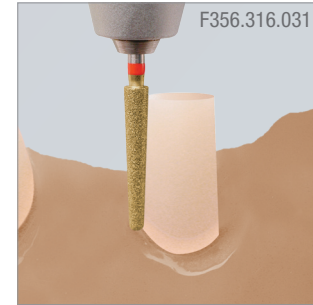
1 MT Diamond Rotary Instruments

Diamond rotary instruments for use in an air turbine milling unit with water cooling. 2°/4°/6° diamond rotary instruments with FG shank available in standard and fine grit sizes. 0° cylindrical diamond rotary instruments with FG shank available in standard and fine grit sizes (all diamond instruments are also available with short HP shank).

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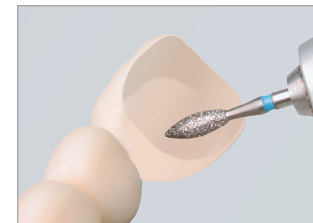
2 WhiteTIGER Diamond Rotary Instruments

Special diamond rotary instruments with multilayered diamond coating and LLD bonding for removing high spots on sintered zircon oxide porcelain materials. The porcelain should be trimmed using a turbine and water cooling. Special features of these diamond rotary instruments are their high cutting capacity, smooth ground surface and maximum service life.

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CeraGloss Polierer

PORCELAIN INSTRUMENT SET

Special rotary instruments for preparing porcelain, pressable porcelain and all-porcelain without heat build-up to prevent microcracks. Preparation is completed in three stages:

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