



KEYSTONE
INDUSTRIES

Itsoclear

HIGH QUALITY CRYSTAL CLEAR CLASP

Itsoclear provides you with the ability to produce and repair a clasp in under 4 minutes with a cosmetic appeal unlike any other clasp.

Itsoclear's superior difference stands out from all the rest

- Saves bench time producing a higher quality dental esthetic clasp that your customers will appreciate while increasing your profit.
- Crystal clear, will not lose its clarity
- Non-abrasive
- Easily adjustable
- No injection machines or hand-held injectors required
- Perfect for new partial dentures as well as repairs and replacing metal clasp

4 easy steps to create your Itsoclear clasp in under 4 minutes

- 1A)** Warm clasp by lightly waving it over flame. (WARNING: Do not overheat the clasp)
- 1B)** Lightly pass 3 prong plier over flame. (WARNING: Overheating plier will cause indentation in clasp)
- 1C)** Adapt accordingly around tooth by re-warming the 3 prong plier as needed.
- 2A)** After clasp is formed around tooth begin to create retention.
- 2B)** Form retention by bending a loop at opposite end of clasp to hold in acrylic.
- 2C)** Once retention is formed the clasp is ready to be set in place on frame.
- 3)** Secure clasp on frame by using glue and accelerator.
- 4)** Finally, place acrylic under retentive area and above the clasp so that the clasp will remain in position.

Your partial denture is now ready to be processed as normal.

REPAIRS or REPLACEMENTS: Cut groove where clasp is to be formed and cut a deep groove in tentative area. Process as normal.

Clear Universal Clasps, 6/Pkg. *Recommended use on pre-molar/anterior teeth.*

Clear Molar Clasps, 6/Pkg. *Recommended use when stronger reinforcement is needed.*

Available in the U.S.



Wrought Wire
Clasp



Wrought Wire
vs Itsoclear



Itsoclear



Itsoclear

DEDICATION – to customers is our 1st Priority

Since 1908, Keystone Industries has maintained a reputation for producing innovative, high-tech products. We continuously strive to exceed customer expectations for product quality, competitive pricing, prompt delivery and knowledgeable customer service.

Headquarters in Cherry Hill



Keystone Industries, USA 616 Hollywood Avenue, Cherry Hill, NJ 08002
Tel: +1 800-333-3131 | Fax: +1 856-663-0381 | www.keystoneind.com

Keystone Industries GmbH, Werner-von-Siemens Str. 14a, 78224 Singen - Germany
Tel: +49 77 31 91 21 01 | Fax: +49 77 31 91 21 02 | sales@keystoneind.com

CONTACT

USA & Canada

Keystone Industries
616 Hollywood Avenue
Cherry Hill, NJ 08002
USA

Keystone Industries - Myerstown
52 West King Street
Myerstown, PA 17067
USA

Mr. Chris McDevitt
Vice President, Domestic
cmcdevitt@keystoneind.com

Tel: +1 800-333-3131
Fax: +1 856-663-0381



International

Keystone Industries GmbH
Werner-von-Siemens Str. 14a
78224 Singen - Germany

Tel: +49 77 31 91 21 01
Fax: +49 77 31 91 21 02

Mr. Mike Prozzillo
Vice President, Keystone
Laboratory & International Sales
mprozzillo@keystoneind.com

Mr. Miguel Perez
Vice President, Keystone International
Business Development
mperez@keystoneind.com

Ms. Monica Rocco
Vice President, International Sales
mrocco@keystoneind.com

PRODUCT RANGE GROUPS

The Keystone product range for lab technicians includes the following groups:

DENTAL PACKAGING

page 5-8

ACCESSORIES | SHIPPING BOXES | STORAGE BOXES

DISPOSABLES/INFECTION CONTROL

page 9-14

BARRIER PRODUCTS | DISPOSABLES | EVACUATION PRODUCTS | FACE MASKS & SHIELDS | NEEDLES | PROCEDURE ACCESSORIES | STERILIZATION PRODUCTS

LABORATORY

page 15-72

ABRASIVES | ACRYLICS | ARTICULATORS | BRUSHES & BUFFS | BURNERS & TORCHES | CASTING | DEBUBLIZERS | DENTURE ACCESSORIES | DIE & MODEL FABRICATION | EQUIPMENT | INSTRUMENTS | LAB SAFETY | LAMP/MAGNIFIER | MISCELLANEOUS | POLISHING | PORCELAIN ACCESSORIES | ROTARY | WAXES

OPERATORY

page 73-78

ARTICULATING | CEMENTS & LINERS | COSMETIC DENTISTRY | ETCHANT GELS | IMPRESSION MATERIAL | MATRIX MATERIALS | MEDICAMENT MATERIALS | NEEDLESS INJECTOR | SURGICAL PRODUCTS

ORAL HEALTH

page 79-82

FLUORIDES | PREVENTATIVES | PROPHY PRODUCTS | TOPICAL ANESTHETICS

PRO-FORM/VACUUM FORMING

page 83-90

THERMAL PLASTICS | VACUUM FORMERS | VACUUM FORMING ACCESSORIES

Service and Repair Centers: page 87

REGULATORY



Keystone Industries has a quality system in compliance with FDA's cGMP requirements and is certified to ISO 9001:2008 and ISO 13485:2003, governing the requirements for medical devices.

In addition, Keystone's quality system strives to conform to all foreign regulatory requirements for the products sold and activities performed.



ACCESSORIES6

STORAGE BOXES.....7

SHIPPING BOXES.....6



DENTAL PACKAGING

ACCESSORIES

Alert Pouch Bag

An excellent, durable ziplock delivery bag made of heavy-duty 2 ml plastic. The 5" x 8" (127 x 203.2 mm) bag comes with a convenient 1.5 ml pouch system that enables the prescription to stay dry with the casework. Liquid- and air-tight ziplock.

BAGS MEET ALL OSHA STANDARDS FOR DELIVERY.



#9557520 1,000/Pkg

#9557510 100/Pkg

Biohazardous Pouch Bag

This bag offers the solution for transporting biohazardous material while limiting occupational exposure to potentially infectious materials. The liquid- and air-tight zipper ensures the contents remain in the bag. The bag is made of heavy-duty plastic. Biohazard labels are visible to ensure that all handlers are aware of the biohazardous contents.

BAGS MEET ALL OSHA STANDARDS FOR DELIVERY.



#95575401,000/Pkg

#9075450100/Pkg

Convuluted Fillers

These Convuluted Fillers give maximum protection to larger dental cases. To maximize protection of appliance or impression, use one filler set per box with a barrier bag. The filler eliminates the need for bubble wrap or tissue.



#9085050Medium Size, 2/Set

#9084820Large Size, 2/Set

Mailer Labels

Make sure your first-class mail looks first class thanks to our mailer labels. These labels work perfect for our plastic mailer boxes carrying important dental appliances. Make sure your mailer is shipped to the right place thanks to these easy-to-read labels. The labels fit our medium and large plastic mailer boxes. Available Size: 5-1/2" x 3-1/2" (139.7 x 88.9 mm).



#9085060Mailer Label, each

Occlusal Surface Protection

Keystone makes fluoride trays for the dental industry. These same trays work great as protective packaging for dentures and partials. Order the large lowers and wrap the occlusal surface of your outgoing cases.



#952513050/Pkg

SHIPPING BOXES

Articulator Boxes

A new, reinforced mailer or delivery box for your expensive articulators and fragile casework. These heavy-duty boxes have a removable handle for carrying and reinforced wall thickness for better protection.

Each box also includes a dense foam filler set to help further protect your valuable work.

Box is 8" x 8" x 8" (203.2 x 203.2 x 203.2 mm).



#9577810Articulator Box, each

#9090570Replacement Foam Filler Set

Model Storage Boxes

This is an ideal box for long-term storage of patient models. These boxes are made from double-strength corrugated cardboard to provide maximum protection.

Each box has four separate compartments and four identification labels for ideal organization. The boxes are shipped flat to optimize storage space and are easy to assemble. The boxes are 7" x 7" x 2" (177.8 x 177.8 x 50.8 mm) in size.



#9582740Model Storage Boxes, 25/Pkg

#9582750Model Storage Boxes, 100/Pkg

Plastic Mailer - Large

Ship away dental work with no worries with this large plastic mailer. The box is perfect for a small articulator and four full models. Made of chemical-tolerant plastic, this box keeps its form during transportation and protects contents. The top closes securely with a unique locking system. Imprinting available, see page 7.



Size: 7-1/2" x 5-1/2" x 4" (190.5 x 139.7 x 101.6 mm)

Imprint Size: 5" x 1-1/4" (127 x 31.7 mm)

Weight: 5-1/2 oz.

#9084990Light Blue

#9084960Desert Flame

#9084980Gray

#9084950Dark Blue

#9084930Beige

#9085010Light Green

Plastic Mailer - Medium

The medium plastic mailer is ideal for transportation of crown and bridge labs along with various other dental items. The chemical-tolerant plastic mailer can withstand shipment transits while keeping valuable dental work safe inside. The box's lid snaps on securely while the entire box only weighs in at 5 ounces. Imprinting available, see page 7.



Size: 6" x 3-3/4" x 3-1/4" (152.4 x 82.5 mm)
 Imprint Size: 5" x 1-1/4" (127 x 31.7 mm)
 Weight: 5 oz.

- | | |
|---------------------------|-----------------------------|
| #9567886 Light Blue | #9567750 Desert Flame |
| #9568010 Yellow | #9567847 Gray |
| #9567600 Beige | #9567717 Dark Blue |
| #9567980 White | |

Plastic Mailer Replacement Parts

Do you need replacement parts for our plastic mailer box? Don't worry, we have you covered.

	Bottoms Only	Tops Only	Inserts Only
Beige	#9085070	#9085090	#9085080
Dark Blue	#9085180	#9085200	#9085190
Desert Flame	#9085210	#9085230	#9085220
Gray	#9085310	#9085330	#9085320
Light Blue	#9085350	#9085370	#9085360
White	#9085610	#9085630	#9085620
Yellow	#9085670	#9085690	#9085680

Shipping Boxes


Get this inexpensive cardboard container for all your dental laboratory's shipping needs. The sturdy 5-1/4" x 5-1/4" x 5" (133.3 x 133.3 x 127 mm) box is ideal for a wide variety of appliances for dental labs and dental practices alike. The boxes are shipped flat and are easily assembled. There are 100 boxes per package.



#9090690 Shipping Boxes

Shipping Containers

These small, protective containers are an excellent way to protect valuable models, dentures and more. The heavy-duty plastic containers provide clear viewing of the internal contents, are prepackaged with protective foam fillers, have a solid locking mechanism to hold the contents. Packaged 100 containers per case.

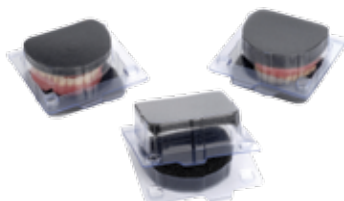


- | | |
|----------------|--|
| #9091180 | Small, 4" x 3-3/4" x 1-3/4" (102 x 95 x 44 mm) |
| #9091170 | Medium 4" x 4" x 2-3/8" (102 x 102 x 60 mm) |
| #9091160 | Large, 4" x 4" x 3-1/8" (102 x 102 x 79 mm) |

STORAGE BOXES

C&B Boxes - Flexible

These 1-inch boxes are made of flexible polypropylene material that will not break. The box is ideal for dental laboratories, which ship crown and bridge work to dentists. The boxes have a positive lock-in mechanism and resist cave-ins to safeguard contents.

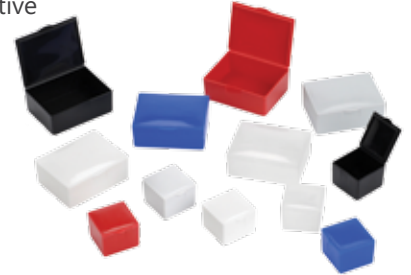

1" (25.4 mm) Flexible Plastic Box - 1000/Pkg
 Top and bottom connect with flexible hinge
 Size: 1" x 1" x 3/4" (25.4 x 25.4 x 19 mm)
 5/8" (16 mm) bottom, 1/8" (3.1 mm) top
 Imprint Size: 3/4" x 3/4" (19 x 19 mm)

- | | |
|---------------------------|----------------------|
| #9590970 Dark Blue | #9590990 Gray |
| #9591040 White | #9590940 Black |
| #9590960 Cherry Red | #9590950 Clear |

1" x 3/8" (25.4 x 9.5 mm) Foam Filler - 1000/Pkg
 Recommend 1 filler per box

- | | |
|---------------------|-------------------------|
| #9580390 Blue | #9580420 Charcoal |
|---------------------|-------------------------|

2" (50.8 mm) Flexible Plastic Box - 500/Pkg
 Top and bottom connect with flexible hinge
 Size: 2-1/8" x 1-5/8" x 3/4" (53.9 x 41.2 x 19 mm)
 5/8" (15.8 mm) bottom, 1/8" (3.1 mm) top
 Imprint Size: 1-3/4" x 1-1/4" (44.4 x 31.7 mm)

- | | |
|---------------------------|----------------------|
| #9590810 Dark Blue | #9590830 Gray |
| #9590880 White | #9590780 Black |
| #9590800 Cherry Red | #9590790 Clear |

2" x 3/4" (50.8 x 19 mm) Foam Filler - 1000/Pkg
 Recommend 1 filler per box

- | | |
|---------------------|-------------------------|
| #9579920 Blue | #9579960 Charcoal |
|---------------------|-------------------------|

C&B Flex with Filler

- | | |
|----------------|-------------------|
| #9571090 | Pack 1" (25.4 mm) |
| #9569640 | Pack 2" (50.8 mm) |

Custom Imprinting

We also provide personal imprinting for these products. On a 1" in size box there is room for a name and phone number. On a 2" size box there's enough room for the name, address and phone number. Ortho and dentures boxes can accommodate 4 or 5 lines of text. Logos can also be printed, but you must mail or email a black and white original version of the logo. We have a variety of print colors, although, we recommend gold or black. We also can accommodate various fonts. For information on custom printing call 800-328-1276.

DENTAL PACKAGING

C&B Boxes - Rigid

These attractive tooth boxes are available in a wide variety of colors. For optimal protection use two fillers to protect the rigid box's contents. Available either plain or custom imprinting. These are a great way to package valuable and important dental appliances.



1" (25.4 mm) Rigid Plastic Box

1,000/Pkg

Hinged snap-type lid

Size: 1" x 1" x 3/4" (25.4 x 25.4 x 19 mm)

1/2" (12.7 mm) bottom, 1/4" (6.3 mm) top

Imprint Size: 3/4" x 3/4" (19 x 19 mm)

- | | |
|-------------------------------|-----------------------------|
| #9571905 Maroon w/clear | #9571245 Pink w/clear |
| #9571845 Green w/clear | #9572175 Gray w/clear |
| #9571447 Black w/clear | #9571955 All Clear |
| #9571445 Blue w/clear | #9571745 All White |
| #9571625 Red w/clear | #9571485 All Blue |
| #9571705 White w/clear | #9571585 All Black |
| #9571785 Yellow w/clear | |

1" x 3/8" (25.4 x 9.5 mm) Foam Filler - 1,000/Pkg

Recommend 2 fillers per box

- | | |
|-------------------------|----------------------|
| #9570905 Blue | #9571320 Red |
| #9571065 Charcoal | #9571100 Green |
| #9571150 Purple | |

2" (50.8 mm) Rigid Plastic Box - 500/Pkg

Hinged snap-type lid

Size: 2-1/8" x 1-5/8" x 3/4" (53.9 x 41.2 x 19 mm)

1/2" (12.7 mm) bottom, 1/4" (6.3 mm) top

Imprint Size: 1-3/4" x 1-1/4" (44.4 x 31.7 mm)

- | | |
|-------------------------------|-----------------------------|
| #9570366 Maroon w/clear | #9570655 Gray w/clear |
| #9570270 Green w/clear | #9570395 All Clear |
| #9570005 Black w/clear | #9570196 All White |
| #9569906 Blue w/clear | #9569910 All Blue |
| #9570085 Red w/clear | #9570035 All Black |
| #9570198 White w/clear | |

2" x 3/8" (50.8 x 9.5 mm) Foam Filler - 1,000/Pkg

Recommend 2 fillers per box

- | | |
|-------------------------|----------------------|
| #9569470 Blue | #9569760 Red |
| #9569630 Charcoal | #9569650 Green |
| #9569700 Purple | |

C&B with Filler

- | | |
|---------------------------------|-------------------------------|
| #9571090 Pack 1" (25.4 mm) | #9569640 ...Pack 2" (50.8 mm) |
|---------------------------------|-------------------------------|

Heavy Duty Strap Boxes

This is an ideal box for a mouthguard with an attached helmet strap. The cases are made of high-density polypropylene with a hinged lid. A slit on the front of the box allows for the mouthguard strap to come out of the box. We provide personal imprinting for this product as well.



- | |
|-----------------------------|
| #9575615Black, 12/Pkg |
| #9575635Gray, 12/Pkg |

Denture Storage Boxes

Made of high density polypropylene, these denture storage boxes make an excellent container for patients with dentures or other dental work. The boxes are 1-3/4 inches (44.4 mm) deep, which is able to fit any size denture. The box is strong, protective and lightweight. Imprinting available, see page 7. 12/Pkg.



- | | |
|-----------------------------|-------------------------------|
| #9576540.....Light Blue | New Age Colors |
| #9576690.....White | #9576541.....New Age Lime |
| #9576360.....Dusty Rose | #9576544.....New Age Lavender |
| #9576320.....Dark Blue | #9576542.....New Age Orange |
| #9576730.....Yellow | #9576545.....New Age Pink |
| #9576600.....Pink | #9576543.....New Age Aqua |
| #9576420.....French Vanilla | #9576546.....New Age Purple |
| #9576210.....Assorted | |

High Gloss Ortho Boxes

High Gloss Ortho Boxes are perfect for storing mouthguards, retainers, bleaching trays and splints. Made of flexible plastic, the box has a hinge design and secure latching system to ensure maximum protection. The box is 3/4-inch (19 mm) deep. Available in packages of 12. Imprinting available, see page 7.



- | | |
|----------------------------------|------------------------|
| #9575185Assorted Marble | #9575155White |
| #9575128Marble Blue/White | #9575165Yellow |
| #9575129Marble Pink/White | #9575125Green |
| #9575127Marble Black/White | #9575190Assorted |
| #9575115Light Blue | #9574460Beige |
| #9575135Dark Blue | #9575105Red |
| New Age Colors | |
| #9575191Lime | #9575193Aqua |
| #9575195Pink | #9575194Lavender |
| #9575192Orange | #9575197Assorted |
| #9575196Purple | |

Lab Work Pans

Lab work pans are a great way to organize your laboratory. Use a different color tray for each day of the week or try using a different color tray for in going and outgoing work. Size: 7" x 5" x 2-1/2" (177.8 x 127 x 63.5 mm). 12/Pkg.



- | | |
|---|---------------------------------|
| #7000372 Red | #7000516 Dark Green |
| #7000371 Blue | #7000378 Gray |
| #7000375 Green | #7000930 New Age Lime |
| #7000373 Yellow | #7000931 New Age Orange |
| #7000374 White | #7000932 New Age Aqua |
| #7000370 Pink | #7000933 New Age Lavender |
| #7000377 Beige | #7000934 New Age Hot Pink |
| #7000518 Mauve | #7000935 New Age Purple |
| #1060085 Work Pan Liners, 144/Box | |



BARRIER PRODUCTS 10
DISPOSABLES 11
EVACUATION 11
FACE MASKS & SHIELDS 12

NEEDLES13
PROCEDURE ACCESSORIES13
STERILIZATION PRODUCTS14

DISPOSABLES/INFECTION CONTROL

BARRIER PRODUCTS

Barrier Film



Our Barrier Film offers a finger-lift edge, which allows for quick and easy placement and removal. This is ideal to use in dental settings where sanitation is the utmost concern. This barrier film is easy to grasp for quick removal and features a light tack adhesive that easily attaches to surfaces without leaving a sticky residue. Film comes in 4" x 6" (101.6 x 152.4 mm) sheets.



#10-05010.....Clear, 1,200/Box

#10-05110.....Blue, 1,200/Box

Chair Sleeves Covers



These clear chair sleeve covers will fit any dental chair out there. The sleeve covers reduce the risk of cross-contamination from airborne aerosols and safeguard a longer lasting chair. They're easy to put on or take off to save time. The half chair sleeve is 27-1/2" x 24" (698.5 x 609.6 mm) while the full chair sleeve is 29" x 80" (736.6 x 2,032 mm).



#10-05610.....Half Chair Sleeve, Clear, 225/Box

#10-05710.....Full Chair Sleeve, Clear, 125/Box

Digital X-Ray Sensor Sleeves



Protect your valuable X-Ray sensors with these durable plastic sleeves at an affordable price. The finger lift edge allows for quick and easy placement and removal. This is perfect to make sure hard-to-reach areas on the sensor remain clean. Also protect the X-Ray sensor from splatter or other hazardous materials. 500/Box.



#10-06010.....Clear, 1-3/8" x 8-1/4" (34.9 x 209.6 mm)

#10-05910.....Clear, 1-5/8" x 8-1/2" (41.2 x 215.9 mm)

Handpiece Sleeves



These Handpiece Barrier Sleeves fit over most handpiece dental instruments and reduces the risk of cross-contamination from airborne aerosols. Made of high quality clear plastic, the sleeve is designed to cover low-speed and high-speed handpieces. It's easy to use and discard after use. 500/Box.



#10-06410.....High Speed, Clear, 1" x 8" (25.4 x 203.2 mm)

#10-06510.....Low Speed, Clear, 1-1/2" x 8" (38.1 x 203.2 mm)

Headrest Covers



Keep your dental chairs clean with our Headrest Covers, which are made of durable waterproof clear plastic. These covers are disposable and a great way to prevent contamination between patients and ensure a longer chair life. The cover is designed to fit over the top of the headrest and is easy to put on or take off the chair. 250/Box.



#10-05410..... Clear, 9-1/2" x 11" x 2" (241.3 x 279.4 x 50.8 mm)

#10-05510..... Clear, 9-1/2" x 14" x 2" (241.3 x 355.6 x 50.8 mm)

Light Handle Sleeves



These clear plastic sleeves fit over any T-style handles on overhead lights. The plastic covering reduces the risk of cross-contamination from airborne aerosols. The plastic material slides easily over the handle and provides a good fit. The sleeve is 4" x 5-3/4" (101.6 x 146 mm).



#10-06210.....Light Handle Sleeves, 500/Box

Syringe Sleeve Covers



These disposable sleeves are made to cover 3-way syringes, HVE and saliva ejector valves. The convenient pre-cut opening allows the user to simply slide the syringe tip through the end of the bag. This item reduces the risk of cross-contamination from airborne aerosols. The Syringe Sleeve is 2-1/2" x 10" (63.5 x 254 mm).



#10-05210.....Blue, 500/Box

#10-05310.....Clear, 500/Box

Tray Sleeves



There's no more need for paper tray covers thanks to these disposable tray sleeves that cover the entire ADEC/ Size B – Ritter tray. The cover is easy to put on and take off. These sleeves eliminate contamination and make clean up effortless. Overlapping folds keep the instrument trays from falling out. The Tray Sleeve is 10-1/2" x 14" (266.7 x 355.6 mm).



#10-05810.....Clear Size B, 500/Box

X-Ray Sleeves



These universal clear plastic sleeves fit over most X-Ray machines and reduce the risk of cross-contamination from airborne aerosols. The plastic will not adversely affect the X-Ray performance and will keep the valuable X-Ray machine in a good-working condition. This is perfect to make sure hard-to-reach areas on the machine remain clean. The X-Ray Sleeve is 15" x 26" (381 x 660.4 mm).



#10-06310.....X-Ray Sleeves, 250/Box

DISPOSABLES

Cotton Rolls



These medium non-sterile cotton rolls are made of high quality 100% cotton. They are soft, pliable with minimal lint. The highly absorbent rolls maintain shape when wet and will not adhere to mucous membranes. 2,000/Box.



#20-00110.....#2, 1-1/2" x 3/8" (38.1 x 9.5 mm)

Cotton Tipped Applicators



These quality applicators are versatile and add ease of applications. They are made of wooden dowels with a tight 100% cotton bud that minimizes lint and prevents shredding. The soft pliable cotton doesn't damage mucosa and the wooden stem is strong.



#20-00210.....3" (76.2 mm), 1,000/Box

#20-00310.....6" (152.4 mm), 1,000/Box

Non-Woven Sponges



These Non-Woven Sponges are perfect for general use. The 4-ply, non-sterile sponge is soft, smooth, strong and virtually lint free. The standard sponges are 30-gram weight rayon/polyester blend while the plus size sponges are made from 35-gram weight rayon/polyester blend. The lighter weights provide good absorbency with little adhesion to wounds. These sponges are ideal for sustained patient use, disinfecting and general cleaning.



#10-20010.....2" x 2" (50.8 x 50.8 mm), 5,000/Box

#10-20110.....Plus 2" x 2" (50.8 x 50.8 mm), 5,000/Box

#10-20210.....3" x 3" (76.2 x 76.2 mm), 4,000/Box

#10-20310.....4" x 4" (101.6 x 101.6 mm), 2,000/Box

Plastic Cups

These heavy 5-ounce plastic cups are double coated for a high-gloss finish. The outside walls of the cups are embossed to provide sufficient grip. These plastic drinking cups come in various colors to fit your dental office decor.



#10-01410.....Blue, 5 oz., 1000/Box

#10-01510.....Green, 5 oz., 1000/Box

#10-02810.....Lavender, 5 oz., 1000/Box

#10-01610.....Rose, 5 oz., 1000/Box

#10-02610.....White, 5 oz., 1000/Box

EVACUATION PRODUCTS

High Volume Evacuator Tips

Our High Volume Evacuator Tips have a vented and non-vented combination while featuring a softer end for patient comfort. Engineered and tested for durability, stiffness and function, these evacuator tips fit all standard HVE valves.



#7000310D-1, 20 bags of 50

#7000311D-1, 10 bags of 100

#7000305D-2, 20 bags of 50

#7000306D-2, 10 bags of 100

#7000312D-3, 20 bags of 50

#7000313D-3, 10 bags of 100

#7000300R-1, each



High Volume Evacuator Tips - Scented

Give your patients a refreshing smell of mint with our scented Mint HVEs. Our High Volume Evacuator Tips have a vented and non-vented combination while featuring a softer end for patient comfort. Engineered and tested for durability, stiffness and function, these evacuator tips fit all standard HVE valves.



#7000811D3, Mint Scented, 20 bags of 50



DISPOSABLES/INFECTION CONTROL

Saliva Ejectors

Our Saliva Ejectors are soft and pliable to uniquely contour each patient's mouth and hold shape. The tips are soft and bonded to the tube for maximum patient safety. These ejectors provide optimal suction without aspirating tissue and ensure non-clogging operation.



- #12-00110.....Clear/Blue Tip, 100/Pkg
- #12-00210.....White/White Tip, 100/Pkg



Surgical Aspirator Tips

These Surgical Aspirator Tips have smooth edges for maximum patient comfort. Made of durable, high quality polyethylene, the aspirator tips have a 30-degree angle and fit all high-volume evacuation systems. This item is also autoclavable up to 135 degrees Celsius and may be used for cheek retraction since the tips do not bend.



- #12-01010.....Green, 1/4" (6.3 mm) Orifice, 25/Pkg
- #12-01110.....White, 1/8" (3.1 mm) Orifice, 25/Pkg
- #12-01210.....Blue, 1/6" (4.2 mm) Orifice, 25/Pkg

FACE MASKS & SHIELDS

Procedural Ear Loop Face Masks

These Ear Loop Masks feature slip-on convenience and provide fluid resistance. The wearer reduces potential exposure to blood and body fluids when wearing these masks. The mask is made of lightweight, 3-ply material that is fluid resistant with a Bacterial Filtration Efficiency (BFE) and Particle Filtration Efficiency (PFE) greater than 98%. The soft, flexible nosepiece and ear loop allows the wearer to slip on and remove the mask quickly and easily. The folds in the mask expand for full coverage from top of the nose to underneath the chin. The malleable nosepiece offers a more custom fit.



- #10-18010.....Blue, 50/Box
- #10-18110.....Pink, 50/Box
- #10-18210.....Green, 50/Box



Optima Procedural Face Mask

This lightweight, 3-ply material is fluid resistant with a Bacterial Filtration Efficiency (BFE) and Particle Filtration Efficiency (PFE) greater than 98%. The mask has a soft, flexible nosepiece and is ultrasonically sealed ear loops that offer great comfort and durability.



- #10-18310.....Blue, 50/Box
- #10-18410.....Green, 50/Box
- #10-18610.....Pink, 50/Box



Procedural Ear Loop with Visor Masks

Our fog-free procedure mask with wraparound splash-guard visor gives optimum protection during medical and dental procedures. Both inner and outer layers of the mask are fluid resistant while the cloth part of the mask is made of lightweight, 3-ply material. This mask has a Bacterial Filtration Efficiency (BFE) and Particle Filtration Efficiency (PFE) greater than 98%. The malleable nosepiece offers a more comfortable custom fit. A clear plastic sheet shields the eyes but enables you to still have a full field of vision. Overall, it's comfortable to wear without being restrictive.



- #10-19010.....Blue, 50/Box



Surgical Molded Cone Masks

These splash repellent barrier masks have a comfortable elastic headband to easily take on and off. For added comfort, the mask has a flared edge to prevent irritation along with a soft, flexible nosepiece. The mask is fluid resistant and has a Bacterial Filtration Efficiency (BFE) greater than 98%.



- #10-17010.....Blue, 50/Box
- #10-17110.....Pink, 50/Box



Surgical Tie-on Masks

This Surgical Tie-On Face Mask provides a custom fit around the head along with optimal protection from fluids. The mask is made of lightweight, 3-ply material and has a fluid resistant outer and inner layer. This clean and tidy item is durable and allows for greater comfort and secure fit. It has a Bacterial Filtration Efficiency (BFE) and Particle Filtration Efficiency (PFE) greater than 98%.



- #10-18510.....Blue, 50/Box



HB Anti-Fog Face Shield

When working on a patient or in the lab, often times infectious, hazardous or undesirable substances will splatter. The HB Anti-Fog Shield is a lightweight, disposable face shield that provides protection of eyes and face from accidental exposure to hazardous substances. Keystone's HB Anti-Fog Shield offers OSHA-level protection against splash of bodily fluids in the face for medical, dental and emergency medical healthcare workers.



#1983000Full Face Shield, 25/Pkg

NEEDLES

Needles

We offer sterile Prehma Disposable Dental Needles that come in short and long sizes. The needle is a sharp tri-bevel point for maximum comfort. The needles have a universal screw-in system that works with an inch-type of metric-type syringes. The plastic hubs are pre-threaded for an easy, straight attachment into the syringe. The needle is siliconized for smooth motion during injection.




- #03-20010.....27g Short Plastic Hub, 100/Box
- #03-20110.....27g Long Plastic Hub, 100/Box
- #03-20210.....30g Short Plastic Hub, 100/Box
- #03-20310.....30g X-Short Plastic Hub, 100/Box

PROCEDURE ACCESSORIES

Air/Water Syringe Tips

These Air/Water Syringe Tips are designed to replace a practice's existing metal autoclavable tips. There is no adapter required. The tips have an inner core water chamber that seats into the syringe, eliminating water contamination and delivers superior air/water spray. No sharp edges which reduces o-ring wear. Made from FDA - approved plastic.

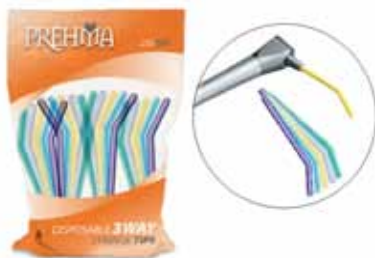

- #10-04610.....Blue, 250/Bag
- #10-04510.....White, 250/Bag
- #10-04710.....Assorted, 250/Bag

Micro Applicators

These Micro Applicators are designed for precise placement and easy application of dental materials in a limited area. The stronger bending portion improves control and the stiffer head provides the ability to scrub. The non-absorbent, non-linting fibers suspend solutions as small as 1/8" (3.1 mm) of a drop with no dripping or spilling. This item is suitable for use with sealants, bonding agents, etchants, hemostatic solutions, cavity liners and disclosing solutions.



- #10-13013.....X-Small, 100 Applicators/Tube, 4/Box
- #10-13113.....Small, 100 Applicators/Tube, 4/Box
- #10-13213.....Medium, 100 Applicators/Tube, 4/Box **CE**

Pre-bent Dispensing Tips

These dispensing tips are pre-bent to the proper angle for placing etchants, sealants, cements and other flowable materials. With blunt ends, the dispensing tip has a universal design. We offer a high-quality, economical alternative to other supplier products. Works great for dispensing etchants, pit and fissure sealants, flowable composites and resins.



- #07-04477.....Pink, 18 gauge, 100/Pkg
- #07-04577.....Black, 19 gauge, 100/Pkg
- #07-04677.....Yellow, 20 gauge, 100/Pkg
- #07-04777.....Gray, 22 gauge, 100/Pkg
- #07-03077.....Blue, 25 gauge, 100/Pkg
- #07-04077.....Pink, 18 gauge, 25/Pkg
- #07-04177.....Black, 19 gauge, 25/Pkg
- #07-04277.....Yellow, 20 gauge, 25/Pkg
- #07-04377.....Gray, 22 gauge, 25/Pkg
- #07-02977.....Blue, 25 gauge, 25/Pkg

CE

Pro-Guard I Bibs

These large 13" x 18" (330.2 x 457.2 mm) dental bibs have unique qualities that will make your job easier. Our technique secures the layers of tissue and poly to prevent separation of the layers. The 2-ply tissue added with single-ply poly adds maximum absorption and patient protection. The horizontal embossing has reinforced edges for added strength and durability.



- #10-00110.....Blue, 500/Box
- #10-00210.....Green, 500/Box
- #10-00310.....Rose, 500/Box
- #10-00410.....White, 500/Box
- #10-00510.....Lavender, 500/Box

DISPOSABLES/INFECTION CONTROL

Pro-Guard II Bibs

These large 13" x 18" (330.2 x 457.2 mm) dental bibs have unique qualities that will make your job easier. Our technique secures the layers of tissue and poly to prevent separation of the layers. The 2-ply tissue added with single-ply poly adds maximum absorption and patient protection. The horizontal embossing has reinforced edges for added strength and durability.



- #10-00610.....Blue, 500/Box
- #10-00710.....Green, 500/Box
- #10-00810.....Rose, 500/Box
- #10-00910.....White, 500/Box
- #10-01110.....Lavender, 500/Box

Overgloves

These Overgloves are a great alternative to the more expensive latex gloves. They can be used with procedures that do not require tactile sensitivity such as opening incoming cases, disinfecting casework and general surface cleaning.



- #9045660Large, 1,000/Box
- #9045670Medium, 1,000/Box
- #9045680Small, 1,000/Box

Protective Eyewear

We offer both clear protective safety glasses and UV Protective Safety Glasses. Both are made of polycarbonate single piece lens construction. The unique spherical design delivers a distortion-free view even in peripheral vision areas. These glasses are secure and have a comfortable fit even when wearing over prescription glasses. These are an economical choice while still providing needed safety precautions. The UV glasses filter more than 99% of UV and dangerous blue light (up to 540nm).



- #36-00110.....Clear Protective Safety Glasses, 10/Box
- #36-00210.....U.V. Protective Safety Glasses, 10/Box

STERILIZATION PRODUCTS

Sterilization Pouches



These Sterilization Pouches are made with a built-in, lead-free internal indicator. The internal indicator ensures that the correct sterilization temperature is reached inside the pouch, ensuring sterile instruments and tools. This eliminates the need for internal indicator strip inside the pouch. The external indicator will change color when the optimal sterilization temperature is reached. 200/Box.



- #10-50010.....2-1/4" x 4" (57.1 x 101.6 mm)
- #10-50110.....3-1/4" x 5-1/2" (82.5 x 139.7 mm)
- #10-50210.....2-3/4" x 10" (69.8 x 254 mm)
- #10-50310.....3-1/2" x 10" (88.9 x 254 mm)
- #10-50410.....5-1/4" x 10" (133.3 x 254 mm)
- #10-50510.....7-1/2" x 13" (190.5 x 330.2 mm)

Dual Sterilization Pouches



These Dual Sterilization Pouches are manufactured with lead-free, built-in dual internal and external indicators to ensure the correct sterilization was reached in the autoclave chamber, as well as inside the instrument compartment. The built-in internal process indicator arrows change color with sterilization heat and are visible and compatible with steam and ETO agents. The blue-tinted transparent film proves easy instrument viewing and identification of punctures and tears in the film. The self-seal adhesive strip easily and securely seals pouch, providing a reliable seal until the pouch is opened. The corner tack seals to prevent corners from curling after sterilization. 200/Box.



- #10-51410.....2-1/4" x 4" (57.1 x 101.6 mm)
- #10-51510.....3-1/4" x 5-1/2" (82.5 x 139.7 mm)
- #10-51610.....2-3/4" x 10" (69.8 x 254 mm)
- #10-51710.....3-1/2" x 10" (88.9 x 254 mm)
- #10-51810.....5-1/4" x 10" (133.3 x 254 mm)
- #10-51910.....7-1/2" x 13" (190.5 x 330.2 mm)



ABRASIVES	16	EQUIPMENT	45
ACRYLICS.....	26	INSTRUMENTS	49
ARTICULATORS	30	LAB SAFETY	51
BRUSHES & BUFFS	33	LAMP/MAGNIFIER.....	52
BURNERS & TORCHES	35	MISCELLANEOUS	52
CASTING.....	36	POLISHING	53
DEBUBLIZERS	38	PORCELAIN ACCESSORIES	54
DENTURE ACCESSORIES.....	38	ROTARY	58
DIE & MODEL FABRICATION	42	WAXES	64



LABORATORY

ABRASIVES

Green Mounted Stones

Made from finest quality silicon carbide grit, these Green Mounted Stones are fast and cool cutting. This economical choice has excellent durability and 14 popular shapes. These Green Mounted Stones are guaranteed to meet your porcelain, composite, gold and silver finishing requirements. Sold by the dozen.



- #1631205I-3.....3/32" Shank (23.8 mm)
- #1631194I-5.....3/32" Shank (23.8 mm)
- #1631210I-6.....3/32" Shank (23.8 mm)
- #1631206I-7.....3/32" Shank (23.8 mm)
- #1631195I-9.....3/32" Shank (23.8 mm)
- #1631203F2.....3/32" Shank (23.8 mm)
- #1631198K7.....3/32" Shank (23.8 mm)
- #1631199P2.....3/32" Shank (23.8 mm)
- #1631208T1.....3/32" Shank (23.8 mm)
- #1631196T4.....3/32" Shank (23.8 mm)
- #1631200W3.....3/32" Shank (23.8 mm)
- #1631201W6.....3/32" Shank (23.8 mm)

Red Mounted Points

For years, these have been the standard point for chrome processors. Available in handpiece only. Sold by the dozen.



- #1631081R1.....3/32" Shank (23.8 mm)
- #1631091R2.....3/32" Shank (23.8 mm)
- #1631111R5.....3/32" Shank (23.8 mm)
- #1631191R76.....3/32" Shank (23.8 mm)
- #1631101R4.....3/32" Shank (23.8 mm)
- #1631141R8.....3/32" Shank (23.8 mm)

Blue Mounted Points

Blue Mounted Points are used for finishing chrome alloys and non-precious metals. The points are made of aluminum oxide and hold their shape while remaining fast cutting and long lasting. 100/Box.



- #1631070Large Barrel
- #1631072Small Barrel
- #1631074Inverted Cone
- #1631076Small Taper

Pink & White Mounted Stones

The Pink and White Mounted Points are made of non-contaminating aluminum oxide. The pink mount is used for precious ceramic metals while the white point is used on non-precious ceramic metals. 72/Box.



- #1631225#20 Pink.....3/32" Shank (23.8 mm)
- #1631235#59 Pink.....3/32" Shank (23.8 mm)
- #1631240#39 Pink.....3/32" Shank (23.8 mm)
- #1631230#42 Pink.....3/32" Shank (23.8 mm)
- #1631245#30 Pink.....3/32" Shank (23.8 mm)
- #1631250#11 Pink.....3/32" Shank (23.8 mm)
- #1631265#20 White.....3/32" Shank (23.8 mm)
- #1631275#59 White.....3/32" Shank (23.8 mm)
- #1631280#39 White.....3/32" Shank (23.8 mm)
- #1631270#42 White.....3/32" Shank (23.8 mm)
- #1631285#30 White.....3/32" Shank (23.8 mm)
- #1631290#11 White.....3/32" Shank (23.8 mm)

Superior Rubber – Polishing Wheels and Clasp Polishers



DESCRIPTION	Polishing Wheel 7/8"x 1/8" (22.2 x 3.1 mm)	Clasp Polisher 15/16" x 1/4" (23.8 x 6.3 mm)
SUPERIOR REGULAR – An industry standard for years. Pure fine silicon carbide abrasive in a synthetic rubber binder combines an excellent finish on chrome with long wear. Also popular for polishing gold. Dark grey color. 240 grit. 100/Box.	#1900550	#1640010
SUPERIOR HARD – Same quality as SUPERIOR REGULAR only harder. Provides exceptionally long wear on chrome. Dark gray in color. 240 grit. 100/Box.	#1900560	#1640020
SUPERIOR MEDIUM SOFT – Pure medium grit silicon carbide abrasive in a flexible rubber binder for fast scratch-free finishing of chrome. Light grey in color. 150 grit. 100/Box.	#1900540	—
SUPERIOR BLUE REFLEX – Softer and more flexible than the “Blue Gloss”. This wheel is made with a new rubber binder containing a highly friable pure fine silicon carbide abrasive. Made for both chrome and noble metals. 220 grit. 100/Box.	#1900795	#1640075
SUPERIOR BLUE – The popular “Blue Gloss” wheel improved with new polishing additives. Contains pure fine treated silicon carbide abrasive in a unique cushioned binder. Provides a lustrous finish on chrome, semi-precious and precious metals. 220 grit. 100/Box.	#1900810	#1640070
SUPERIOR GREEN – An “All Purpose” polishing wheel for all metals. Pure medium fine silicon carbide abrasive in a premium flexible rubber bond provides fast high quality finishing. Quickly becoming an industry favorite. 180 grit. 100/Box.	#1900805	#1640060
SUPERIOR BLACK – Developed for fast chrome finishing, this harder but still flexible premium rubber bond with pure medium coarse silicon carbide, combines excellent finish and wear qualities. Very popular with chrome labs. 150 grit. 100/Box.	#1900800	#1640050
SUPERIOR RED – A specially formulated resilient binder combined with carefully graded pure fine silicon carbide abrasive provides a high quality lustrous finish on precious and non-precious metals. More flexible and not too aggressive, it is a favorite for finishing precious alloys. 240 grit. 100/Box.	#1900815	#1640080
SUPERIOR HARD RED – Same quality as SUPERIOR RED only harder. Provides exceptionally long wear on precious and non-precious alloys. 240 grit. 100/Box.	#1900816	#1640081
SUPERIOR WHITE – Non-contaminating for use on ceramic gold and porcelain. Specially formulated with pure fine aluminum oxide abrasive in a cushioned binder to provide a polished finish to porcelain. 220 grit. 100/Box.	#1900820	—
SUPERIOR GREEN UTILITY – Pure coarse silicon carbide abrasive in a highly flexible binder for fast, effective polishing on gold, porcelain and acrylic. 90 grit. 100/Box.	#1900580	—

LABORATORY

Knife Edge Rubber Wheels

Knife Edge Rubber Wheels are made of aluminum carbide and come in several different styles for different purposes.



Blue - Fine for polishing non-precious, semi-precious and chrome cobalt. Silicon carbide. 100/Box.

#19008355/8" (15.8 mm)

#19008857/8" (22.2 mm)

White - Fine and non-contaminating for extra fine polishing extremely flexible. Aluminum oxide. 100/Box.

#19008455/8" (15.8 mm)

#19008957/8" (22.2 mm)

Green - Medium fine "all purpose" for polishing metal. Silicon carbide. 100/Box.

#19008305/8" (15.8 mm)

#19008807/8" (22.2 mm)

Red - Fine and extremely flexible. Silicon carbide. 100/Box.

#19008405/8" (15.8 mm)

#19009907/8" (22.2 mm)

Black - Medium coarse for finishing and polishing chrome cobalt metals. Silicon carbide. 100/Box.

#19008255/8" (15.8 mm)

#19008757/8" (22.2 mm)

Bullet Shaped Clasp Polisher

This Bullet Shaped Clasp Polisher is used for polishing. Dark gray in color. Sold 100/Box.



#1640030Coarse grit

Deluxe Silicone Points & Wheels


Sold 100/Box.

Black.....for all metals. Medium grit.

White.....for all metals, amalgam, porcelain & composites. Coarse grit.


Bluefor final polishing of porcelain or metals. Fine grit.

Pinkfor porcelain. X-fine grit.

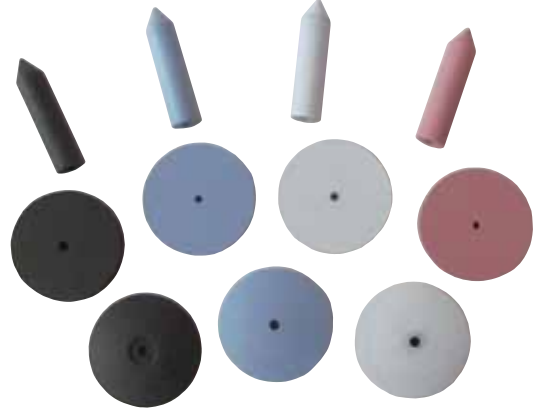

100/Box	Wheels Square Edge	Wheels Knife Edge	Points
Black	#1909000	#1909005	#1909010
White	#1909015	#1909020	#1909025
Blue	#1909030	#1909035	#1909040
Pink	#1909045	-	#1909055

Interproximal Rubber Wheel

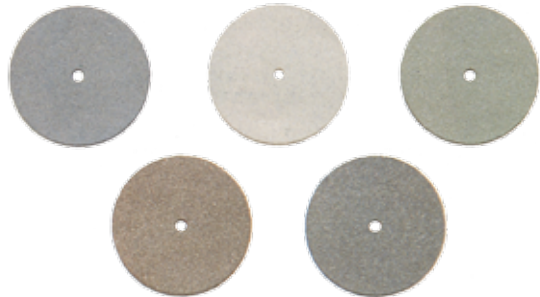

White - Fine and non-contaminating for extra fine polishing. Aluminum oxide. 25/Pkg.

#19008707/8" x 0.040 (22.2 x 1.01 mm)

Green - Medium fine "all purpose" for polishing metal. Silicon carbide. 25/Pkg.

#19008557/8" x 0.040 (22.2 x 1.01 mm)

Red - Fine and extremely flexible. Silicon carbide. 25/Pkg.

#19008657/8" x 0.040 (22.2 x 1.01 mm)

Black - Medium coarse for finishing and polishing chrome cobalt metals. Silicon carbide. 25/Pkg.


#19008507/8" x 0.040 (22.2 x 1.01 mm)

Blue - Fine for polishing non-precious, semi-precious and chrome cobalt. Silicon carbide. 25/Pkg.

#19008607/8" x 0.040 (22.2 x 1.01 mm)

Knife Edge Pumice Wheels

This Knife Edge Pumice Wheel is a flexible impregnated rubber wheel. It is especially suited for porcelain and gold.

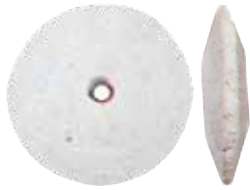

- #19005905/8" dia. (15.8 mm)..... 100/Box
- #19006005/8" dia. (15.8 mm)..... 500/Box

Point & Wheel Truer

This stone facilitates speedy trueing of points and wheels. Perfect for laboratories which need to quickly true abrasives. Quickly and easily reshapes and dresses mounted points. Sold individually.



- #1830010Point & Wheel Truer

Grinding Wheel Dresser

The Grinding Wheel Dresser is made of heavy-duty steal cutters to ensure smooth cutting action on grinding wheels, which are grooved or larded. Makes dressing grinding wheels quick and easy. Replacement cutters are available.



- #1900615Wheel Dresser
- #1900617Wheel Dresser Refill

Roughing Disc

This is the 1-1/4" x .085" (31.8 x 2.2mm) Roughing Disc. It's made of a pure, coarse aluminum oxide abrasive in a heavy-duty, non-contaminating body. The disc provides exceptionally fast metal removal when cutting or grinding. The disc runs free and cool cutting on all metal alloys. 50/Box.



- #13004251-1/4" x .085" (31.8 x 2.2 mm)

Red Engine Wheels

These Red Engine Wheels are made of aluminum oxide grit that performs a superior job on chrome or gold. The wheels come in square edge and knife edge in 7/8" diameter or 5/8" diameters. 12/Pkg.



- #19003807/8" x 1/8" S.E. (22.2 x 3.1 mm)
- #19004307/8" K.E. (22.2 mm)
- #19004205/8" K.E. (15.8 mm)

Green Engine Wheels

These Green Engine Wheels are made of silicon carbide grit for gold and porcelain work. The wheels come in square edge and knife edge in 7/8" diameter or 5/8" diameters. 12/Pkg.



- #19005207/8" x 1/8" S.E. (22.2 x 3.1 mm)
- #19005107/8" K.E. (22.2 mm)
- #19004905/8" K.E. (15.8 mm)

Red Flash Finishing Wheels

A non-contaminating, aluminum oxide wheel for rapid reduction of all ceramic metals. Reddish brown in color. Sold 100/Box.



- #19002907/8" x 1/8" (22.2 x 3.1 mm)

LABORATORY

Rubber Acrylic & Baseplate Wheel

The Rubber Acrylic and Baseplate Wheel is a coarse grit, rubber bonded wheel. The wheel is excellent for grinding acrylic at normal lathe speed and for baseplates at low speed.



#19000503" x 3/8" (76 x 9.5 mm)

Red Grinding Wheel

This Red Grinding Wheel is specially manufactured for the fine polishing of all metals. These wheels are sold individually but are long-lasting.



#13001763" x 3/8" x 1/4" (76 x 9.5 x 6.35 mm)

Green Grinding Wheel

These Green Grinding Wheels are specially suited for grinding porcelain teeth. The long-lasting discs will not wear or chip porcelain. The wheels have 1/4" lead reinforced center hole and comes in 120 grit.



#19000403" x 3/8" (76 x 9.5 mm)

#19000303" x 1/4" (76 x 6.35 mm)

#19000102-1/2" x 1/4" (63.5 x 6.35 mm)

White Acrylic Knock-down Wheels

These White Acrylic Knock-down Wheels are developed for trimming base plates, acrylic, and stone models. The aluminum oxide grit facilitates rapid shaping with minimum pressure. The wheels generate little heat and will not clog.



#19000904" x 1/2" (101 x 12.7 mm)

#19000803" x 3/8" (76 x 9.5 mm)

Black Utility Grinding Wheel

The Black Utility Grinding Wheel is excellent for trimming denture base plate materials, stone models and acrylics. The top-grade wheel delivers rapid cutting and will not clog at low speeds. The wheel has a 1/4" lead reinforced center hole. Comes in 46-60 grit.



#19000603" x 1/4" (76 x 6.35 mm)

Pure Blast White Aluminum Oxide

Pure Blast White Aluminum Oxide is used in medical, dental, aircraft and automotive applications. In the cosmetics and skin care industries, White Aluminum Oxide Microdermabrasion Crystals are used in skin exfoliation products. This is a dental-grade abrasive that comes in 50 micron, 240 grit. Overall, it's a 100% dependable cleaning agent.



#101000215 lbs. (6.8 kg)

#101000450 lbs. (22.68 kg)

Red Flash Disc

The Red Flash Disc is the most popular and finest disc available from Keystone Industries. The cut-off wheel quality is strong, fast and free cutting on all chrome, precious and semi-precious alloys. Made of aluminum oxide, the disc can withstand up to 25,000 rpm. 100/Box.



#130043015/16" x 0.025" (23.8 x 0.63 mm)

Ultra-Flex Disc

The Ultra-flex Disc is the thinnest disc available to the crown and bridge craftsman. This disc is two-thirds as thin as and more flexible than the Ultra-thin Disc, supplementing the Ultra-thin where maximum thinness is required. These are excellent for interproximals and margins. Made of silicon carbide, the gray color discs work best between 10,000 and 12,000 rpm. 25/Box.



#13004757/8" x 0.006" (22.2 x 0.15 mm)

Veri-Thin® Disc

These Veri-thin Discs are much thinner than the regular Joe Dandy Discs, but still maintains the strength and fast, cool cutting qualities. These discs, made of silicon carbide, are especially suited for cutting and finishing porcelain and gold. Ideal speeds for the discs are between 10,000 and 12,000 rpm. 50/Box.



#13004507/8" x 0.015" (22.2 x 0.38 mm)

#13004403/4" x 0.015" (19 x 0.38 mm)

Ultra-Thin® Disc

Ultra-thin Abrasive Discs have the exceptional combination of thinness, strength and cutting qualities, which have made them an indispensable industry standard. These abrasives are used for fine cutting and finishing on porcelain interproximals and margins. The pure fine silicon carbide abrasive is a unique formulation that will not discolor porcelain. Best used between 10,000 and 12,000 rpm.



#130047015/16" x 0.025" (23.8 x 0.63 mm)

Dura-Thin® Disc

These Dura-thin Discs are the ultimate, non-contaminating companion to the Ultra-thin Discs. Made of fine aluminum oxide abrasive is specially formulated for fine cutting and finishing of ceramic metals. Ideal speeds for the discs are between 10,000 and 12,000 rpm. 25/Box.



#13004607/8" x 0.010" (22.2 x 0.25 mm)

Cut-off Wheels

These strong Cut-off Wheels are long lasting and fast cutting. The wheels can be used on chrome cobalt and other non-precious metals. 100/Box.



#13006501-3/8" x 0.027" (35 x 0.7 mm)

#13006601" x 0.027" (25 x 0.7 mm)

LABORATORY

Joe Dandy Type Discs

The Joe Dandy Type Discs are made of pure rubber-bonded silicon carbide abrasive in a special binder. They are used for fast, cool cutting and finishing on porcelain, acrylic, gold, alloys and other materials. 100/Box.

- 0.023" (0.6 mm) thick



- #13001801/2" (12.7 mm) ... Flat Double Cutting
- #13002301/2" (12.7 mm) ... Flat Safe-Sided
- #13003801/2" (12.7 mm) ... Cup Inside Cutting
- #13001905/8" (15.8 mm) ... Flat Double Cutting
- #13002405/8" (15.8 mm) ... Flat Safe-Sided
- #13002905/8" (15.8 mm) ... Cup Double Cutting
- #13002003/4" (19 mm) Flat Double Cutting
- #13002503/4" (19 mm) Flat Safe-Sided
- #13003003/4" (19 mm) Cup Double Cutting
- #13003503/4" (19 mm) Cup Outside Cutting
- #13002107/8" (22.2 mm) ... Flat Double Cutting
- #13002607/8" (22.2 mm) ... Flat Safe-Sided
- #13003107/8" (22.2 mm) ... Cup Double Cutting
- #13003607/8" (22.2 mm) ... Cup Outside Cutting
- #13002201" (23.8 mm) Flat Double Cutting
- #13002701" (23.8 mm) Flat Safe-Sided
- #13003201" (23.8 mm) Cup Double Cutting

Keystone Cut-off Wheels

These Cut-off Wheels are strong and durable. Made with pure aluminum oxide provides non-contaminating fast free cutting of all metals. The wheels come in a reddish-brown color and have a max of 25,000 rpm. 100/Box.



- #1900320 Thick, 1-1/4" dia. x 0.062" (31.8 x 1.6 mm)
- #1900310 Medium, 1-1/2" dia. x 0.040" (38.2 x 1.1 mm)
- #1900300 Thin, 1-1/2" dia. x 0.025" (38.2 x 0.63 mm)

Brown High Speed Disc

This Brown High Speed Disc is for those who favor the long-time standard for chrome and gold. Made of aluminum oxide. 100/Box.



- #130017015/16" x 0.025" (0.94 x 0.63 mm)

Roughing Discs

These Roughing Discs are made of a pure, coarse aluminum oxide abrasive in a heavy-duty, non-contaminating bond. The disc provides exceptional fast metal removal when cutting or grinding. The disc runs free and cool cutting on all metal alloys.



- #13006801-1/4" x 0.086" (32 x 2.2 mm)

Porcelain Finishing Discs

These Porcelain Finishing Discs are a fine quality abrasive paper discs perfect for finishing gold and porcelain. The discs are made of silicon carbide-like sandpaper. 500/Box.



- #1300080 Medium, 120 grit3/4" (19 mm)
- #1300140 Coarse, 80 grit7/8" (22.2 mm)
- #1300130 Medium, 120 grit7/8" (22.2 mm)
- #1300120 Fine, 220 grit7/8" (22.2 mm)

Arbor Bands

Keystone Arbor Bands are designed for aggressive trimming and finishing of acrylics, silicone relines and other materials. Arbor band mandrels are made of high quality materials with adjustable diameter rubber heads, designed to firmly hold arbor bands in place.



Quantity	Coarse 60 grit	Medium 80 grit	Fine 120 grit
1/4" (0.63mm).. 100#1090340#1090310#1090280
1/4" (0.63mm).1000#1090360#1090330#1090300
3/8" (9.5mm).... 100#1090250#1090220#1090190
3/8" (9.5mm)...1000#1090270#1090240#1090210
1/2" (12.7mm).. 100#1090160#1090130#1090100
1/2" (12.7mm).1000#1090180#1090150#1090120
3/4" (19mm)..... 100#1090070#1090040#1090010
3/4" (19mm)....1000#1090090#1090060#1090030

Heavy Bonded Arbor Band

These Heavy Bonded Arbor Bands have a coarse grit that is labor saving and long lasting.



100/Box	500/Box
#10903903/8" (9.53 mm)#1090400
#10903703/4" (19 mm)#1090380

Arbor Band Mandrels

These mandrels contain silicon carbide abrasive bands designed for aggressive trimming and finishing of acrylics, silicone relines, and other materials. We offer the mandrel and high quality rubber head, which are sold separately.



Mandrels	Rubber Head Only
#15200101/4" (6.3 mm) 3/32" shankN/A
#15200203/8" (9.53 mm) 3/32" shank#1502505
#15200301/2" (12.7 mm) 3/32" shank#1502510
#15200353/4" (19 mm) 3/32" shank#1502515

Field Abrasive Rolls

Field "Strong Stitch" Abrasive Rolls are available in various coarsenesses. The abrasive rolls are used for finishing and polishing.



100/Box

#1720010#3 (1/2" dia x 3/4" ht x 5/32" hole)
#1720050#4 (1/2" dia x 1" ht x 5/32" hole)
#1720070#5 (3/4" dia x 1" ht x 5/16" hole)
#1720090#6 (13/16" dia x 1" ht x 7/16" hole)
#1720030#7 (1/2" dia x 3/4" ht x 5/16" hole)

1000/Box

#1720020#3 (1/2" dia x 3/4" ht x 5/32" hole)
#1720060#4 (1/2" dia x 1" ht x 5/32" hole)
#1720100#6 (13/16" dia x 1" ht x 7/16" hole)
#1720040#7 (1/2" dia x 3/4" ht x 5/16" hole)

Field Roll Mandrels

Field Roll Mandrels are long lasting and durable.



#1730020No. 3 Roll.....1/4" shank (6.3 mm)
#1730040No. 4 Roll.....1/4" shank (6.3 mm)
#1730050No. 5 Roll.....1/4" shank (6.3 mm)
#1730060No. 6 Roll.....1/4" shank (6.3 mm)
#1730070No. 7 Roll.....3/32" shank (2.38 mm)

Econo Cutters

These Econo Cutters are a great combination of economical and performance. The cutters can withstand speeds up to 24,000 rpm. The .035 thick cutters are thin but still strong enough to get the job done.



#13004802-1/2" dia. (63.5 mm)
#13004901-1/2" dia. (38.1 mm)
#13005004" dia. (101.6 mm)

LABORATORY

Econo Cutter Mandrel

These Econo Cutter Mandrels are special mandrels required for 2-1/2" (63.5 mm) diameter and 4" (101.6 mm) diameter econo cutters.



#1300510Econo Cutter Mandrel

Carbo Model Trimmer Wheel

Keystone Carborundum Model Trimmer Wheels are reversible and available in two sizes. These discs are compatible to Handler, Torit, Buffalo, Foster and Tooth Master units.



For Handler, Torit, Buffalo, Foster and Tooth Master units. Coarse, 1 hole.

#190062010" x 1/4" x 1" (254 x 6.3 x 25.4 mm)

Coarse, 1 hole.

#190063012" x 1/4" x 1" (304.8 x 6.3 x 25.4 mm)

Coarse wheel for Foster unit only, 2 hole.

#190063512" x 1/4" x 1" (304.8 x 6.3 x 25.4 mm)

Waterproof Model Trimmer Discs

These superior quality coated abrasive discs will cut models three to four times faster than most discs.

These Waterproof Model Trimmer Discs reduce noise by 75% and are available in two sizes. Backing plates and plastic washers are available.



Complete set, backing wheel and washer, 6 discs

#190064010" (254 mm)

Replacement set, 6 discs

#190065010" (254 mm)

Plastic backing wheel, 1 disc

#190066010" (254 mm)

Plastic washer, 1 disc

#1900670for 10" (254 mm) & 12" (304.8 mm)

Complete set, backing wheel and washer, 4 discs

#190068012" (304.8 mm)

Replacement set, 4 discs

#190069012" (304.8 mm)

Refill, 25/Pkg

#190069512" (304.8 mm)

Plastic backing wheel, 1 disc

#190070012" (304.8 mm)

Mizzy Heatless Wheels

Heatless Original Wheels (grey) are made of a silicone carbide abrasive to make a unique, non-loading binder for rapid reduction through the final shaping of non-precious, semi-precious, precious alloys, porcelain, acrylic, denture teeth and natural dentition, without generating excessive heat. These wheels come in 15 difference sizes and in regular grit. They also withstand a maximum of 20,000 rpm.

The White Heatless Wheels are non-contaminating, aluminum oxide wheels that can be used on either porcelain or precious and semi-precious metal. They cut fast, run true, wear evenly, and won't ever clog. They also come in 15 different sizes. Sold 50/Box.



Regular

- #6130100 Size 1 3/16" (4.8 mm) x 1".....#6150100
- #6130200 Size 2 1/8" (3.1 mm) x 1".....#6150200
- #6130300 Size 3 3/32" (2.4 mm) x 1".....#6150300
- #6130400 Size 4 3/16" (4.8 mm) x 7/8".....#6150400
- #6130500 Size 5 1/8" (3.1 mm) x 7/8".....#6150500
- #6130600 Size 6 3/32" (2.4 mm) x 7/8".....#6150600
- #6130700 Size 7 3/16" (4.8 mm) x 3/4".....#6150700
- #6130800 Size 8 1/8" (3.1 mm) x 3/4".....#6150800
- #6130900 Size 9 3/32" (2.4 mm) x 3/4".....#6150900
- #6131000 Size 10 3/16" (4.8 mm) x 5/8".....#6151000
- #6131100 Size 11 1/8" (3.1 mm) x 5/8".....#6151100
- #6131200 Size 12 3/32" (2.4 mm) x 5/8".....#6151200
- #6131300 Size 13 3/16" (4.8 mm) x 1/2".....#6151300
- #6131400 Size 14 1/8" (3.1 mm) x 1/2".....#6151400
- #6131500 Size 15 3/32" (2.4 mm) x 1/2".....#6151500

White

Red Heatless Wheels

The Red Heatless Wheels have a ceramic-coated abrasive grain that ensures a smooth, free, cutting wheel at the coolest temperatures. This is due to the unusual micro-chipping action of the crystals as they wear. The wheels are made of non-contaminating pure alpha aluminum oxide hexagonal crystals with a tough ceramic coating. Great for use on all non-precious and semi-precious metals, porcelains, cercons and lava – there's no need to change back to a silicon carbide wheel. The wheels are long lasting and exhibits excellent uniform wear and available in the most popular Mizzy sizes.



- #6131600Size 1
- #6131603Size 3
- #6131605Size 5
- #6131608Size 8

Key-flex Discs

Key-flex Discs are made of stainless steel with diamond particles. Easy, smooth cutting discs that last a long time. Available in three different sizes: Key-flex Discs, Key-flex Super, Key-flex Micro.

0.3mm thick, 7/8" diameter

#1290500Key-Flex Thin



0.3mm thick, 7/8" diameter
double-sided 1/8" band

#1290510Key-Flex Super



0.2mm thick, 7/8" diameter
double-sided 1/8" band

#1290520Key-Flex Micro



Flexible Diamond Discs

These flexible Diamond Discs are perfect for dental labs. All discs are mounted and measure 7/8" diameter.

For use with ceramics and composites.
Double sided. 5-10,000 rpm.

#1290710Kool-Flex 0.17mm



For use with ceramics, composites and
acrylics. Double sided. 10-20,000 rpm.

#1290715Multi-Flex 0.17mm



For use with ceramics. Double sided.
10-20,000 rpm.

#1290700Ultra-Flex 0.20mm



For use with ceramics and composites.
Single sided. 5-10,000 rpm.

#1290690Ultra-Flex 0.17mm



For use with ceramics, composites and
acrylics. Double sided. 5-10,000 rpm.

#1290695Mini-Flex 0.10mm



For use with ceramics, plaster and acrylic
veneers. Double sided. 10-20,000 rpm.

#1290705Al-Flex Full Cover 0.20mm



Ti-Coated Diamond Discs

These discs are Titanium Nitrate coated diamond discs. They are sold by each with one mandrel.

Contour Perforated Coarse discs are used for contouring ceramics and acrylics. (0.014) 22mm dia.

#1290820 Single Pack #1290821 Double Pack



Contour Perforated Fine discs are used for contouring ceramics and acrylics. (0.014) 22mm dia.

#1290800 Single Pack #1290801 Double Pack



Cut Flex Coarse discs are used for cutting ceramics and acrylics and can be used for heavier cutting. (0.014) 22mm dia.

#1290810 Single Pack #1290811 Double Pack



Cut Flex Fine discs are used for separating ceramics and acrylics. (0.007) 22mm dia.

#1290790 Single Pack #1290791 Double Pack



Super Flex Coarse discs are very thin and flexible. These discs can be used for light contouring and working in interproximal slots. Not for cutting. (0.014) 22mm dia.

#1290825 Single Pack #1290826 Double Pack



Sep Flex Fine .007 discs are used for separating and contouring of metals, ceramics and acrylic. (0.007) 22mm dia.

#1290775 Single Pack #1290776 Double Pack



Sep Flex Fine .006 discs are single sided and used for separating metals, ceramics and acrylic. (0.006) 22mm dia.

#1290780 Single Pack #1290781 Double Pack



Sep Flex Coarse discs are used for separating and contouring metals, ceramics, acrylic and plaster. (0.007) 22mm dia.

#1290815 Single Pack #1290816 Double Pack



Top Flex Perforated Fine discs are used for contouring ceramics. (0.007) 22mm dia.

#1290795 Single Pack #1290796 Double Pack



ACRYLICS

Diamond D® Acrylic

Advanced natural-looking dentures require artistry as well as technical expertise. Diamond D® was specially formulated for exceptional working and handling characteristics for anatomic sculpting. This revolutionary new resin has just the right translucency and color variations found in natural tissue as well as ultra impact and flexural strength. The demand for function and aesthetics in dental fabrication is growing. Diamond D® can help you take your business to a new level.



1 lb. Powder with 8 oz. Liquid (0.45 kg/0.25 L)

Heat Cure	Self Cure
#1013061Original.....	#1013073
#1013062Light.....	#1013074
#1013063Light Reddish Pink.....	#1013075
#1013064Chroma Essence.....	N/A
#1013112Dark Veined.....	#1013115

1 lb. Powder Only (0.45 kg)

Heat Cure	Self Cure
#1013020Original.....	#1013050
#1013024Light.....	#1013054
#1013028Light Reddish Pink.....	#1013058
#1013032Chroma Essence.....	N/A
#1013105Dark Veined.....	#1013109

5 lb. Powder with 1 Quart Liquid (2.27 kg/0.95 L)

Heat Cure	Self Cure
#1013065Original.....	#1013077
#1013066Light.....	#1013078
#1013067Light Reddish Pink.....	#1013079
#1013068Chroma Essence.....	N/A
#1013113Dark Veined.....	#1013116

5 lb. Powder Only (2.27 kg)

Heat Cure	Self Cure
#1013021Original.....	#1013051
#1013025Light.....	#1013055
#1013029Light Reddish Pink.....	#1013059
#1013033Chroma Essence.....	N/A
#1013106Dark Veined.....	#1013110

25 lb. Powder with 4 Quarts Liquid (11.34 kg/3.79 L)

Heat Cure	Self Cure
#1013069Original.....	#1013081
#1013070Light.....	#1013082
#1013071Light Reddish Pink.....	#1013083
#1013072Chroma Essence.....	N/A
#1013114Dark Veined.....	N/A

25 lb. Powder Only (11.34 kg)

Heat Cure	Self Cure
#1013022Original.....	#1013052
#1013026Light.....	#1013056
#1013030Light Reddish Pink.....	#1013060
#1013034Chroma Essence.....	N/A
#1013107Dark Veined.....	#1013111

Liquids

Heat Cure	Self Cure
#10130148 oz. Monomer (0.25 L).....	#1013017
#10130151 Quart Monomer (0.95 L).....	#1013018

20 Minute Liquids

Heat Cure	Self Cure
N/A8 oz. Monomer (0.25 L).....	#1013093
N/A1 Quart Monomer (0.95 L).....	#1013094

Diamond D® Ultra Wash

Diamond D Ultra Wash eliminates skinning of separator from model surface. It also enhances bond of acrylic to teeth and promotes cleaner deflasking and denture break out. This wash allows for total wax elimination from denture mold. Dental specific product used in place of detergent.



#10131201 Gallon (3.79 L)

Sledgehammer Heat Cure Acrylic



Sledgehammer Heat Cure is a high impact, cross-linked, color stable denture base with excellent working properties. It's compatible with most injection systems. Available in 4 aesthetics, cadmium free shades with matching repair, pour and microwavable materials. Can be used with Sledgehammer 20 Minute Monomer for shortened cure time.



1 lb. Powder with 8 oz. Liquid (0.45 kg/0.25 L)

#1000470LRP	#1000471 Original
#1001965Light Fibered Plus	#1000535 Light Fibered
#1000472 Clear	

5 lb. Powder with 1 Quart Liquid (2.27 kg/0.95 L)

#1000473LRP	#1000474 Original
#1001966Light Fibered Plus	#1000536 Light Fibered
#1000475 Clear	

25 lb. Powder with 4 Quarts Liquid (11.34 kg/3.79 L)

#1000476LRP	#1000477 Original
#1001967Light Fibered Plus	#1000537 Light Fibered
#1000478 Clear	

Sledgehammer Heat Cure Acrylic

1 lb. Powder Only (0.45 kg)

#1000494LRP	#1000498 Original
#1001961Light Fibered Plus	#1000543 Light Fibered
#1000502Clear	

5 lb. Powder Only (2.27 kg)

#1000495LRP	#1000499 Original
#1001962Light Fibered Plus	#1000544 Light Fibered
#1000503Clear	

25 lb. Powder Only (11.34 kg)

#1000496LRP	#1000500 Original
#1001963Light Fibered Plus	#1000545 Light Fibered
#1000504Clear	

Liquid Only

#1000516 8 oz. (0.25 L)	#10005171 Quart (0.95 L)
-------------------------------	--------------------------------

Sledgehammer Self Cure Acrylic



Easy to mix, cross-linked, non-slumping formula that matches all Sledgehammer shades as well as other leading brands. Great bond strength makes it perfect for relines and repairs using the salt and pepper or dough technique. A high quality, cadmium free acrylic offered at a significant savings.

1 lb. Powder with 8 oz. Liquid (0.45 kg/0.25 L)

#1000482LRP	#1000483 Original
#1001973Light Fibered Plus	#1000539 Light Fibered
#1000484Clear	

5 lb. Powder with 1 Quart Liquid (2.27 kg/0.95 L)

#1000485LRP	#1000486 Original
#1001974Light Fibered Plus	#1000540 Light Fibered
#1000487Clear	

25 lb. Powder with 4 Quarts Liquid (11.34 kg/3.79 L)

#1000488LRP	#1000489 Original
#1001975Light Fibered Plus	#1000541 Light Fibered
#1000490Clear	

1 lb. Powder Only (0.45 kg)

#1000507LRP	#1000510 Original
#1001970Light Fibered Plus	#1000547 Light Fibered
#1000513Clear	

5 lb. Powder Only (2.27 kg)

#1000508LRP	#1000511 Original
#1001971Light Fibered Plus	#1000548 Light Fibered
#1000514Clear	

25 lb. Powder Only (11.34 kg)

#1000509LRP	#1000512 Original
#1001972Light Fibered Plus	#1000549 Light Fibered
#1000515Clear	

Liquid Only

#1000521 8 oz. (0.25 L)	#10005221 Quart (0.95 L)
-------------------------------	--------------------------------

Millennium Acrylic



A superior quality highly impact-resistant denture material which has been chemically engineered to resist material fatigue, while maintaining a highly developed standard of denture aesthetics.



1 lb. Powder with 8 oz. Liquid (0.45 kg/0.25 L)

Heat Cure

#6010026Light Veined	#6010038
#6010027Deep Veined	#6010039
#6010028Clear	#6010040

Self Cure

5 lb. Powder with 1 Quart Liquid (2.27 kg/0.95 L)

Heat Cure

#6010030Light Veined	#6010042
#6010031Deep Veined	#6010043
#6010032Clear	#6010044

Self Cure

25 lb. Powder with 1 gallon Liquid (11.34 kg/3.79 L)

Heat Cure

#6010034Light Veined	N/A
#6010035Deep Veined	N/A
#6010036Clear	N/A

Self Cure

1 lb. Powder Only (0.45 kg)

Heat Cure

#6010002Light Veined	#6010014
#6010003Deep Veined	#6010015
#6010004Clear	#6010016

Self Cure

5 lb. Powder Only (2.27 kg)

Heat Cure

#6010006Light Veined	#6010018
#6010007Deep Veined	#6010019
#6010008Clear	#6010020

Self Cure

25 lb. Powder Only (11.34 kg)

Heat Cure

#6010010Light Veined	#6010022
#6010011Deep Veined	N/A
N/AClear	#6010024

Self Cure

Liquid Only

Heat Cure

#6010045 8 oz. (0.25 L)	#6010048
#6010046 1 Quart (0.95 L)	#6010049

Self Cure

Millennium Pour Acrylic

Keystone Industries is devoted to providing labs with options for denture fabrication. So, Millennium Pour Denture Base Acrylic is developed to deliver all the choices labs are looking for in a base acrylic.



CE

Field proven when used with conventional pour methods, Millennium Pour will meet expectations in ease of use, color stability, and shade. Other features include: Porosity free curing, consistent particle size, easy flowing for consistent fill, industry standard shades, and superior finish.

1 lb. Powder with 8 oz. Liquid (0.45 kg/0.25 L)

- #1014352Light Fibered Original
- #1014353Light Fibered Plus
- #1014350Clear
- #1014351Heavy Veined

5 lb. Powder with 1 Quart Liquid (2.27 kg/0.95 L)

- #1014362Light Fibered Original
- #1014363Light Fibered Plus
- #1014360Clear
- #1014361Heavy Veined
- #1014364Light Reddish Pink

25 lb. Powder with 4 Quarts Liquid (11.34 kg/3.79 L)

- #1014372Light Fibered Original
- #1014373Light Fibered Plus
- #1014370Clear
- #1014374Light Reddish Pink

1 lb. Powder Only (0.45 kg)

- #1014330Light Fibered Original
- #1014335Light Fibered Plus
- #1014320Clear
- #1014325Heavy Veined
- #1014340Light Reddish Pink

5 lb. Powder Only (2.27 kg)

- #1014331Light Fibered Original
- #1014336Light Fibered Plus
- #1014321Clear
- #1014326Heavy Veined
- #1014341Light Reddish Pink

25 lb. Powder Only (11.34 kg)

- #1014332Light Fibered Original
- #1014337Light Fibered Plus
- #1014327Heavy Veined
- #1014342Light Reddish Pink

Liquid Only

- #10143108 oz. (0.25 L)
- #10143111 Quart (0.95 L)

Sledgehammer 20 Minute Monomer

CE

This is an excellent product for the rapid curing of Sledgehammer Heat Cure acrylics. Available in both 8- and 16-ounce containers. Kits with Sledgehammer Heat Cure powder and '20 minute' monomer available upon request.



#10005908 oz. (0.25 L)

#10005911 Quart (0.95 L)

Versacryl Thermo-soft Adjustable Reline

CE

This unique product has been developed to create thermo-adjustable relines. Imagine having the luxury to simply heat a denture and re-adjust the fit in seconds and if it's not satisfactory the procedure can be repeated anytime by the dentist, the technician or even the patient themselves.

The Versacryl Reline is not a regular "soft reline" material. It is a heat sensitive liner that softens in warm water between 120°F - 205°F. At body temperature the Versacryl has an extremely comfortable cushiony effect that yields to pressure and actually creates better suction than conventional soft liners.

Versacryl is a cross-linked acrylic denture material guaranteed to create a chemical bond with all other denture acrylics.

Versacryl is available in Heat Cure or Self Cure and no special equipment is needed to process.



Lab Kit includes:

- 2 x 50 g Powder (clear and pink veined)
- 1 x 60 ml Hardener Monomer Liquid
- 1 x 60 ml Softener Monomer Liquid
- 2 measuring cups, eye dropper and instructions

#1014032Heat Cure Lab Kit

#1014031Self Cure Lab Kit

Reline Kit includes:

- 2 x 50 g Powder (pink)
- 2 x 60 ml Pre-mixed Monomer Liquids
- 2 measuring cups, eye dropper and instructions

#1014034Heat Cure Lab Kit

#1014033Self Cure Lab Kit

Bulk Sizes

#10140231 lb. Bulk Pink Powder

#10140221 lb. Bulk Clear Powder

#10140191 qt. Pre-mixed Heat Cure Monomer

#10140181 qt. Pre-mixed Self Cure Monomer

Pro-flex NFC Resin

Pro-flex is a semi-flexible acrylic that has been chemically altered to give the final appliances both strength and semi-flexibility. This material is excellent for making flexible clasps, sub-lingual flexible flange areas, and semi-flexible partials and splints. It can be polished, shaped, relined, and repaired like regular acrylics. When this unique product is coupled with Pro-flex strengtheners, a semi-flexible, durable, strong appliance is produced. NFC Resin is available in standard packaging of 120 grams of liquid and 160 grams of powder or bulk package of 720 grams of liquid and 960 grams of powder. Denture/Partial Intro Kits also available.



- #9640970Light Pink
- #9640990Medium Pink
- #9640900Dark Pink
- #9640950NFC Intro Kit Partial

Pro-flex NFC Clear

The Pro-flex NFC Resin also is in clear. It is used to make hard but slightly flexible splints. This combination of characteristics allows the technician to make splints that flex over the height of contour, over any irregular dentition and provide splints that require no further retention. The flexible nature of the finished splint coupled with the hardness of the NFC resin makes for comfortable yet long-lasting splint.



- #9640960Clear Splint Kit
- #9640880Clear NFC

ProCure Soft Reline

This is a great chairside, dual cure, bonded and microfilled soft reline composite. The ProCure soft reline mixes into an easy-to-apply impression taking consistency that can be applied to a prepared denture. The denture is then fitted into the mouth for a one step, one appointment fitting. The end result gives a long-lasting (two years or better) soft reline that your patients will love. ProCure packages the Soft reline in 8 individual units in the Professional Kit or one Bulk Economy Kit (average 25-30 relines).



- #9143210Pro Kit
- #9143240Bulk Kit

ProCure Hard Reline

This is an excellent chair side, dual cure, hard reline composite that can be added directly to a prepared denture. Mix material to impression-like consistency, apply to denture, and fit at chair side. No heat processing, no methyl methacrylate, and no heat discomfort to oral tissue. The result gives a permanent reline with no thermally induced expansion or shrinkage. The composite has been formulated to absorb very little water, be impervious to bacterial build-up, and is color stable. ProCure packages the Hard Reline in 8 individual units in the Professional Kit or one Bulk Economy Kit (average 25-30 relines).



- #9143100Pro Kit
- #9634740Bulk Kit

ProCure Custom Tray Wafers

The ProCure Tray Material comes in a soft, pliable and ready-to-use wafer. The trays can be quickly formed onto model to make a fast accurate and stable custom impression tray. A handle can be attached and the tray shaped while in softened state and then cured and hardened. Ready in just minutes.



- #9634930Custom Tray Wafers, 50 wafers

Hatho Thermoplastic Polishing Kit

Polishing set designed for an efficient high shine on all flexible partial materials.

Includes:

- 2 Hatho Flex Polishers
- 1 Mira Buff
- 1 Yellow Mira Buff
- 1 Polistar Compound and 1 Polistar White Compound



- #1670160Thermoplastic Polishing Kit

All Purpose Duplicating Material

This All-Purpose Duplicating Material can be used with all stone and investment models also for pour acrylic dentures.

- A quality, reversible duplicating material for use with all investment types
- Ideal for use with all duplicating conditioners
- No wash required

- #13103101 gallon (3.78 L)
- #13103155 gallon (18.92 L)



LABORATORY

Itsoclear

Itsoclear provides the ability to produce and repair a clasp in under 4 minutes with cosmetic appeal unlike any other clasp.



#1009150Clear Universal Clasps, 6/Pkg

#1009151Clear Molar Clasps, 6/Pkg

Dappen Dishes

We offer both glass and disposable Dappen Dishes that work great in various dental applications. The glass Dappen Dish is made of solid heat-resistant glass. The thick and heavy base makes the dish stable and protects from spills. The dish is ideal for mixing acrylics, amalgam, resins, sealants and other materials.

Glass Dappen Dishes

#06-00510.....Blue, 1,000/Pkg

#06-00610.....Clear, 1,000/Pkg

#06-00710.....Green, 1,000/Pkg



Disposable Dappen Dishes

#10-14010.....White, 1,000/Pkg

#10-14310.....Yellow, 1,000/Pkg

#10-14510.....Blue, 1,000/Pkg

#10-14410.....Assorted, 1,000/Pkg

Reline Bond

Reline Bond forms a positive and lasting fusion between old denture material and the added new material. It eliminates unsightly white or bleaching lines which often appear on relines.

Available in a 4 oz. can.



#1011355Reline Bond

ARTICULATORS

Apex #2 Chrome Articulator

The Apex #2 Chrome Articulator has a higher arch than the K-Chrome Articulator, but still provides maximum accuracy. Allows for easy and accurate work on dental work.



#1050000Apex #2 Chrome Articulator

Apex #2 Bronze Articulator

The Apex #2 Bronze Articulator has a higher arch than the K-Chrome Articulator, but still provides maximum accuracy. Allows for easy and accurate work on dental work.



#1050005Apex #2 Bronze Articulator

Crown & Bridge K-Chrome Articulator

The Keystone Crown & Bridge K-Chrome Articulator is specifically engineered for crown and bridgework. It's easy to adjust and easy to separate parts. These articulators are machined for quality and accuracy.



#1050140C&B K-Chrome Articulator

Deluxe Magnetic Articulator

The Deluxe Magnetic Articulator comes with magnetic split casting to ensure easy removal and exact settings of models. This articulator is excellent for checking cases. It's completely interchangeable, lightweight and Teflon coated.



#1050075Deluxe Magnetic Articulator

#1050076Magnets, 4/Pkg

#1050077Curved Disc, each

Deluxe Magnetic Articulator with Pin

The Deluxe Magnetic Articulator with Pin is an excellent articulator for denture work. This item has the same advantages as the Deluxe Magnetic Articulator, but it has incisal pins for the use with denture production.



#1050074Deluxe Magnetic Articulator

#1050076Magnets, 4/Pkg

#1050077Curved Disc, each

Foster Articulators

Foster Articulators have been widely used in the dental industry since the 1940s. Today's Foster Articulators still maintain the high quality while still improving tremendously for today's needs. These are the most accurate and durable Foster Articulators ever. The simple design and ease of use allows the articulator to be used on most cases that enter the dental lab. The lateral protrusive movements are smooth and precise.



- #1050025Chrome Foster 8 (#51)
- #1050020Bronze Foster 8 (#51)



- #1050040Chrome Foster Apex 2 (#50)
- #1050030Bronze Foster Apex 2 (#50)



- #1050010Chrome Foster (#57)
- #1050015Bronze Foster (#57)



- #1050110Foster (#54) Articulator with Incisal Pin
- #1046730Incisal Pin Only



- #1036682Foster 7 (#56) Articulator w/ Anterior Pin




- #1036681Foster (#60) Crown & Bridge Articulator

Galetti Articulator

The Galetti Articulator is solidly constructed of metal alloy casting. The high quality articulator provides easy, speedy and firm grasps of models of any size. This item has perfect occlusal-articulator balance to prevent cuspidal interferences and maintains a balanced prosthetic appliance.

The Galetti Articulator allows for easy separation of upper and lower components to form two different stable working bases. The ball bearing joint allows for free motion of the upper appliance to check the occlusal-articulator balance. The hinge axis with bilateral motion gives the capability for anterior, posterior and lateral movements to show working and balancing movements.

No plaster work or accessories are required with Galetti Articulators and they are easy to clean.

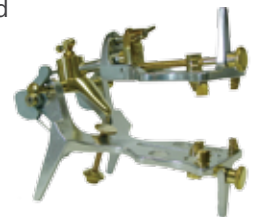

- #1050200Galetti Articulator

High Arch Articulator

The High Arch Plastic Articulator is for mounting models in the dental lab. It's made of high impact plastic with a metal rear-locking adjustment screw for a longer life and reusability. The item features lateral movement and adjustable incisal stop. The plastic articulator is lightweight, which mean a lower cost for shipping.



- #1050146High Arch Articulator

K-Chrome Denture Articulator

The K-Chrome Articulator was machined to produce maximum accuracy for this type of articulator. It makes for easy access to all surfaces, easy to adjust, and can be taken apart and assembled quickly.



- #1050145K-Chrome Denture Articulator

K-Brass Denture Articulator

The K-Brass Denture Articulator has the same features as the K-Chrome Denture Articulator but is made of softer and longer-lasting brass. The simple articulator is machined to produce maximum accuracy. The articulator gives easy access to all surfaces, easy to adjust and can be taken apart and assembled quickly.



- #1050147K-Brass Denture Articulator

Keystone Deluxe Articulator

The Keystone Deluxe Articulator is excellent for sending important cases between labs and the doctors. The articulator is lightweight, Teflon-coated and made with very accurate, interchangeable mouths. The item has a full range of movements, including lateral excursions. This is a high-quality articulator is set at a low, economical price.


#1050095Deluxe Articulator



Lab-O-Mate Articulator

The Keystone Lab-O-Mate Articulator is easy to use in dental labs. The incisal guide pin allows for adjustment of the vertical dimension during teeth alignment and adjustments. Made of high-quality materials and is long-lasting.

#1050012Lab-O-Mate Articulator

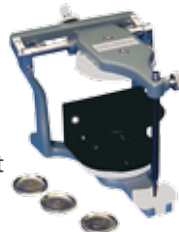

Luxury Articulator

Our Luxury Articulators still offer the finest quality while remaining at an affordable price. This Luxury Articulator is a four-piece magnetic mounting disc set with two plastic steel mounting magnet plates. There is a curved plate, a flat plate and long incisal and central pins. The mount is backed with black Teflon coating. The articulator is easy to clean.



#1050085Luxury Articulator

#1050001Mounting Disc Set, 4 pieces

#1050002Plastic Mounting Magnet Plates, 2/Pkg

#1050003Curved Plate

#1050004Flat Plate

#1050006Long Incisal Pin

#1050007Central Pin

#1050008White plate

Magnet X

The Magnet X is a quick mount model system that assures articulation accuracy every time. This articulated features a one-stage base pour for fast model attachment. It can be used with any pin system and is compatible with most articulators. Most of the parts are reusable. It's easy to pour the models, apply isolator for plaster and then mount on the articulator in the correct position every time.

#1050065Complete Kit

#1050060Base with Magnet

#1050067Black Plugs, 100 pieces

#1050068Washers, 50 pieces

#1050120Magnets, 6 pieces

#1150105Spatula, each




Plasterless Articulator

The Keystone Plasterless Articulator is simple to operate while still maintaining a high quality without plaster. Simply mount the model using the clamping set bolts, which allow for easy attachment and detachment from the articulator.

The articulator firmly grasps models of any size so the occlusion can be checked and rechecked. The Keystone Plasterless Articulator is made of highly polished aluminum so it can be easily cleaned and will not rust.

#1050150Plasterless Articulator

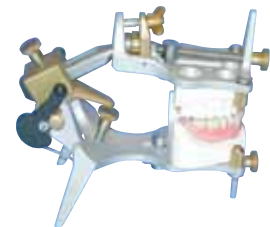

Plastic C&B Disposable Articulator

This plastic Crown and Bridge Disposable Articulator is an economical way to hold dental work. Available in 50/Box.


#1050165Plastic C&B Disposable Articulator



Precision Articulator

The Precision Articulator is great for transferring mounted cases between the same articulator without the loss of accuracy. The incisal tables can unscrew and be easily replaced. The articulator has smooth lateral and protrusive movements. There's an anterior leg for easy viewing.

#1050100Precision Articulator

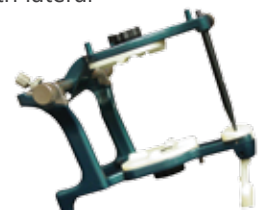

Spring Chrome Articulator

A spring-loaded articulator for crown and bridgework. Reversible top permits two heights. Also, allows for protrusive and retrusive lateral movements.

#1050080Spring Chrome Articulator



BRUSHES & BUFFS




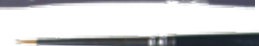











Red Sable Brushes

Our Red Sable Brushes combine excellent quality with an affordable price. All brushes available for private label. Minimum quantities required.

- #1170380 0 Right Angle 
- #1170370 00 Right Angle 
- #1170360 000 Right Angle 
- #1170410 2 Flat 
- #1170420 4 Flat 
- #1170430 6 Flat 
- #1170260 000 
- #1170270 00 
- #1170280 0 
- #1170290 1 
- #1170300 2 
- #1170310 3 
- #1170320 4 
- #1170330 5 
- #1170340 6 
- #1170350 7 
- #1170355 14 Whipper 

Synthetic Ceramist Brushes

These Synthetic Ceramist Brushes combine excellent quality with an affordable price. All of these brushes are available for private label. Minimum quantities are required.

- #1171220 2 Flat 
- #1171230 4 Flat 
- #1171240 6 Flat 
- #1171000 000 
- #1171010 00 
- #1171020 0 
- #1171030 1 
- #1171040 2 
- #1171050 3 
- #1171060 4 
- #1171070 5 
- #1171080 6 
- #1171090 7 
- #1171200 8 
- #1171205 10 
















Keystone Kolinsky Ceramist Brushes

Keystone offers an all-natural high quality Kolinsky sable bristles. The elasticity of hair recovers its original shape after stroke. These brushes give the ability to control a porcelain flow, while holding and dispensing material predictably.

- #1171260 0 
- #1171270 2 
- #1171275 4 
- #1171280 6 
- #1171285 8 
- #1170185 Big Brush 

Kolinsky Brushes

These are legendary natural Kolinsky Brushes. These brushes have a small bead inside that helps manage the humidity on the brush. You'll see the difference in your work immediately after using Kolinsky Brushes.

- #1170705 000 
- #1170710 00 
- #1170715 0 
- #1170720 1 
- #1170725 2 
- #1170730 3 
- #1170735 4 
- #1170740 5 
- #1170745 6 
- #1170750 7 
- #1170755 8 
- #1170760 2 Flat 
- #1170765 4 Flat 
- #1170700 Big Brush 
- #1170770 Big Brush Synthetic 

Diamond D® Sep Brush

There are two types of Diamond D Sep Brushes. The Diamond D Sep Brush is used to apply tin foil substitute to stone and plaster surfaces. The Diamond D Sep Brush II is used to remove excess tin foil substitute from denture teeth.



- #1170405 Diamond D Sep Brush
- #1170400 Diamond D Sep Brush II

LABORATORY

Laboratory Hand Brush

This brush has four rows of stiff black bristles set in a wooden handle. The brush is durable and convenient for any lab.



#1170250Laboratory Hand Brush

Felt Cones


Felt Cones are used with threaded mandrel or tapered spindle. These cones are made of medium hard pure woolen felt. Sold in packs of 12.



- #13400502" x 1" (50.8 x 25.4 mm) pointed
- #13400101" x 1/2" (25.4 x 12.7 mm) pointed
- #13400301-1/2" x 3/4" (38.1 x 19 mm) pointed
- #13400602" x 1" (50.8 x 25.4 mm) blunt
- #13400201" x 1/2" (25.4 x 12.7 mm) blunt
- #13400401-1/2" x 3/4" (38.1 x 19 mm) blunt

Mounted Felt Cones Kit

The Felt Cone Mounted Kit provides all your finishing needs. All these Felt Cones are a hard density and come in a set of 7 assorted shapes with a 3/32" shank.

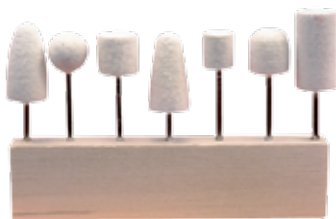

#1340230Mounted Felt Cones Kit

Felt Wheels

These Felt Wheels are excellent for chrome work.



- | | |
|---|------------------|
| 100/Pkg | 1,000/Pkg |
| #1340130 ... 1" dia. x 1/8" (25.4 x 3.1 mm) Hard... | #1340150 |
| #1340160 ... 1" dia. x 3/16" (25.4 x .7 mm) Med... | #1340180 |
| #1340190 ... 1" dia. x 1/4" (25.4 x 6.3 mm) Soft | #1340210 |

Felt Cylinders

These Felt Cylinders are a great for use on threaded mandrels.



- | | |
|---|------------------|
| 100/Pkg | 1,000/Pkg |
| #1340070 1/4" dia. x 1/2" (6.3 x 12.7 mm) | #1340090 |
| #1340100 1/4" dia. x 3/8" (6.3 x 9.5 mm) | #1340120 |

Leather Center Buffs

Our Leather Center Buffs is very rigid on a tapered spindle. Made of quality muslin material with a nice leather center. Sold in packages of 12.



- #11801003" (76.2 mm) x 35 ply
- #11801104" (101.6 mm) x 40 ply

Lead Center Buffs

These quality Lead Center Buffs have the same muslin ply with a lead center for rigidly. Sold in packages of 12.



- #11800803" (76.2 mm) x 36 ply
- #11800904" (101.6 mm) x 36 ply

Baby Muslin Buffs

These Baby Muslin Buffs are ideal for polishing small and difficult-to-reach areas. The buffs come in 16-Ply.



- | | |
|--|----------------|
| 50/Pkg | 100/Pkg |
| #11800101" dia. (25.4 mm) x 16 ply | #1180015 |
| #11800201-1/2" dia. (38.1 mm) x 16 ply | #1180025 |

Chamois Buffs

These Chamois Buffs are top quality stitched, leather-center chamois buffs. This buff produces a very high luster and is 12-ply.



- #11800402-1/2" x 1/2" (63.5 x 12.7 mm), 12 ply, each
- #11800303" x 1/2" (76.2 x 12.7 mm), 12 ply, each
- #11800501" (25.4 mm), 12 ply, 12/Pkg

Flannel Buffs

Our Flannel Buffs are made of high-quality flannel and are virtually lint free. Use these buffs with a fine rouge to produce a final high luster on gold or acrylic. The 20-ply buffs also can be used as a wipe off wheel without composition.



- #11800653" (76.2 mm), 20 ply, each
- #11800754" (101.6 mm), 20 ply, each

Muslin Buffs

These Muslin Buffs are made of carefully selected material and each ply is a full cut. The uniform stitches will not pull out and long-lasting.



- | | |
|---------------------------|---------------------------|
| #11801202" x 30 ply | #11801604" x 30 ply |
| #11801303" x 24 ply | #11801704" x 42 ply |
| #11801353" x 30 ply | #11801805" x 30 ply |
| #11801403" x 35 ply | #11801905" x 45 ply |
| #11801504" x 24 ply | #11802006" x 30 ply |

Yeti Dia Quick

Dia Quick is a diamond impregnated felt wheel used for quick polishing of ceramics, metal and synthetic surfaces. Available in 10/Pkg.

North American Sales Only.


- #1860012Dia Quick



BURNERS & TORCHES

30B Blowpipe

For gas and compressed air uses in a lab, this 30B Blowpipe is the most modern and attractive in its field. It operates on any fuel gas (except acetylene) including natural, mixed, propane, butane and more with compressed air. All these fuel gases are handled with equal ease thanks to this Blowpipe.

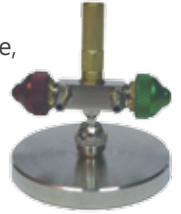

- #182017530B Blowpipe

Bunsen Burner

This Bunsen Burner is available for natural gas only. It has a chrome plated brass with adjustable air inlets. Gas inlet suitable for 8mm (5/16") ID tubing. Dimensions: 100mm (4") height, 75mm (3") base diameter.



- #1190010Model #111 Bunsen Burner
- #1012920Latex Tubing 1/4" I.D. x 3/32", 1 ft.
- #1012921Latex Tubing 1/4" I.D. x 3/32", 50 ft. roll

Deluxe Bunsen Burner

This Deluxe Bunsen Burner is made for natural gas only. The adjustable flame control provides ease of use. Also the base permits 30-degree angle or diagonal position of the flame. Works great in labs that need flexibility and adjustments.



- #1190012Deluxe Bunsen Burner

Hanau Alcohol Torch

This Hanau Alcohol Torch is used as an alcohol lamp or as a blowtorch. The instrument is used to melt control the shape and flow of wax when taking impressions. Also the torches are used during the fabrication of dentures, setting up teeth, waxing, light soldering or even jewelry making. It can be used as a torch to create either a needle point or brush-type flame.



- #1820010Hanau Alcohol Torch

Plastic Alcohol Torch

This Plastic Alcohol Torch is used for easy, quick applications in the lab. This plastic torch uses denatured alcohol only. Wick is long-lasting and keeps a steady flame.



- #1820015Alcohol Torch
- #1820018Wicks, 12/Pkg

LABORATORY

Microtorch

This Microtorch is a 2-in-1 open flame torch and flameless heat tool with a heat shrink attachment. The torch is excellent use to heat materials, ignite materials, pinpoint torch, shrink tubing, solder and desolder, terminate specialty connectors and more!



#1820005Microtorch

Microtorch

A hand held, self-igniting, butane powered microtorch creates a precise flame emitting from a nozzle head, allowing direct heat application to an object.



#9139680Microtorch

National Blowpipe

National Blowpipes are economical and easy to operate. There is no other torch as well-known as these pipes. National Blowpipes operate on any fuel gas because of the patented proportional gas mixer. There's no need to change mixers, instead just select the proper tip. There is a rotating leak-proof union to ensure safety and compliance.



- #1820020Model 3A-B-3 Blowpipe w/ wrench, N1 and N2 tips
- #1820021Model 3A Blowpipe w/ wrench only (no tips)
- #1820030N 0 Tip
- #1820040N 1 Tip
- #1820050N 2 Tip
- #1820062N 4 Veriflo Tip w/ Adapter
- #1820070Ox 1 Tip
- #1820080Ox 2 Tip
- #1820090Ox 3 Tip
- #1820100Ox 4 Tip
- #1820120Multi Hole High Heat Oxygen Tip HTM-2
- #182014012" Torch Tip Extension

Harris Automatic Casting Torch (#50-10S)

The Harris Automatic Casting Torch reduces operating costs while remaining convenient and safe. The torch switches from operating flame to pilot light by releasing the thumb lever. These items are UL listed.



- #1821025Tip
- #1821011Handle
- #1820183Mixer
- #1821060Tubing

Harris Manual Casting Torch (#16S)

The Harris Manual Casting Torch has a lightweight, medium duty handle and is UL listed.



- #1821025Tip
- #1821084Handle
- #1821083Mixer
- #1821060Tubing

Accessories for Casting Torches

- #1821030Multi Hole Casting Tip 1390-H
- #1821050Single Orifice Casting Tip 1390-8N
- #1821040Soldering Tip 1390-2N
- #1821025D-50-C Tip Tube
- #1821015D50 XL Tip Tube, 7"

CASTING

Hi-Heat Split Casting Rings

These Hi-Heat Split Casting Rings are perfect for chrome castings. The rings are manufactured from special fire resistant stainless steel that can withstand high temperatures. Clamps are supplied with the rings, but replacements are sold individually.



- #17001502-1/2" x 2-1/2" (57.1 x 57.1 mm)
- #17001603" x 2-1/2" (76.2 x 63.5 mm)
- #1700180Extra Clamp for Rings

Casting Rings

All Keystone Casting Rings are made of a stainless steel. Even after facing repeat heat applications, they maintain precision fitting and stable in size. This ensures a long life.



- #10700701-1/4" x 1-1/2" (31.7 x 38.1 mm)
- #10700801-1/2" x 1-1/2" (38.1 x 38.1 mm)
- #10700901-3/4" x 1-1/2" (44.4 x 38.1 mm)
- #17001001-3/4" x 1-3/4" (44.4 x 44.4 mm)
- #17001102" x 2" (50.8 x 50.8 mm)
- #17001202-1/2" x 2" (63.5 x 50.8 mm)
- #17001303" x 2-1/2" (76.2 x 63.5 mm)
- #17001403-1/2" x 2-1/2" (88.9 x 63.5 mm)
- #10700401-3/4" x 2" (44.4 x 50.8 mm)
- #10700501-1/4" x 1-1/4" (31.7 x 31.7 mm)
- #10700601-1/4" x 1-3/4" (31.7 x 44.4 mm)
- #10700851-1/2" x 1-3/4" (38.1 x 44.4 mm)

Crucible Formers

The Crucible Formers have a tapered ring design for easy removal of mold. These are made of rubber for long lasting and reliable performance. These eliminate the need for ring liner.



- #1700141.....1-1/4" (31.7mm) #1700145.....2-1/2" (63.5mm)
- #1700142.....1-1/2" (38.1mm) #1700146.....3" (76.2mm)
- #1700143.....1-3/4" (44.4mm) #1700147.....3-1/2" (88.9mm)
- #1700144.....2" (50.8mm)

Casting Ring Liner

Our Casting Ring Liner is made of refractory fibers that have a melting point in excess of 3,200°F. The Liner is asbestos free and sold in 50-foot rolls. Various available sizes.



- #1060040 ... 1-1/4"h x 1/32"t (31.7 x .79 mm)
- #1060045 ... 1-1/4"h x 1/16"t (31.7 x 1.5 mm)
- #1060050 ... 1-1/2"h x 1/32"t (38.1 x .79 mm)
- #1060055 ... 1-1/2"h x 1/16"t (38.1 x 1.5 mm)
- #1060060 ... 1-3/4"h x 1/32"t (44.4 x .79 mm)
- #1060065 ... 1-3/4"h x 1/16"t (44.4 x 1.5 mm)
- #1060070 ... 2"h x 1/32"t (50.8 x .79 mm)
- #1060075 ... 2"h x 1/16"t (50.8 x 1.5 mm)
- #1060080 ... 3"h x 1/16"t (76.2 x 1.5 mm)

Standard Slotted Type Crucible #30

These high quality Standard Slotted Type Crucibles offer optimum performance. This crucible fits standard type Kerr and Emesco casting machines. Sold individually.



- #5920200Standard Slotted Type Crucible #30

Standard Hooded Type Crucible #31


This high quality Standard Hooded Type Crucible offers optimum performance in induction furnaces. This crucible fits standard type Kerr, Lucas and Emesco casting machines. Sold individually.



- #5920235Standard Hooded Type Crucible #31

Standard Slotted Type Crucible #32

These high quality Standard Slotted Type Crucibles offer optimum performance in furnaces. This crucible fits standard type Kerr and Emesco casting machines. The Standard Slotted Type Crucible comes with a hole. Sold individually.

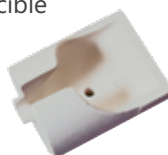

- #5920201Standard Slotted Type Crucible #32

Induction Crucible II #35


This high quality Induction Crucible II offers optimum performance in induction furnaces. This crucible fits standard type Howmedica and electronic casting machines.



- #5920210Induction Crucible II #35

Tri-Dynamics Hooded Type Crucible #35

This Tri-Dynamics Hooded type Crucible fits centrifugal casting machines, including Kerr and Emesco. It's designed with a flat inner table to allow for less turbulence. This crucible is coded for easy identification of metals. Sold individually.

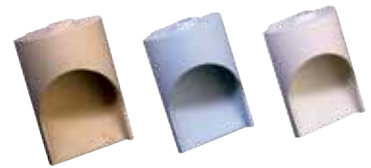

- #5920220Gold
- #5920221Blue
- #5920222White

Centrifugal Type Crucible #46

This Centrifugal Type Crucible fits standard Dentsply type casting machines. Sold individually.



- #5920250Centrifugal Type Crucible #46

Two Stage Slotted Crucible #47

This Two Stage Slotted Crucible fits standard type Kerr and Emesco casting machines. Sold individually.



- #5920230Two Stage Slotted Crucible #47

Induction Crucible #49

These Induction Crucibles offer optimum performance in induction furnaces. This crucible fits standard type Tycomatic and Unitek casting machines. Sold individually.



- #5920240Induction Crucible #49

Induction Hooded Crucible #49A

These Induction Crucible Hooded offer optimum performance in induction furnaces. This crucible fits standard type Tycomatic casting machines. Sold individually.



- #5920245Induction Hooded Crucible #49A

LABORATORY

Tri-Dynamics Slotted Type Crucible #59

This Tri-Dynamics Slotted Crucible fits centrifugal casting machines, including Kerr and Emesco. It's designed with a flat inner table to allow for less turbulence. This crucible is coded for easy identification of metals.

- #5920260Gold
- #5920261Blue
- #5920262White



High Heat Silica Crucible

This High Heat Silica Crucible is manufactured specially for Keystone Industries. The high-quality silica is able to withstand high temperatures while remaining non-contaminating.



- #1280010Type A, 2/Pkg

Wesgo Crucible

Wesgo Crucibles are able to withstand high temperatures and are non-contaminating. High quality to ensure optimum performance. Sold individually.



- #1280060Small A
- #1280070Slotted A
- #1280050Type A
- #1280080Type D

DEBUBBLIZERS

Klean Cast Debubbler

The Klean Cast Debubbler can be used on wax patterns, impressions and models. The non-contaminating, alcohol-free solution eliminates surface oils from the wax to ensure a smooth finish. Does not contain any solvents, therefore, it does not distort or effect the wax in any way.



- #101410132 oz. (.95 L)
- #10141008 oz. (.23 L)

Mizzy Blue Debubbler

Blue Debubbler produces smooth, accurate, bubble-free castings. The solution is formulated to gently reduce the surface tension of impressions prior to the pouring of models and wax patterns. This allows for a better flow and adaptation of the dental material being poured, virtually eliminating the formation of bubbles and voids on the surface of models and castings.



- #612180032 oz. (.95 L)
- #61219001 gallon (3.79L)

Sure-Flo

Sure-Flo is a liquid debubbler that's great for eliminating bubbles in models. Easy, ready-to-use and helps save time and money.



- #9046760Sure-Flo, 16 oz. (.48 L)

Yeti Wax Debubbler

Our Yeti Wax Debubbler reduces tension on all wax and also silicon surfaces. The investment material adapts precisely to the waxwork to ensure the best finish possible. **North American Sales Only.**

- #1860118Yeti Wax Debubbler, 100 ml

DENTURE ACCESSORIES

ProCure Base Plate Material

ProCure Base Material is a premixed composite wafer, ready for immediate adaption. They are easy to form, stable, and accurate. Ideal for making base plate for a variety of try-in situations. Roughened surface has excellent adherence to all waxes. The Procure base plates are packaged in a tub with 50 pre-made composite wafers per package.



- #9634940Base Plate Material

Diamond D® Ultra Sep

The Diamond D Ultra Sep is a state-of-the-art tin foil substitute that penetrates and adheres to stone and plaster. Then, the stone and plaster breaks away easily from the acrylic after processing. The Diamond D Ultra Sep will produce unsurpassed results and save time in denture finishing.



- #10101701 Gallon (3.79L)
- #10101751 Quart (0.95L)

Diamond D® Wax Gloss

Our Diamond D Wax Gloss leaves a beautiful glossy finish on all waxes, denture wax try-ins, and bite blocks. This will certainly add to the final presentation quality to wax setups. Available in 1 quart bottles.



#10101801 Quart (0.95 L)

Kefoil

Kefoil is a specially formulated multi-purpose liquid separator and is an excellent tin foil substitute. An alcohol-free substance.



#1360020Clear, 1 gallon (3.79L)

#1360025Pink, 1 gallon (3.79L)

Diamond D® Sep Brush

There are two types of Diamond D Sep Brushes. The Diamond D Sep Brush is used to apply tin foil substitute to stone and plaster surfaces. The Diamond D Sep Brush II is used to remove excess tin foil substitute from denture teeth.



#1170405Diamond D Sep Brush

#1170400Diamond D Sep Brush II

Palatinal Brush

The Palatinal Brush has a unique design that is ideal for polishing hard to reach palate area. The new designed brush if fitted on a tapered spindle and a plastic center with automatic centration. The brush is used with pumice for high shine.



#1670152Palatinal Brush

3 Post Reline Jig


This is a great little addition to our laboratory equipment. The 3 Post Reline Jig is perfect for repairs, relines and other dental applications. With three points there's better accuracy and more stability when using the jig. The item has a sturdy build and an easy removable top plate. Sold individually.



#11900143 Post Reline Jig

Reline Jig

This Reline Jig is a chrome two-post jig used for accurate reline applications. It's made of chrome plate zinc alloy and the posts features tapered tops to aid in upper plate alignment.

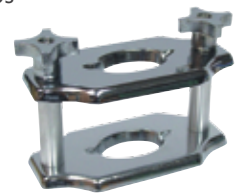

#1502400Reline Jig

Relief Metal

Our Relief Metal is soft and 25 gauge. The soft metal sheets are 1 lb. and 4" wide.



#1530090Relief Metal

Tin Foil

We have good quality Tin Foil that is 6" wide. The 1-pound roll comes in two different thicknesses: .001 and .002.



#1370010001 Thickness, Thin - 53 ft. (16.2 m)

#1370020002 Thickness, Thick - 26 ft. (7.9 m)

Plastic Sheets

Made of Veri-Thin material, .001 thickness. Available in 5" x 5" or 4" x 4" sheets.



#16100104" x 4" (101.6 x 101.6 mm)

#16100205" x 5" (127 x 127 mm)

LABORATORY

Duplicating Flasks

These Duplicating Flasks are used in producing full or partial dentures. High quality and long lasting.



- #1350090Raised Bottom
- #1350080Flat Bottom

Denture Flask

These Denture Flasks are forged in bronze to ensure a long and durable life. The flask halves fit tight together with stainless steel guide pins.

Flask Sizing:

Upper Dentures Flask: bottom section 5/8"

Deep Lower Dentures Flask: bottom section 5/8"

Deep at front, 1-1/8" at back



- #1350100Upper Denture
- #1350120Lower Denture

Flask Compress

This Flask Compress is used in curing a single flask. It is built of bronze alloy with stainless steel compensatory springs. The high-quality materials and build make it a long-lasting tool.



- #1350095Flask Compress

Tray Handles

These aluminum Tray Handles come in 100/Pkg.



- #1410010Tray Handles

Stainless Steel Clasp Wire



These Stainless Steel Clasp Wires are available in half-round and round configurations.



Half Round (5 x 1 ft. strips)

- #190072012 gauge
- #190073014 gauge
- #190074015 gauge
- #190075016 gauge

Round (10 x 1 ft. strips)

- #190076017 gauge
- #190077018 gauge
- #190078019 gauge
- #190079020 gauge

Stainless Steel Wire Clasps



These Stainless Steel Wire Clasps are anterior small. Items come in straight, left, and right. Available in 10/Pkg or 100/Pkg.



10/Pkg

- #1271160Anterior Small, Straight.....#1271390
- #1271170Anterior Small, Left.....#1271370
- #1271180Anterior Small, Right.....#1271380
- #1271190Bicuspid Medium, Straight.....#1271400
- #1271200Bicuspid Medium, Left.....#1271410
- #1271210Bicuspid Medium, Right.....#1271420
- #1271440Molar Large, Straight#1271470
- #1271460Molar Large, Left#1271490
- #1271450Molar Large, Right#1271480

100/Pkg

Stainless Steel Ball Clasps



Our Stainless Steel Ball Clasps come in two different sizes:

- Small - .028" dia. (.7mm)
- Large - .032" dia. (.8mm)



10/Pkg

- #1270010Large .032" dia. (.8 mm).....#1270020
- #1270005Small .028" dia. (.7 mm).....#1270006

100/Pkg

Clasp Kit

The Keystone Industries' Clasp kit has all your needs with 12 each of various clasps. You get 96 total clasps. The clasps in the kit are: 2L, 2S, 7L, 7R, 8, 9, 10, and 11.



- #1271500Clasp Kit



Stainless Steel Wire Clasps



Sizes 2L and 2S with rest
For clasping low-crowned tooth



10/Pkg		100/Pkg
#12710002L.....	#1271220
#12710102S.....	#1271230

Sizes 3L and 3S with rest
For clasping with scoop shaped rest



10/Pkg		100/Pkg
#12710203L.....	#1271240
#12710303S.....	#1271250

Size 13
For clasping a tooth undercut on the defect side



10/Pkg		100/Pkg
#127112013-O.....	#1271330

Double-armed clasp for bicuspid #4 and molar #5



10/Pkg		100/Pkg
#12710404.....	#1271260
#12710505.....	#1271270

Sizes 7
Clasp for anterior tooth with rest



10/Pkg		100/Pkg
#12710607L.....	#1271280
#12710707R.....	#1271290

Sizes 14
Back action usable for roach, back-action, ring and reverse type



10/Pkg		100/Pkg
#127114014L.....	N/A
#127115014R.....	#1271360

Stainless Steel Wire Clasps



Size 8 with rest
Clasp for molar



10/Pkg		100/Pkg
#12710808.....	#1271430

Size 9 with rest
Clasp for bicuspid



10/Pkg		100/Pkg
#12710909.....	#1271300

Size 10 with rest
Clasp for lower bicuspid



10/Pkg		100/Pkg
#127110010.....	#1271310

Size 11 without rest
Clasp for anterior tooth



10/Pkg		100/Pkg
#127111011.....	#1271320

Lingual Bars

Lingual bars are used for fabricating Gelb TMJ Splints, Bruxism appliances, lingual arches and partial dentures. The cut out ends afford positive retention. Available in 12/Pkg.



#1110040Small
#1110030Medium
#1110020Large
#1110010Extra Large


Grid Strengtheners



These Grid Strengtheners have round perforations (2.5mm) to assure excellent resin bond to establish a strong stabilized denture base for acrylics. Available in 12/Pkg.

#1800085Stainless Steel

#1800086Gold Plated

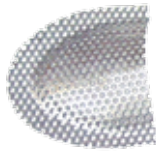

Strengtheners



These strengtheners are twisted, braided strips that are 3-1/2" (88.9 mm) long. Available in 10/Pkg or 100/Pkg.



10/Pkg

#1800025 Gold Plated#1810035

#1800035 Stainless Steel#1810045

100/Pkg

Frego Strengtheners



Gold plated upper mesh material. Sold individually.

#1800050Frego Strengtheners



Wire Mesh

We have varieties of wire mesh to meet all demands. We have a Stainless Mesh that is X-Fine and Med-Fine which are both 6" x 6". We also have a 4" x 4" Gold Mesh. Sold individually.



#1810050X-Fine Stainless Mesh, 6" x 6" (152.4 x 152.4 mm)

#1810060Medium-Fine Stainless Mesh, 6" x 6"

#1810070Gold Mesh, 4" x 4" (101.6 x 101.6 mm)

Impulse Bag Sealer - Model HS 3000

The Model HS 3000 is an Impulse Bag Sealer is a low cost solution. The easy-to-use sealer is a lightweight, portable table model that watertight seals thin as well as thick materials. When sealing, there are no hot surfaces, just plug in and use – there's no warm-up needed. The sealer is made of sturdy cast aluminum construction and allows for easy, timely temperature control.



#1440015Impulse Bag Sealer

#1440025Heating Element

Heat Sealing Bags

These Heat Sealing Bags are versatile and easy to use. Dental cases slip easily into the bags. Bags are 5" x 6" (127 x 152.4 mm) and .002" thick. Available in 500/Pkg.



#1450010Heat Sealing Bags

DIE & MODEL FABRICATION

Double Dowel Pins

Our Double Dowel Pins are state-of-the-art twin pins with die guards and sleeves. They are easy to use with a Pindex Machine. The pins add stability which eliminates rotation. They also are extremely precise with no distortion.



#1310055Double Dowel Pins, 1,000/Pkg

Reverse Dowel Pins

Reverse Dowel Pins are economical plastic stabilizing pins. They are tapered to insure fit with most drills. Best used when removing die is required (porcelain veneer).



#1310070Reverse Dowel Pins, 1,000/Pkg

Stick Dowel Pins

These Stick Dowel Pins are made of all brass.



#1310040Stick Dowel Pins, 100/Pkg

Swiss Dowel Pins

These Swiss Dowel Pins are made of top quality brass.



#1310011Small, 100/Pkg

#1310010Small, 1,000/Pkg

#1310021Medium, 100/Pkg

#1300015Medium, 1,000/Pkg

J-Pin™

This J-Pin is a die locating device with positive 2-point indexing. The all-brass piece is made with retention wire. It can easily be bent to 180-degrees and fixed into an impression.



#1310200Small Anterior (805-JT), 500/Pkg

#1310210Medium Universal (805-JS), 500/Pkg

Tunnelmate

Tunnelmate is quick and easy to use. By reducing labor costs and time, it's an ideal product to have in a laboratory. Available in 12/Pkg.



#1290275Tunnelmate

Adjustable Saw Frame

This Adjustable Saw Frame is for unpinned blades. Has a nice ergonomic wood handle and solid construction.



#1380010Adjustable Saw Frame

Offset Pin Saw Frame

This offset Pin Saw Frame uses 5" (127 mm) blades. Also available is an Opti-Saw (off-set) with 12 blades.



#1140285Offset Pin Saw Frame
#5940285Saw with 12/Pkg Blades

Pin Saw Frame

This Pin Saw Frame is for 5" (127 mm) blades. Frame is adjustable.



#1140280Pin Saw Frame

Swiss Cut Offset Saw Frame

The teeth of this Swiss Cut Offset Sawframe are machined by our precision setting process that guarantees a cutting performance unequalled by any other blade in the world. The blades are made of the finest alloy steel that lasts longer and stays sharper. The rounded back design makes it easy to cut straight.



#1140290Swiss Cut Offset Saw Frame

Saw Blades

We offer various pinned Saw Blades. The sharp, precise blades have 18 teeth per inch.



Pinned 5" (127 mm) length, .055" wide, .010" thick

#1140170Blue, 100/Pkg #1140160Blue, 10/Pkg

#1140175Brown, 100/Pkg

Pinned 4" (101.6 mm) length, .065" wide, .010" thick

#1140190Blue, 100/Pkg #1140180Blue, 10/Pkg

Pinned 5" (127 mm) length, .110" wide, .007" thick

#1140250Blue, 100/Pkg #1140240Blue, 10/Pkg

Boley Gauge

The Boley Gauge is the definitive dental tool used for a variety of procedures both in the laboratory and clinically. It's perfect for determining dimensions of length, width, and thickness in millimeter increments.



#1400010Boley Gauge

Double Sided Caliper

These Double Sided Metal Calipers are a perfect calibration instrument for metal or wax.



#1210030For metal

#1210040For wax

Iwansson Calipers

Iwansson Calipers are a double sided caliper for either metal or wax.



#1210050For metal

#1210060For wax

Dial Caliper

This precise Dial Caliper has a full measuring range from 0-10 mm. The scale is clearly visible due to an oversized face for easy reading. Simple to operate, saves time and effort.



#1210005Dial Caliper

Die Spacer

These Die Spacers are fast drying and heat resistant. It provides an excellent uniform seating of the restoration. Comes in 1 oz. bottles.



#1028520Orange

#1028524Yellow

#1028523Blue

#1028525Silver

#1028527Thinner

NewArea Die Spacer

Various die spacer in 15ml bottles.

International Sales Only.



#1911600Clear

#1911603Silver

#1911601Blue

#1911604Gold

#1911602Red

#1911605Thinner

Yeti Die Spacer

Yeti Die Spacer seals plaster and serves to stimulate the space needed for fixing cement between the crown and the die. This item is able to fulfill even the highest demands. Comes in 20ml bottles. **North American Sales Only.**



#1860006Clear

#1860070Blue

#1860005Red

#1860136Dentin

#1860004Yellow

#1860216Silver

#1860007Gold

#1860225Thinner

LABORATORY

Yeti Steam Off Spacer

Steam Off Spacer evaporable lacquers are available in 5 dentin colors and one bleach color for all standard shades. This results in an easy staining full ceramics. The product also provides easy fitting of the inlay and the spacer can be steamed off to balance out of expansion of pressed ceramics. **North American Sales Only.**



- #1910562Set, 6/Pkg
- #1910563Vanilla
- #1910565Sahara
- #1910567Gold
- #1910569Sand
- #1910571Terra
- #1910573Snow

NewArea Special Liquids

Various special liquids in 15ml bottles.

International Sales Only.



- #1911600Contact Marker, Red
- #1911601Contact Marker, Blue
- #1911602Thinner for Contact Marker
- #1911603Water Spacer, Grey
- #1911604Water Spacer, Beige
- #1911605Isolating Lube Die, Wax
- #1911605Uni Die Hardener

Yeti Light Spacer

Yeti Light Spacer is a great curing liquid. Available in 3 micron Transparent, 10 micron red, 10 micron blue. Comes in a 10ml bottle with application brush.



North American Sales Only.

- #1860213Transparent 3 Micron, 10 ml
- #1860214Red 10 Micron, 10 ml
- #1860215Blue 10 Micron, 10 ml

Yeti Die Hardener

This Die Hardener cures, seals and creates a smooth plaster surface. The hardener is absorbed by gypsum and does not build up any dispersion layers. Use on wax-copings yields the best results. The hardener is also stain and water repellent.

North American Sales Only.

- #1860056Die Hardener, 250 ml



Yeti Blue Marker

Yeti Blue Marker is a water soluble high spot indicator. The marker leaves a thin even layer on the plaster die and can be easily washed off. It dries quickly and covers well. Also adheres to metal, ceramics, plaster and synthetics. **North American Sales Only.**



- #1860130Blue Marker 20ml with Thinner, 2/Pkg
- #1860106Special Thinner 20 ml, 3/Pkg

Yeti Dia Glaze

Dia Glaze is a polishing medium that obtains perfect surfaces of ceramic restorations. If ceramic is fired at too high of temperature in order to obtain a gleaming surface, it can appear artificial inside the mouth. Then, natural looks can only be achieved through polishing. **North American Sales Only.**



- #1860053Dia Glaze, Set

Yeti Sakura Pencil



Yeti Sakura Pencil is a reliable blue marking pencil for the margin line. Convenient and can be used on plaster. Pencil and refill are sold separately. **North American Sales Only.**

- #1860010Pencil, each
- #1860011Refill, 12/Pkg

Yeti Die Bloc

Yeti Die-Bloc is an economical, yet a high performing light cure blocking out material. It is suitable to be used with all spacers. The Die Block Paste comes in a 3g containers and the Die Block Liquid comes in 2ml. **North American Sales Only.**



- #1860387Die Bloc Paste, 3g
- #1860388Die Bloc Liquid, 2ml

ProCure Block Out Material

ProCure now has a low viscosity, pigmented light-cured resin in syringe form that is excellent for blocking out models. This unique resin can be used in a variety of situations including bleaching reservoirs, die prep situations, and for filling voids and chips in prepared models. The kit comes with 5 pre-filled 1.2 cc syringes and 15 tips. Tips can be ordered separately.



- #9637020Block Out Material

Slaycris Slik Die Lubricant

Slaycris Slik Die Lubricant is good for all types of C&B work. It's water soluble and especially useful in reversible hydrocolloid technique.



- #1287105Slik Die Lubricant, 4 oz. (.12 L)

Die Lube

This Die Lube is a quick-dry lubricant. It flows smoothly on any surface and contains no mineral oil. It evaporates quickly into a microscopic thin film that doesn't lose detail or fit.



- #1287000Die Lube, 4 oz. (.12 L)
- #1287100Die Lube, 32 oz. (.95 L)

Plug-It

Plug-It contains a special cyanoacrylate material of accelerator and base that permits a controlled setting time. The item is useful for filling air bubbles, basing up decay areas and short cusps, stabilizing dies during finishing procedures, and connecting bridge segments.

Kit contains: base, accelerator, solvent and pump.



- #1028675Kit
- #1028680Base, .5 oz.
- #1028685Accelerator, 2 oz.
- #1028695Accelerator, 1 quart
- #1028690Solvent, 1 oz.
- #102868116 gauge applicator tip

Permabond 910

Permabond 910 is the original 100% methyl cyanoacrylate adhesive that adheres excellently to metal surfaces. The low viscosity liquid allows rapid curing at room temperatures when pressed into a thin film between two parts. It will cure to fixture strength in 10 seconds on most surfaces and rapidly develops high strength with fully cure obtained in 4 hours.



- #1020050Permabond 910, 1 oz.

Keybond

This Keybond has a thin viscosity for hard-to-bond surfaces.



- #1020065Keybond, 20g

Tri-Dynamics Tri-Epoxy

Tri-Dynamics Tri-Epoxy is a new advanced formula that is dimensionally stable and aesthetically pleasing. It's easy to release from the mold. All boxes include 12 epoxy resin jars, 12 epoxy hardener syringes & 12 stirrers.



- | | 1 lb. Bulk | 30 grams | 50 grams | 100 grams |
|------------|---|----------------|----------------|---------------|
| Pink..... | #5920400 | #5920420 | #5920440 |#5920460 |
| Blue..... | #5920405 | #5920425 | #5920445 |#5920465 |
| Green..... | #5920410 | #5920430 | #5920450 |#5920470 |
| Gray..... | #5920415 | #5920435 | #5920455 |#5920475 |
| White..... | #5920390 | #5920438 | #5920458 |#5920480 |
| | | | | |
| #5920550 |Bulk Hardener, 10 oz. | | | |
| #5920555 |Filled Hardener Syringe, 3 cc, 12/Pkg | | | |
| #5920560 |Filled Hardener Syringe, 5 cc, 12/Pkg | | | |
| #5920565 |Filled Hardener Syringe, 10 cc, 12/Pkg | | | |

EQUIPMENT

Heat Regulated Pressure Pot

The Keystone Dental Pressure Pot is specifically designed with dental laboratory use in mind. Recommended for pour, self curing, and ortho resins. Although each pot is factory pre-set for use at 20 psi, it is designed with an external pressurization regulator for manual adjustment to use lower pressures. For user safety, a valve is set for depressurization of the pot when filled over 25 psi. A manually adjusted heating element allows for a full range of temperature settings. Excellent for newer flexible and heat cured acrylics.

- 8 quart capacity (4 pour flasks)
- Quick connect air line connector
- Countertop design
- 120 volt
- Variable temperature control
- Variable pressure control
- Limited 1-year warranty



- #9622761Heat Regulated Pressure Pot

Pro-form Pressure Pail

The Proline Pressure Pail has a lot of excellent features that other pressure pots do not. Internal pressure can be generated either hydraulically using water or pneumatically using air from a compressor. The Pressure Pail is large enough (6" diameter by 6" deep), to hold a standard reline jig or smaller articulator. Easy crank handle, replaceable O-ring, and all-aluminum construction makes this pot excellent for curing all types of cold-cured acrylics. Use for repairs, relines, splint production, retainers and more. Airline adapter sold separately.



- #9622760Pressure Pail
- #9622780Airline Adapter

Denture Curing Unit

- All stainless construction for easy cleaning and anti-corrosion
- Can be used for both boil out and curing
- Thermostatic control for variable holding temperatures
- 115 volt/1380 watt
- One piece unit sealed, no seams that leak
- Cures up to 4 double flask compresses (8) denture arches
- UL listed
- Valve/spigot for easy emptying of water



- #1050159Denture Curing Unit

LABORATORY

ProCure 300 Light Oven

An advanced third generation light curing unit that provides superior lighting intensity and excellent depth of cure. The ProCure 300 has been designed to provide the maximum in light intensity without the negative heat build up.



This new generation light features:

- A highly reflective chamber for uniform light coverage
- A rotating glass platform to ensure uniform light
- A built-in fan to maintain optimum temperature to eliminate warping
- Two powerful halogen bulbs to ensure excellent intensity and depth of cure
- A protective welder's shield for eye protection
- Automatic cut-off switch for ease of use and eye protection
- One-year warranty
- Only in 110V

#9634590ProCure 300 Light Oven

#9634600ProCure 300 Light Oven w/ Sample Pack

Pro-form Vibrator

The Pro-form Vibrator is a heavy-duty vibrator powerful enough to be used in all types of dental laboratories but small enough to work great in a dental office. Heavy-duty steel spring construction provides maximum vibration and minimal noise. The unit has a weighted aluminum base for stability, a four-inch diameter rubber top, a three-speed control switch, and rubber suction feet to hold the unit in place.



#7000352Pro-form Vibrator

Ultra Sonic 2300 General Purpose Cleaner

This General Purpose Cleaner is an ultra-high performance general purpose cleaner/ degreaser. The cleaner is ammonia free. 100 gram package makes 1 gallon of solution.



#1850060Ultra Sonic 2300

Ultra Sonic 2400 Plaster & Stone Remover

The Ultra Sonic 2400 is an excellently performing plaster and stone remover. The remover completely dissolves dental plasters and stone, as well as casting and soldering investments.



#1850065Ultra Sonic 2400, 1 gallon (3.78 L)

Power Wash

The Power Wash requires no plumbing. Just plug it in, controlled water pressure! It draws clean water above settled dirt and below floating wax.



#1690010Power Wash

The Trap

The Trap's top drain valve and all metal construction eliminate the need for costly plumbing, clean-ups and repairs. The compact design of the trap is ideal for all dental lab and dental office sub-sink areas.



The built in viewing window eliminates the guess work of when it's time to change the liner. Comes with a 10-year warranty.

#7000360 The Trap

#7000364 J Bolt

#7000402 Trap Liners, 3/Pkg

#7000401 Trap Liner

#7000403 Trap Liners, 6/Pkg

#7000361 Trap Gasket

Dispos-a-trap

The Dispos-a-trap combines convenience with economy for laboratories. These are safe, reliable and sanitary containers. The containers are translucent to allow for easy monitoring of waste levels.



#7000365 3-1/2 gal. Complete

#7000366 5 gal. Complete

#7000367 3-1/2 gal. (bucket only)

#7000368 5 gal. (bucket only)

#7000369 Hoses only

Harris Oxygen Regulator

We offer two sizes of Harris Oxygen Regulators.

The large regulator is for G and H tanks while the small regulator is for D and E tanks.



#1821090Large (G&H tank)

#1820195Small (D&E tank)

Air Gun

The Air Gun is designed to eliminate the overheating of restorations, grinding tools, or fingers, while providing a clear view of the immediate working area.



#1030010Air hose with Nozzle

#1030020Air Hose only

#1030030Nozzle only

Optimist Ceramic Spray System

The Optimist Ceramic Spray System allows technicians with minimal experience to do all opaquing. It's quick and efficient to operate.



- #5920280Complete Kit (includes everything listed below plus 18 jars)
- #5920282Airbrush only
- #5920283Jar Adapter
- #5920284Air Filter and Moisture Trap
- #592028510 ft. Recoil Air Hose
- #5920286Airbrush Wrench
- #5920288Airbrush Liquid, 1 quart
- #5920290Regulator
- #5920287Empty 3.25 oz. jar

Hydro-air Carver

The Hydro-Air Carver is a fully pneumatic air turbine with or without water spray. The Hydro-Air Carver is a versatile, compact unit with a lightweight handpiece for precise shaping of hard materials. A convenient switch on the handpiece allows the water spray to be switched on and off quickly.

Air and water regulator controls on the front of the main unit allows for fine adjustments of the spray system. The spray feature traps dust, while protecting and prolonging the life of the grinding tools. The same cooling spray prevents overheating of the material being worked on, thereby avoiding the risk of fracture or discoloration.

The Hydro-Air Carver consists of base with filter, pressure regulator, manometer, water container and spray regulators; On/Off foot control; Handpiece and Rotor cartridge (plus spare rotor cartridge).

Technical Data:

- Speed 300,000 rpm
- Operated by compressed air
- Operating pressure 2.8-3.2 bar
- Water tank 350 ml
- Chuck diameter 1.6 mm
- Lubricate manually (special lubricant is provided)
- Warranty: Hydro-Air Carver/Foot-control: 12 months; Handpiece: 12 months; Rotor cartridge: 3 months.



#1530175Hydro-Air Carver

Acculab Model V-400

The Acculab Model V-400 is a versatile, yet accurate weighing machine. The V-Series is a portable top loader. The machine features auto calibration, tare, four weighing modes, parts counting, audio function tone, large stainless steel platform. The AC adapter is included with a built-in lock down bracket, large LCD display, spring ring and easy to use.



- #1500160110 Volt
- #1500166220 Volt Adapter Only

Mini Scale

The Mini Pocket Scale is easy and convenient to use whenever and wherever. The pocket-sized digital electronic scale has a high precision strain gauge sensor. It is battery operated. The scale is excellent for weighing wax coping from pressable ceramics and all dental alloys.



#1050090Mini Scale

Keystone Three-Well Wax Pot

The Keystone Three-Well Wax Pot is an economical, electric wax heater with three separate compartments. This is excellent for multiple wax build-ups. The temperatures are controlled (between 50F and 350F) and the machine is designed for easy access to waxes. Comes with a 1-year warranty. Machines are available at 110V and 220V.



- #70003453-Well Wax Pot, 110 Volt
- #70003463-Well Wax Pot, 220 Volt

Yeti Dippy NT Light

The Yeti Dippy NT Light has a transformer built-in unit. The efficient device is an alternative unit that's reasonably priced. There's no need for an additional transformer that takes up a lot of space. It's easy to control the heating system around the tank and has an ergonomic design. **North American Sales Only.**



#1860274Yeti Dippy NT Light

LABORATORY

Yeti Dippy Pro Waxer Digital

The Yeti Dippy Pro Digital Dipping Unit is a convenient alternative machine to analog technology. Precisely adjust temperature from 149°F to 212°F while providing an even temperature distribution. Ventilation slits add improved ventilation. Additionally, a digital display shows the actual temperature. **North American Sales Only.**



#1860088Yeti Dippy Pro Digital, 115V

Yeti Pro Waxer Duo

The Pro Waxer Duo is a processor controlled waxer with a two line LCD display. The waxer proves precise and easy control. The display shows the set and the actual temperature of both handpieces at the same time.

The Pro Waxer provides the ability to adjust the temperature of each handpiece, while automatically memorizing of the latest data adjustments, even after power failure. **North American Sales Only.**



#1860001Yeti Pro Waxer Duo

Yeti Modeling Tips for Waxers

Modeling Tips for Waxers are tips easily changed due to a simple pin system. Works great and economical. **North American Sales Only.**



#1860019PKT 1, Thin

#1860020PKT 2, Thick

#1860021Hollenback Horizontal

#1860029Knife Shape

#1860031Corkscrew, Thin

#1860032Corkscrew, Thick

Plastic Belts

These Plastic Belts are extremely smooth running. They will outlast most conventional cotton belts and comes in various sizes to fit all needs.



#1475105 6'4" (1.93 m) #14751239'1" (2.77 m)

#1475110 6'6" (1.98 m) #14751259'2" (2.80 m)

#1475119 8'4" (2.54 m) #14751409'8" (2.95 m)

#1475145 8'6" (2.59 m) #14751509'10-1/2" (3 m)

Cotton Belts

Our Cotton Belt is a quality four-ply cotton belt, which are endless woven.



#14700106'6" (1.98 m)

#14700309'6" (2.90 m)

#14700409'8" (2.95 m)

Super Belts

These Super Belts are vulcanized, nylon fiber reinforced belts that offer superior quality and durability. The belts are under a 1-year warranty.



#14701006'6" (1.98 m)

#14701109'6" (2.90 m)

#14701209'8" (2.95 m)

Plastic Beaker

This is a standard 400 ml Plastic Beaker. Used for handling and storing strip-it and other lab chemicals.



#1622706Plastic Beaker

Plastic Liquid Dispensers

These plastic liquid dispensers are used to dispense Stripit, ceramic and acrylic liquids, light oils, non-flammable cleaning solutions and distilled water. The dispenser bottle is an exclusive molded one-piece cap and angle spout. They are simple, reliable and durable. Flow can be adjusted from single drops to razor-sharp stream by simply trimming the spout end.



#1655410125 ml

#1655412250 ml

INSTRUMENTS

Adjustable Saw Frame

This Adjustable Saw Frame is for unpinned blades. Has a nice ergonomic wood handle and solid construction.



#1380010Adjustable Saw Frame

Swiss Cut Offset Saw Frame

The teeth of this Swiss Cut Offset Sawframe are machined by our precision setting process that guarantees a cutting performance unequaled by any other blade in the world. The blades are made of the finest alloy steel that lasts longer and stays sharper. The rounded back design makes it easy to cut straight.

Our unique temper process allows the blade to be incredibly flexible and long lasting. The pinned blade size is: 5" length, .055" wide, .010" thick.



#1140290Swiss Cut Offset Saw Frame

Wheel Saw

This Wheel Saw is a high quality cutting wheel used for hard sheet resins and dual laminates. The cutting surface is on the edge that contacts the plastic. Fits most slow speed handpieces.



#9630100Wheel Saw

Blade Handles

These Blade Handles have an ergonomic shield design that provides a non-slip grip. The handles minimize the risk of injury during assembly, passing, disassembly and disposal. The handle has one-handed activation of the protective shield while still featuring pencil, palm and finger grips. Accommodates both left- and right-handed use and operation.



#1130090Metal Size 3 for #10, #11, #15 blades
 #1130110Green Plastic Size 5 for #10, #11, #15 blades
 #1130120Red Plastic Size 6 for #25 blades

Bard-Parker Blades


These Bard-Parker Blades are interchangeable and made from quality carbon steel. The long lasting blades have uniform sharpness and come in several sizes and styles.



#1130010#10, Dozen #1130050#10, 150/Box
 #1130020#11, Dozen #1130060#11, 150/Box
 #1130030#15, Dozen #1130070#15, 150/Box
 #1130040#25, Dozen #1130080#25, 150/Box

Plaster Knives

Our Plaster Knives are made of high quality carbon steel with wood handles.

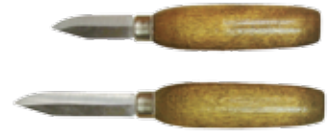

#1490030Short Blade, 1-3/4" (44.4 mm) long
 #1490040Long Blade, 2-1/2" (63.5 mm) long

Handy Knife

The Handy Knife fits in palm of hand and is useful for carving and many other applications.



#1490010Handy Knife

Grifhold Knife

All Grifhold products are made to the highest specifications and priced competitively for consistent savings. Grifhold has earned international recognition for delivering high-quality custom knives, blades and specialty hand tools.



#1130180#24
 #1130185Replacement Blades, 5/Pkg

Straight Handle Chisel

This Straight Handle Chisel is #15 Left.



#1250050Straight Handle Chisel

Scrapers

Scraper #3 is a sturdy instrument with a solid wooden handle. Scraper #6 is a sturdy instrument with a solid wooden handle. Sold individually.



#1760030Scraper #3
 #1760060Scraper #6

Plaster Nippers

These Plaster Nippers are strong enough to cut plaster and stone. Available in stainless steel or carbon steel.



#1550030Stainless Steel
 #1550020Carbon Steel

LABORATORY

Crown & Bridge Scissors

Our Crown & Bridge Scissors are durable and provide comfort to the user. This dental instrument is made of stainless steel and 4" long, providing the dental professional with a reliable scissor.



#1750010Straight Scissors

#1750020Curved Scissors

Hemostats

We sell several different types of Hemostats to fit all needs.



#1460030Kelly Straight 5-1/2" (139.7 mm)

#1460040Kelly Curved 5-1/2" (139.7 mm)

#1460010Mosquito Straight 5" (127 mm)

#1460020Mosquito Curved 5" (127 mm)

Utility Scissors

These All-Purpose Utility Scissors are great for bulk trimming of thermoplastics, especially Pro-form mouthguards. The easy grip handle and stainless steel construction is very handy throughout the lab or office.



#9139720Utility Scissors

Scissor Type Crown Holder

This Scissor Type Crown Holder is made of stainless steel and interchangeable diamond tips. Crowns and bridges are gripped within the crown. The diamond tip ensures that the grip contact is made above the crown margin.



#1620046Scissor Type Crown Holder

Tweezer Type Crown Holder

These Tweezers are made of stainless steel. By using them there is no danger of damage to the crown margins when the restoration is gripped securely.



#1620045Tweezer Type Crown Holder

College Type Pliers

The College Pliers #871 is a cotton and dressing pliers. It's 6" long with serrated tips.



#1620010College Type Pliers

Spatula

These Cement Poliamide Spatulas are sold in a 10/Pkg.

#1150051Spatulas, 10/Pkg



Bar Bending Pliers

These Bar Bending Pliers are size #100.



#1620025Bar Bending Pliers

Clasp Bending Pliers

These Clasp Bending Pliers are 3 pronged, size #200.



#1620020Clasp Bending Pliers

Carvers

These Carvers are ideal for carving margins and interproximal areas on wax bridges. They come in a variety of sizes and styles.



#1780020Baltimore Spatula #7.5



#1780010Grittman Spatula #31



#1780030Cement Spatula #24



#1780040Wax Spatula #7



#1220100Vehe Carver



#1220070LeCron Inlay Carver #5



#1220080Modified Roach



#1220110Ward Carver #1



#1220090Roach Carver



#1220050Hollenback Carver #3

#1220060Hollenback Carver #3S

Akers Carver


The Akers Carver is a precision instrument for wax caring and setup work. It comes complete with blade and spoon.



#1220010Akers Carver

Ceramic Sculpting Set

The Ceramic Sculpting Set is made of lightweight stainless steel. The easy grip makes it comfortable for technicians. Sets come in sleek and convenient storage case.

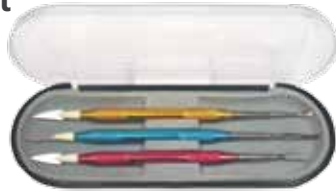

- #5260094Yellow handle with gingival blade
- #5260095Red handle with incisal blade
- #5260096Blue handle with knife back
- #5260090Complete Kit with all 3 instruments

Replacement Tips

- #5260091#1 Gingival Blade, yellow
- #5260092#2 Incisal Blade, red
- #5260093#3 Knife Back, blue

Magna Bar

The Magna Bar is a wood base with magnetic bars for holding burs, mandrels, lab tools, etc.

- #151001012" long
- #151002018" long
- #151003024" long



Boley Gauge

The Boley Gauge is the definitive dental tool used for a variety of procedures both in the laboratory and clinically. It's perfect for determining dimensions of length, width, and thickness in millimeter increments.

- #1400010Boley Gauge



Dial Caliper

This precise Dial Caliper has a full measuring range from 0-10 mm. The scale is clearly visible due to an oversized face for easy reading. Simple to operate, saving time and effort.



- #1210005Dial Caliper

Iwansson Calipers

Iwansson Calipers are a double sided caliper for either metal or wax.

- #1210050For metal
- #1210060For wax



Double Sided Caliper

These Double Sided Metal Calipers are a perfect calibration instrument for metal or wax.

- #1210030For metal
- #1210040For wax



LAB SAFETY

Face Masks

These Face Masks are surgical quality and disposable. They have a rated protection of 90% at 5 micron particle size.



- #1520100Face Masks

Respirators

There are two types of respirator masks: one for dust and one for gas. Both are NIOSH/MSHA Approved and meet OSHA Guidelines.

The Blue Respirators (8246) are used for dust/mist protection and where solvents and organic vapors are a bother. The Gray Respirators (8247) are used for acid gas protection (chlorine, hydrogen and sulfur dioxide). They also provide dust/mist protection.



- #5920685Blue (8246)
- #5920690Gray (8247)

Goggles

These heavy duty Goggles are recommended for use with acids. They are flexible with a Lexan polycarbonate lens. ANSI Approved Z87-1-1979.



- #5920655Goggles

Eyewear

We offer both clear protective safety glasses and UV Protective Safety Glasses. Both are made of polycarbonate single piece lens construction. The unique spherical design delivers a distortion-free view even in peripheral vision areas. These glasses are secure and have a comfortable fit even when wearing over prescription glasses. These are an economical choice while still providing needed safety precautions. The UV glasses filter more than 99% of UV and dangerous blue light (up to 540nm).



- #36-00110.....Clear Protective Safety Glasses
- #36-00210.....UV Protective Safety Glasses

LABORATORY

Overgloves

These Overgloves are a great alternative to the more expensive latex gloves. They can be used with procedures that do not require tactile sensitivity such as opening incoming cases, disinfecting casework and general surface cleaning.



- #9045660Large, 1,000/Box
- #9045670Medium, 1,000/Box
- #9045680Small, 1,000/Box

LAMP/MAGNIFIER

Megaview

The Megaview provides a wide field of vision and is lightweight. This is a binocular head mount loupe with flip up lenses for easy use and vision. The adjustable headpiece can be worn with regular glasses and has interchangeable lenses for different magnifying strengths.



- #1400080Megaview #14000822.0x lens
- #14000811.7x lens #14000832.5x lens

Headband Magnifier

This Headband Magnifier gives 4x power and has dual side lights. One touch design switches between 1.8x, 2.3x, 3.7x and 4.8x magnification. The piece of equipment is lightweight, weighing in at 110g.



- #1150120Headband Magnifier

Head Mounted Loupe

The Head Mounted Loupe provides a wide field of vision with four interchangeable lenses. The head mount's compact, extra-strength light provides added illumination. It also can be worn with glasses.



- #1400085Head Mounted Loupe

MISCELLANEOUS

Blue Safe Acid Neutralizer

Blue Safe Acid Neutralizer avoids acid-related accidents in the dental lab or office. Before disposing acid, use Blue Safe to neutralize the acid. It can be used on any type of acid. Just scoop and sprinkle the neutralizer powder onto the acid – wait for the color to turn yellow to dark blue.



- #5920670Blue Safe Acid Neutralizer, 3 lb.

Nitric Acid

Our Nitric Acid is a Reagent-grade acid that meets all etching requirements. The .5N nitric acid is to be used with biobond, biocast and other non-beryllium alloys.



- #5920635Nitric Acid

Pattern Adhesive

This Pattern Adhesive is a special liquid formulated to provide better adhesive characteristics for plastic preform patterns. Available in a 4 oz. (118cc) bottle with applicator brush cap.



- #1600480Pattern Adhesive

Stripit

Stripit removes oxidation and contamination from both semi-precious and precious metals. Available in 1-gallon bottle. The formula also is used for various other applications:

- Removes investments from high and low heat castings and soldering flux
- Removes porcelain and opaque from coping and frameworks
- Prevents bubbling of ceramic structure due to contamination from inorganic silicates imbedded in castings
- Reduces corrosion of metal parts in the laboratory which may be caused by acid fumes



- #16559004 quarts (3.78 L)

Relief Metal

Our Relief Metal is soft and 25 gauge. The soft metal sheets are 1 lb. and 4" wide.



- #1530090Relief Metal

P.D. Attachments

These Precision Dowel Attachments are easy to adjust. The male and female dowels are interchangeable and both have a 2-degree taper on the sides and 1-degree on the face.



#1631701Small Set

#1631702Large Set

POLISHING

Polishing Compounds

Composite Polish

Produces a quick, high shine on all plastic restorations.

#16601001 lb. bar (.453 kg)



Red Rouge

Used for final polish on gold.

#16600701 lb. bar (.453 kg)



Tripoli

Used for removing scratches in preparation of denture materials and metal prior to final polish.

#16600302-1/2 lb. bar (1.13 kg)



Bobbing Compound

Fast cutting action on metals (Jewelers Polish).

#16600801 lb. bar (.453 kg)



Yellow High Shine

Used to create exceptional high gloss on denture materials.

#16600902-1/2 lb. bar (1.13 kg)



White Diamond Compound

Used for a clean, dry, fast pre-polish on plastics.

#16600401 lb. bar (.453 kg)



Green Plate Polish

A high shine compound for use on acrylic denture materials.

#16600503 lb. bar (1.36 kg)



Shure Shine Plate Polish

Best selling polish for use on acrylic denture materials.

#16600602-1/2 lb. bar (1.13 kg)



Hatho Polishing Compounds

Brown - Designed for prepolishing of precious metals. Use with lathe brushes, Toro miniature brushes, Geza and Poly-Buffs. Ideal for polishing deep scratches. Prepolishing of crown and bridge work.

#1670109 100g



Green - Designed for high shine polishing of metals and chrome cobalt framework. A very fine compound used for final polish. Initial surface must be prepared with a prepolysh. Use with muslin, chamois, micro-fiber leather or flannel buffs.

#1670094 100g

#1670121250g



Blue - Designed for high shine polishing of precious, non-precious metals and light curing composites.

A universal compound for high shine polish. Special high grade abrasives inside, similar to diamonds. Use with muslin, chamois, micro-fiber, leather or flannel buffs.

#1670095 100g

#1670123250g



Beige - Designed for high shine polishing of acrylics and polymer based crowns & bridges. A universal compound for high shine polish. Initial surface must be prepared with a pumice prepolysh. Use with muslin, chamois, micro-fiber, leather, flannel or felt buffs.

#1670110 100g

#1670124250g



White - Designed for fine prepolishing of gold alloys and chrome cobalt framework. Use with lathe brushes, Toro miniature brushes, Geza and Poly-Buffs. Provides a smooth finish.

#1670093 100g

#1670120250g



Red - Designed for final high shine polishing of gold alloys. The least abrasive Hatho polishing compound.

Initial surface must be prepared with a prepolysh. Use with muslin, chamois, micro-fiber, leather or flannel buffs.

#1670114 100g


#1670122250g



Diamond Polish - Designed for high shine polishing of precious and non-precious metals, titaniums, acrylics, composites and ceramics.

The fastest and most efficient method to polish every dental material. Initial surface must be prepared with a prepolysh. Use with miniature brushes combined with polishing discs, geza and Toro brushes or muslin, chamois, micro-fiber, leather, flannel and felt buffs, micro-fiber, leather or flannel buffs.

#1670096 2.5g

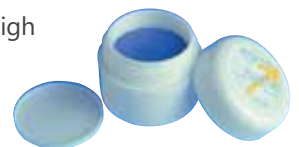

LABORATORY

Hatho Acrypol Polishing Kit

The Hatho Acrypol Polishing Kit is for efficient high-shine polishing of acrylic dentures and teeth. The kit includes: 2 Poly-Buffs Chung King Lathe Brushes with Scotch Brite.



#1670133Acrypol Polishing Kit

Hatho Polistar Polishing Kit

Polishing set for an efficient high-shine polishing of acrylic dentures and teeth. The product is silica free.

Kit Includes: 300g of Polistar Lintgrey, 125ml Polistar Emulsion, and 1 Cotton Buff



#1670200Polistar Polishing Kit

Hatho Thermoplastic Polishing Kit

Polishing set designed for an efficient high shine on all flexible partial materials.

Kit Includes: 2 Hatho Flex Polishers, 1 Mira Buff, 1 Yellow Mira Buff 1 Polistar Compound and 1 Polistar White Compound



#1670160Thermoplastic Polishing Kit

Mizzy Mint Flavored Diamond Polish

Mizzy Mint Flavored Diamond Polish is formulated to polish all restorative materials to a glaze high shine in seconds. It's highly recommended for restoring the glaze finish on ground porcelain, tooth or composite surfaces. The polish will shine all precious or non-precious metals to a smooth mirror-like finish.



#6121400Mint Flavored Diamond Polish

Laboratory Pumice Powder


Our Laboratory Pumice Powder is the best quality powdered pumice available. The pumice is available in four grades of coarseness and three different sizes.



	5 lb.	25 lb.	50 lb.
Flour FF.....	#1700011.....	#1700016.....	#1700040
Fine 0 1/2	#1700008.....	#1700013.....	#1700017
Medium 0 3/4.....	#1700009.....	#1700014.....	#1700018
Coarse 1	#1700010.....	#1700015.....	#1700030

Porte Polishers

Provides excellent cleaning of interproximal and other difficult areas. Brushes are available in 12/Box. Polishers have a HP shank.

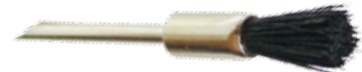

#1650010Porte Polisher

PORCELAIN ACCESSORIES

Insta-Frac

Insta-Frac is an instant refractory material used for repairing and adding to ceramic crowns, porcelain laminates, Dicor crowns, inlays, onlays, etc. It's also used for making custom firing pegs and all types of porcelain add-ons, except margins. The material is carbon free and non-contaminating. Can be used up to 3,200-degrees Fahrenheit.



#5920305Insta-Frac

Furnace Tongs

These Furnace Tongs are 15" (381 mm) long to provide ideal reach. The scissor-type tongs can securely grab dental work from the furnace.



#1620037Furnace Tongs

Porcelain Tongs

These Porcelain Tongs are 12" (304.8 mm) long to help pick up and place precious dental materials.



#1620035Porcelain Tongs

Wire Pins


Wire Pins are single individual firing pins that can also be used for positioning work on the tray. Pins come in assorted 6/Pkg.



#1680098Wire Pins, 6/pkg assorted

Metal Pins

We offer several types of metal pins for dental work.

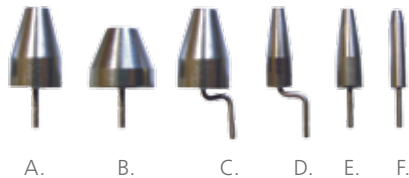

- #1680091A. Posterior Metal Pin Peg, 5/Pkg
- #1680096B. Molar Pin Peg, 5/Pkg
- #1680092C. Posterior Swivel Pin Peg, 5/Pkg
- #1680094D. Anterior Swivel Pin Peg, 5/Pkg
- #1680093E. Anterior Metal Pin Peg, 5/Pkg
- #1680089F. Metal Round Head Pin Peg, 6/Pkg
- #1680087Assorted (A, B, C, D, E), 5/Pkg

V Pegs

V Pegs are single firing pins that can be taken out from the honeycomb base tray. Single individual firing pins can also be used for positioning work on the tray. Fan-shaped pins have 4 thin arms (0.4" diameter) that can be bent individually.



#1680045V Pegs, 5/Pkg

Uni-Pegs

The Uni-Pegs are easy to position and made of high-fused micro grain porcelain composition. The pegs' triangle shape provides excellent balance, support and stability. These can be assembled to form a round tray.



#1680080Uni-Pegs, 6/Pkg

Honey-Comb Pegs

Honey-Comb Pegs are made of nickel-chrome steel, which enable users to custom design honey-comb trays.



#1680085Honey Comb Pegs, 4/Pkg

Universal Pegs

These Universal Pegs are perfect stability and close placement. They are ideal for all firing needs within a laboratory. The pegs can be fired up to 2,200-degrees Fahrenheit. The Universal Pegs are interlocking with absolute precision for exact positioning.



#5920002Universal Pegs, 12/Pkg

Porcelain Stacking Trays

These Porcelain Stacking Trays save space and time. They come with covers.



#1680010Porcelain Stacking Trays, 5/Pkg

Porcelain Slant Tile

This Porcelain Slant Tile comes with five wells and five slants.



#1680020Porcelain Slant Tile

Nine Well Section Pallet

The Nine Well Section Palette has nine sections and comes with dry and vent. Stackable and sold individually.



#5001013Nine Well Section Pallet

Twenty Well Mixing Pallet

This Mixing Palette is stackable and offers ceramists the convenience of mixing and storing several shades of porcelain or the combination of needed for a single or multi-color shade. The small reservoir holes in the palette allows for easy fluid replenishing. The glazed surface protects the brushes. It has 20 small wells on the palette. Sold individually.



#5001004Twenty Well Mixing Pallet

Twenty-Three Well Staining Pallet

This Mixing Palette is stackable and offers ceramists the convenience of mixing and storing several shades of porcelain or the combination of needed for a single or multi-color shade. The small reservoir holes in the palette allows for easy fluid replenishing. The glazed surface protects the brushes. There are more than 20 wells on this palette. Sold individually.



#5001014Twenty-Three Well Staining Pallet

Porcelain Slant Tile


This Porcelain Slant tile is the same as Porcelain Slant Tile but features dry and vent. Has 10 sections and makes work easier.



#5001002Porcelain Slant Tile

Honey-Comb Square Tray Set


The Honey-Comb Square Tray set has a unique structure that heats rapidly and evenly, virtually eliminating any hot spots. The set contains one large and one small tray per box.

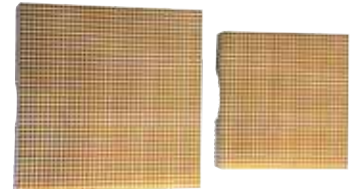

#1680030Honey-Comb Square Tray Set, 2/Set

Honey-Comb Round Furnace Tray


The Honey-Comb Round Furnace tray is 2-7/8" dia. x 1/2" (73 x 12.7 mm) thick. Heats evenly and produces high quality porcelain appliances.

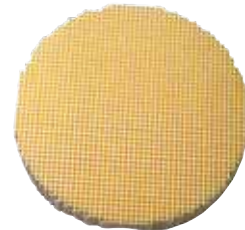

#1680050Honey-Comb Round Furnace Tray

Honey-Comb Octagon Tray


The Honey-Comb Octagon Tray is 7/16" (11.1 mm) thick. Heats evenly and produces high quality porcelain appliances.

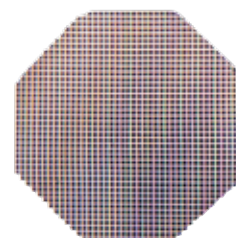

#1680065Honey-Comb Octagon Tray

Spider Tray

The Keystone Spider Tray is a new firing tray utilizing a fixed fan shaped pin specifically developed for all types of delicate ceramic work with an emphasis on all-ceramic restorations. With 8 bendable, thin metal prongs, the tray can adapt to a variety of cases. The unique shape and thinness of the blades allow a more uniform cooling of the ceramic restoration.

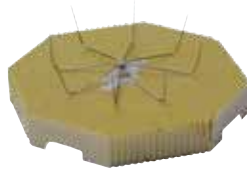

#1680040Spider Tray

Heat-Sink Sagger Trays

The Heat-Sink Sagger Trays provide rapid cooling while being used in repeated firings. The tray is designed for easy insertion and removal from furnace. It can be fired up to 2,800-degrees Fahrenheit.



#59200752 Trays, 2-1/4" x 2-1/4" x 3/8"
(57.1 x 57.1 x 9.5 mm)

Soft Pillows

These Soft Pillows are used for firing ceramic bridges when you need added support and making additions and corrections to veneers crowns and bridges. They will not cause adverse reactions to ceramic.



#5920110Round, 6/Pkg
#5920115Square, 6/Pkg

Carbon Insert Ceramic Tray

These Carbon Insert Ceramic Trays absorb silver vapor that is released during each firing. It's a must-use with silver palladium alloys. Items also come with 6 anterior pegs or 6 carbon replacements. Size: 2-1/4" x 2-1/4" x 1/4" (57.1 x 57.1 x 6.3 mm).



#5920040C010, 2/Pkg
#5920045C008, 2/Pkg plus 6 anterior pegs
#5920055C012, 1 plus 6 carbon replacements

Ceramic Tray


This Ceramic Tray is designed for easy insertion and removal from the furnace. Items also come with 6 anterior pegs, 6 posterior pegs or 6 etch pegs. Size: 2-1/4" x 2-1/4" x 1/4" (57.1 x 57.1 x 6.3 mm).



#5920020B002, 2/Pkg
#5920025B006, 2/Pkg plus 6 anterior pegs
#5920030B007, 2/Pkg plus 6 posterior pegs
#5920035ET2, 2/Pkg plus 6 etch pegs

Roundhouse Trays

Roundhouse Trays allow for easy heating and cooling. They are designed for easy insertion and removal from furnace.

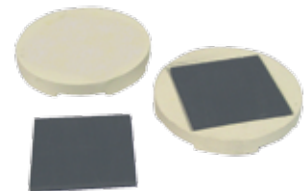

#5920060RT1 Ceramic Tray, 2/Pkg
#5920065RT2 Ceramic Insert Tray, 2/Pkg

H-Base Pegs

H-Base Pegs feature a revolutionary tunnel, which offers closer placement and greater stability. Interlocking allows absolute precision for exact positioning. The pegs can be fired up to 2,200-degrees Fahrenheit.



#5920011B008 Molar & Bicuspid Peg, 8/Pkg



#5920010B001 Posterior Peg, 8/Pkg



#5920015ET1 Etch Peg, 6/Pkg



#5920005B004 Anterior Peg, 8/Pkg

ROTARY

Hatho Habras Discs

Ideal for heavy cleaning (coarse), removing light oxide (medium), pre-polishing, semi-finishing, blending, texturizing and polishing (fine). This is an abrasive that does not generate the heat associated with rubber wheels. Embedded cubitron mineral abrasive is much harder than most metal. Removes oxides and polish without using compounds. Ideal for cleaning metal work and hard-to-reach areas. 10,000 rpm. Recommended: 4 discs mounted together.

Coarse Yellow - 80 grit. Remove deep scratches from precious metals, non-precious metals, acrylic dentures and composites.

- #16700834 Discs/Mandrel
- #16700974 Discs w/o Mandrel
- #167010148 Discs Bulk Package



Medium Red - 220 grit. Removes fine scratches from precious metals, non-precious metals, acrylic dentures and composites.

- #16700844 Discs/Mandrel
- #16700984 Discs w/o Mandrel
- #167010248 Discs Bulk Package



Fine Blue - 400 grit. Smooths precious metals, non-precious metals, acrylic dentures and composites.

- #16700854 Discs/Mandrel
- #16700994 Discs w/o Mandrel
- #167010348 Discs Bulk Package



Super Fine Pink - Pumice. Shine precious metals, non-precious metals, acrylic dentures and composites.

- #16700864 Discs/Mandrel
- #16701004 Discs w/o Mandrel
- #167010448 Discs Bulk Package



High Gloss Green - 6µ. High Gloss precious metals, non-precious metals, acrylic dentures and composites.

- #16701084 Discs/Mandrel
- #16701124 Discs w/o Mandrel
- #167011648 Discs Bulk Package



High Shine Orange - 1µ. High Shine precious metals, non-precious metals, acrylic dentures and composites.

- #16701074 Discs/Mandrel
- #16701114 Discs w/o Mandrel
- #167011548 Discs Bulk Package



Habras Disc Kit

A unique three-dimensional, abrasive and flexible design that provides a uniform finish. Ideal for heavy cleaning, removing light oxide, pre-polishing, semi-finishing, blending, texturizing and polishing.



- #16701136-Piece Kit, 10,000 rpm

Hatho Geza Brushes

Premium bristle materials (Goat Hair and Chung King) offer the perfect miniature brush for deep, interproximal and occlusal polishing. 3/32" shank, 15,000 rpm, 12/Pkg.

- Geza 100 HP**
#1670087
White Goat Hair



- Geza 120 HP**
#1670089
White Chung King



- Geza 121 HP**
#1670088
Black Chung King



- Geza 122 HP**
1670090
Gray Chung King



Hatho Habras Miniature Brushes

Long-lasting polyimide filaments. All brushes are embedded with polish, eliminating the need for polishing paste. The light gray brushes are embedded with aluminum oxide and are designed for pre-polishing ceramics and light-cured materials. The dark gray brushes are embedded with silicone carbide and are designed for use on all alloys. Long-lasting, durable, reduced dust while polishing, and a high sheen. 4/Pkg.

Aluminum Oxide - for ceramics and light cured materials.

Fine

- #16700533/4" Miniature
- #16700547/8" Miniature
- #16700551" Miniature
- #16700561-5/8" Lathe
- #16700572" Lathe



Medium

- #16700633/4" Miniature
- #16700647/8" Miniature
- #16700651" Miniature
- #16700661-5/8" Lathe
- #16700672" Lathe



Coarse

- #16700583/4" Miniature
- #16700597/8" Miniature
- #16700601" Miniature
- #16700611-5/8" Lathe
- #16700622" Lathe



Silicone Carbide - for alloy pre-polishing

Extra Fine

- #16700683/4" Miniature
- #16700697/8" Miniature
- #16700701" Miniature
- #16700711-5/8" Lathe
- #16700722" Lathe



Medium

- #16700733/4" Miniature
- #16700747/8" Miniature
- #16700751" Miniature
- #16700761-5/8" Lathe
- #16700772" Lathe



Hatho Toro Miniature Brushes

Straight handpiece mounted rushes manufactured from the finest natural horse and goat hair. Ideal for cleaning and polishing denture acrylic, cleaning interproximal spaces after denture deflasking and for general laboratory polishing applications.

3/4" (19 mm) 16,000 rpm

- #1670024 Extra Soft, Goat Hair, 12/Pkg
- #1670025 Extra Soft, Goat Hair, 100/Pkg



- #1670026 Soft, Goat Hair, 12/Pkg
- #1670027 Soft, Goat Hair, 100/Pkg



- #1670028 Standard Hard, Horse Hair, 12/Pkg
- #1670029 Standard Hard, Horse Hair, 100/Pkg



- #1670030 Extra Hard, Horse Hair, 12/Pkg
- #1670031 Extra Hard, Horse Hair, 100/Pkg



11/16" (17.5 mm) 16,000 rpm

- #1670032 Extra Soft, Goat Hair, 12/Pkg



- #1670034 Soft, Goat Hair, 12/Pkg
- #1670035 Soft, Goat Hair, 100/Pkg



- #1670036 Standard Hard, Horse Hair, 12/Pkg
- #1670037 Standard Hard, Horse Hair, 100/Pkg



- #1670038 Extra Hard, Horse Hair, 12/Pkg
- #1670039 Extra Hard, Horse Hair, 100/Pkg



9/16" (14.4 mm) 18,000 rpm

- #1670040 Extra Soft, Goat Hair, 12/Pkg
- #1670041 Extra Soft, Goat Hair, 100/Pkg



- #1670042 Soft, Goat Hair, 12/Pkg
- #1670043 Soft, Goat Hair, 100/Pkg



- #1670044 Standard Hard, Horse Hair, 12/Pkg
- #1670045 Standard Hard, Horse Hair, 100/Pkg



- #1670046 Extra Hard, Horse Hair, 12/Pkg
- #1670047 Extra Hard, Horse Hair, 100/Pkg



Hatho Polishing Discs

Ideal for working on softer materials, such as dentures and removal of soft denture relining material. The core is wood encased in plastic, allowing easy mounting/dismounting from the lathe and ensures the disk remains "true".

Medium

- #1670008 4" dia. (101.6 mm), 1" thick, 2,800 rpm
- #1670009 3" dia. (76.2 mm), 1" thick, 1,400 rpm



- #1670010 4" dia. (101.6 mm), 1/2" thick, 2,800 rpm
- #1670011 3" dia (76.2 mm), 1/2" thick, 1,400 rpm

Fine

- #1670012 4" dia. (101.6 mm), 1" thick, 2,800 rpm
- #1670013 3" dia. (76.2 mm), 1" thick, 1,400 rpm



- #1670014 4" dia. (101.6 mm), 1/2" thick, 2,800 rpm
- #1670015 3" dia (76.2 mm), 1/2" thick, 1,400 rpm

Hatho Poly-Buffs Chung King Lathe Brushes

The exterior consists of specially selected Chung King bristles which guarantee long life and high-quality polishing. These brushes are wrapped around a core of grit-free, soft Scotch Brite™ and fine muslin cloth. The core is made from wood encased in plastic, allowing for easy mounting/dismounting.

- #16700223" (76.2 mm) dia., Muslin Cloth
1,400 rpm, 3/Pkg



- #16700171-3/4" (44.4 mm) dia., Scotch Brite
2,800 rpm, 1/Pkg



- #16700182" (50.8 mm) dia., Scotch Brite
2,800 rpm, 1/Pkg



- #16700192-1/2" (63.5 mm) dia.,
Scotch Brite, 1,400 rpm, 1/Pkg



Hatho Special Lathe Brushes

Single row brush is designed for interproximal polishing on a lathe. The dual row brush is for the polishing of non-precious metals (e.g. chrome cobalt). 12/Pkg.



- #16700201-3/4" (44.4 mm) dia., 6,000 rpm
- #16700212" (50.8 mm) dia., 6,000 rpm

Hatho Miniature Handpiece Scotch Brite™ Brushes

Mounted on mandrel. Same application as Polishing Discs with Scotch Brite™. 12/Pkg.



- #1670016Coarse, 5,000 rpm
- #1670092Medium, 5,000 rpm
- #1670091Fine, 5,000 rpm

Hatho Miniature Polishing Brush

- #1670023Coarse, normal thickness,
8,000 rpm, 12/Pkg



Hatho Miniature Brush with polishing discs

These brushes have an inserted chamois with polishing discs. We provide two types of brushes with white goat hair and grey goat hair. Ideal speed for these brushes is 10,000 rpm.



- #1670051White Goat Hair w/ chamois, 12/Pkg
- #1670052Gray Goat Hair w/ chamois, 12/Pkg

LABORATORY

Abbott Robinson Brushes


Classic, high quality natural mounted bristle brushes ideal for cleaning and polishing of denture acrylic, cleaning interproximal spaces after denture deflasking, and for general laboratory polishing applications. These well-known, industry-standard brushes also excel at applying polishing materials and at polishing of lab-processed composites, metal and ceramic crowns. Durable design features permanently attached bristles for improved performance and extended life without flaring.



Gross		Dozen
#1170010Size 9 Soft.....	#1170011
#1170040Size 9 Standard Stiff.....	#1170041
#1170070Size 9 Stiff.....	#1170071
#1170020Size 11 Soft.....	#1170021
#1170050Size 11 Standard Stiff.....	#1170051
#1170080Size 11 Stiff.....	#1170081
#1170030Size 12 Soft.....	#1170031
#1170060Size 12 Standard Stiff.....	#1170061
#1170090Size 12 Stiff.....	#1170091

Calico Interleaved Lathe Brush

The Calico Interleaved Lathe Brush is a diverse 3-1/8" dia. polishing brush. It has three rows of converging bristles and two rows of Muslin buffs. The bristle is made of Chung King bristle with Muslin buff interwoven. This item is long lasting and saves time and labor. It polishes much faster than the standard brush.

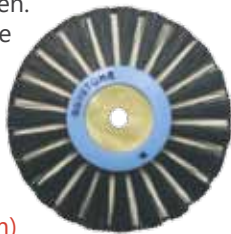

#11702153-1/8" dia. (79.3 mm)

Mounted Wire Brushes

These Mounted Wire Brushes have are either steel or brass mounted wire brushes. They come in two sizes: 3/4" and 1". Available 12/Pkg.



#1170230 Steel 3/4" (19 mm)
 #1170240 Steel 1" (25.4 mm)
 #1170245 Brass 3/4" (19 mm)

Lathe Brushes

For Lathes, Chung King Bristle. Sold in 12/Pkg.



Wood Hub		Plastic Hub
2 Rows - Converging Bristle		
#1170100B12 1-7/8" (47.6 mm) dia.....	#1170180
#1170110B14 2" (50.8 mm) dia.....	#1170190
#1170120B20 2-1/2" (63.5 mm) dia.....	#1170200

Wood Hub		Plastic Hub
3 Rows - Converging Bristle		
#1170130B23 2-3/4" (69.8 mm) dia.....	N/A

Wood Hub		Plastic Hub
Extra Stiff 3 Rows - Semi Converging		
#1170170B24 2-1/2" (63.5 mm) dia.....	#1170210

Wood Hub		Plastic Hub
3 Rows - Upright Bristle		
#1170140B25 2-3/4" (69.8 mm) dia.....	N/A

Wood Hub		Plastic Hub
4 Rows - Converging Bristle		
#1170150B27 3" (76.2 mm) dia.....	N/A

Wood Hub		Plastic Hub
4 Rows - Upright Bristle		
#1170160B29 3" (76.2 mm) dia.....	N/A

Flexis Diamond Discs

These extremely flexible, textile discs have strategically placed patches of diamonds. The ideal speed for Flexis is between 8,000 and 10,000 rpm. Not to be used for separating. Every pack of Flexis includes a mandrel. Available in 4 different grits.



#1300610Black Coarse.....	8/Pkg & 1 Mandrel
#1300620Red Medium.....	8/Pkg & 1 Mandrel
#1300630Green Fine.....	8/Pkg & 1 Mandrel
#1300640Yellow X-Fine.....	8/Pkg & 1 Mandrel
#1300600Assortment.....	4 grain sizes & 1 Mandrel

H.P. Sintered Diamonds


H.P. Sintered Diamond rotary burs come in three different grits: Coarse, Ultra Coarse and Super Coarse. Coarse is used for porcelain while super coarse is used on chrome, cobalt, and non-precious and hard metals. Sold individually.



	Coarse	Super Coarse	Ultra Coarse
#1	#1290665	#1290660	#1290880
#2	#1290735	#1290730	N/A
#3	#1290585	#1290580	#1290835
#4	#1290685	#1290680	N/A
#5	#1290725	#1290720	N/A
#6	#1290615	#1290610	#1290840
#7	#1290635	#1290630	#1290855
#8	#1290675	#1290670	#1290870
#9	#1290575	#1290570	#1290830
#10	#1290595	#1290590	#1290875
#11	#1290605	#1290600	#1290850
#12	#1290625	#1290620	#1290860
#13	#1290645	#1290640	#1290885
#14	#1290655	#1290650	N/A
#15	N/A	N/A	#1290845

H.P. Sintered Diamond Kit

The Sintered Diamond Kit contains 12 various diamond burs. All of these burs are our most popular burs. Kits come in Coarse, Super Coarse and Ultra Coarse. The kit comes with a convenient storage box.

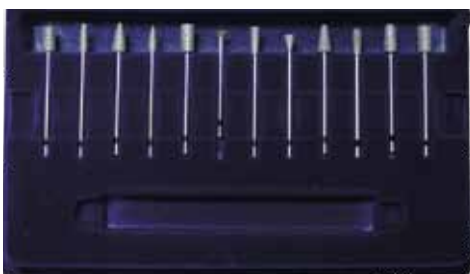


- #1290745 Coarse (Blue)
- #1290740 Super Coarse (Black)
- #1290890 Ultra Coarse (Red)

Keystone Spiral Cut Carbide Burs

Precision burs that require less pressure and cut smoothly with minimum heat. Hand ground to insure quality control.



#1201680 A 1/2	#1201730 A 3/8
#1201690 B 1/2	#1201740 B 3/8
#1201700 C 1/2	#1201750 C 3/8
#1201710 D 1/2	#1201760 D 3/8
#1201720 E 1/2		

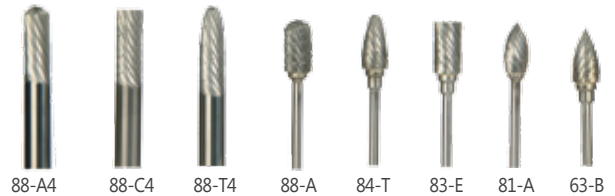

#1201780 88-A4	#1201830 84-T
#1201800 88-C4	#1201840 83-E
#1201810 88-T4	#1201860 81-A
#1201820 88-A	#1201880 63-B



#1201890 61-A	#1201910 52-D
#1201900 53-A	#1201930 51-A
#1201920 52-C	#1201870 73-C
#1201850 82-T	#1201950 41-P



#1202000 NK-1	#1202070 NK-8
#1202010 NK-1	#1201960 44-C
#1202020 NK-3	#1201980 31-P
#1202030 NK-4	#1201970 33-C
#1202040 NK-5	#1201940 32-P
#1202050 NK-6	#1201990 71-G
#1202060 NK-7		

Keystone Diamond Cut Carbide Burs

Coarse Maxi Cut

- For maximum reduction
- Use on acrylic and all metals



#1202181 A 1/2	#1202124 E 1/2
#1202121 B 1/2	#1202125 A 3/8
#1202122 C 1/2	#1202126 B 3/8
#1202123 D 1/2	#1202128 D 3/8



#1202133 88-A	#1202143 52-C
#1202134 84-T	#1202142 52-D
#1202136 82-T	#1202144 51-A
#1202138 73-C	

Keystone Diamond Cut Carbide Burs

Fine Mini Cut

- For minimum reduction
- Use on acrylics, composites, porcelain or metal for that smooth finish



#1202154 D 1/2	#1202163 88-T4
#1202155 E 1/2	#1202165 84-T
#1202159 D 3/8	#1202171 61-A
#1202160 E 3/8	#1202174 52-C

Special Burs

These Special Burs are sharp and cut fast. They are used for clog-free bulk reduction of acrylic, plaster and stone.



#1201955 SP1	#1201956 47XF
#1201915 52DS	#1201957 47XC

Keygold

These Spiral Cut Keystone Carbide Burs require less pressure and cut smooth with minimum heat. They are hand ground to insure quality control. We offer a wide list of bur styles and sizes. Sold individually.



#1202230 A 1/2	#1202255 A 3/8
#1202235 B 1/2	#1202260 B 3/8
#1202240 C 1/2	#1202265 C 3/8
#1202245 D 1/2	#1202270 D 3/8
	#1202275 E 3/8



#1202295 88-T4	#1202320 63-B
#1202300 88-A	#1202325 53-A
#1202305 84-T	#1202330 52-C
#1202310 83-E	#1202335 82-T
#1202315 81-A	



#1202345 52-D	#1202360 44-C
#1202350 73-C	#1202365 31-P
#1202355 41-P	#1202375 32-P

Z-Bur

The special hollowed, fluted design of Keystone's Z-Bur Diamond Bur is perfect for easy removal of sticky tissue conditioners and pastes commonly used to take denture impressions. The bur also works well in removing most soft liners. Normally messy removal of materials is eliminated with the Z-Bur action set at a medium speed (15,000-25,000 rpm). The slower cutting speed reduces burning and melting of the liner. The hollowed design also reduces the clogging of carbide and minimizes flying debris. Once the liner is removed the final preparation of the hard surface can be performed with carbide of choice.



#1201995 Z-Bur

Arbor Band Chuck

We offer two types of Arbor Band Chucks. The first is for standard lathes with 3/8" (9.5 mm) shaft. Also we have a shaft for standard 3/4" bands (1/4" shank). These bands come in left or right hand sides of the lathe. Sold individually.

For standard lathes with 3/8" (9.5 mm) shaft

#1260020 Arbor Band Chuck



For standard lathes with 3/4" (19 mm) bands, 1/4" (6.3 mm) shank

#1260030 Left Hand

#1260040 Right Hand



Bur Chuck

The Bur Chuck is designed to be used with small burs and mandrels as long as they do not exceed 1/4" (6.3 mm) diameter. It is to be used with lathes that have a tapered shaft. Sold individually. For standard shafts.



#1260050 Bur Chuck

Tapered Spindles

Made of steel, these Tapered Spindles are designed to be used with buffing wheels, brushes, and centered wheels. The chuck is built for placement on either right or left hand shafts with 3/8" or standard tapered shaft. Sold individually.

1-1/2" x 4" shank (38.1 x 101.6 mm)

#1790010 Tapered Spindle



For tapered 3/8" (9.5 mm) shaft

#1790060 Left Hand

#1790070 Right Hand



Stone Chucks

The Stone Chuck is designed to accommodate 1/4" to 3/8" wheels and stones. They can be used on standard 3/8" shafts and is available for either right or left hand shafts. Made of steel for long-lasting wear. Sold individually.

For 1/4" (6.3 mm) shank, fits jacobs

#1260060 Stone Chucks



For lathe and knock down wheels with 1/4" (6.3 mm) arbor hole and 3/8" (9.5 mm) shaft

#1260080 Right Hand



Carbide Twist Drills

These Carbide Twist Drills work on a pin machine. Available in 3/Pkg.



#1210000 Carbide Twist Drill

Mandrels

We offer various Mandrels to satisfy all laboratory needs.

Corkscrew 3/32" (2.38 mm) shank mandrel

#1520041 Dozen (12)

#1520040 Gross (144)



Tapered Handpiece mandrel

#1520056 Dozen (12)

#1520055 Gross (144)



3/32" (2.38 mm) shank, 1/16" (1.5 mm) screw, 8mm large head mandrel

#1520090 Dozen (12)

#1520185 Gross (144)



Heavy Duty, 1/8" (3.1 mm) shank, 1/16" (1.5 mm) screw mandrel

#1520061 Dozen (12)

#1520060 Gross (144)



Spiral 3/32" (2.38 mm) shank mandrel

#1520071 Dozen (12)

#1520070 Gross (144)



Spiral with Hex Nut, 3/32" (2.38 mm) shank mandrel

#1520081 Dozen (12)

#1520080 Gross (144)



Heavy Duty, 1/8" (3.1 mm) shank, 1/8" (3.1 mm) screw Mandrel

#1520091 Dozen (12)

#1520095 Gross (144)



H.P. High Speed Reinforced 3/32" (2.38 mm) shank, 1/16" (1.5 mm) screw mandrel

#1520051 Dozen (12)

#1520050 Gross (144)



WAXES

Adhesive-Coated Pressure Sensitive Wax

The Adhesive-Coated Pressure Sensitive Waxes are flexible 4" x 4" wax sheets which are coated on one side with a pressure sensitive adhesive. Available in 32/Box.

- #188001020 gauge
- #188002022 gauge
- #188003024 gauge
- #188004026 gauge
- #188005028 gauge
- #188006030 gauge



AFG Aesthetic Translucent Waxes

Keystone AFG aesthetic translucent waxes, created by Alberto Battistelli, are used to construct wax teeth that faithfully reproduce natural teeth with a layering system similar to that used by prosthodontists for dental ceramics. The pastel coloration with light shades is very similar to dentine and natural enamel and is accompanied by slightly brighter colors. This reproduces individual characterization in a way that is similar to what is normally done with super-colors or intensive masses during the construction of ceramic teeth.

AFG wax kits contain black wax to create perfect marginal closures, an opaque white wax, and an opaque grey wax for the technique of mono-color waxing, which assists in the viewing of anatomical details.

White Wax is for mono-color waxing, opal effects and imitation of decalcified areas in aesthetic waxing. Dentin A1 is for modeling of light dental nucleus. Dentin A3 is for modeling of dental nucleus or the neck. The blue/grey wax is the imitation of the areas of grey-blue transparency. Neutral is for the imitation of the effects of translucence, enamel and dilution of all the masses to make them more translucent. Orange is for modeling of the neck, the back of occlusal zones, the mamelons and the interproximal areas. The light brown is for the coloration of the neck and the proximal areas and the dark brown in for the simulation of caries. **International Sales Only.**



AFG Sculpturing Waxes

- #1911566No. 1 Black, 9g
- #1911567No. 2 White, 9g
- #1911568No. 3 Grey Opaque, 9g
- #1911569No. 4 Dentin A1, 9g
- #1911570No. 5 Dentin A3, 9g
- #1911571No. 6 Neutral, 9g
- #1911572No. 7 Blue/Grey, 9g
- #1911573No. 8 Orange, 9g
- #1911574No. 9 Light Brown, 9g
- #1911575No. 10 Dark Brown, 9g

Base Plate Wax

Our Base Plate Wax is made from the finest ingredients. The wax is great for general purpose uses while remaining an economical wax. Available in red or pink color and comes in 5-pound cases.



- #1880175Red
- #1880170Pink

CE

Mizzy Allcezon Base Plate Wax

Mizzy Allcezon Base Plate Wax is composed of general, all-purpose wax for bites, registration and impressions. Pink color, available in 1 lb. or 5 lb. boxes.



- #6160100Mizzy Allcezon Base Plate Wax, 1 lb.
- #6160300Mizzy Allcezon Base Plate Wax, 5 lb. CE

Yeti Solidus Base Plate Wax

Yeti Solidus is a pink baseplate wax that's also suitable as a bite wax. Solidification point is very high and the wax is very hard. Works well for modeling features. **North American Sales Only.**



- #186018280 Baseplate, 500g
- #186015584 Modelling Wax, 500g

NewCera® Block Out Wax

Block out wax comes in honey color. Melting point: 70°C/158°F. **International Sales Only.**



- #1910543Honey, 70g

Yeti Block Out Wax

Yeti Block Out Wax has good visibility even when a thin coat is applied. It's good for carving (with or without thermo-stick) and does not react with monomer liquids and commercial silicone. Does not remove from metal. **North American Sales Only.**



- #1860222Orange, 45g

Yeti Cutex Block Out Wax

Yeti CUTEX Block-Out Wax is best used after application of the die spacers. The higher melting point of CUTEX avoids bonding to the other YETI dipping waxes, underlining wax or cervical waxes. Not suitable for the casting technique.

North American Sales Only.

#1860057Beige, 20g



Bite Blocks

These Bite Blocks are pink recessed, medium soft with 3/8" wax for occlusal rims. U-shaped and pliable for custom reshaping at room temperatures to fit the individual arch. Available in 100/Box.



#1880240Bite Blocks

#1880250Bite Sticks



Mizzy Byte Ryte Wax

Mizzy Byte Ryte Wax is the sheet form of bite wax with same properties as cake form.



#6160700Yellow, 1 lb.

#6160800Yellow, 5 lb.

Boxing Strip Wax

The Boxing Strip Wax is soft, tacky red wax perfect for boxing impressions.



#1880260Red Regular, 5 lb.

#1880262Red Regular, 1 lb.

#1880264Red Xtra Thin, 1 lb.

NewCera® CAD/CAM Discs

NewCera CAD/CAM Discs has a melting point of 115°C/239°F. **International Sales Only.**

Technical specification: ~ 27,500Up/m
Feed rate: 20-25mm/sec
2 Cutters 2mm
Comes with ridge and without ridge.



#1910555Grey, 20mm

#1910556Beige, 18mm

#1911557Grey with Ridge, 20mm

#1911558Beige with Ridge, 18mm

NewCera® CAD/CAM Sculpturing Wax

NewCera CAD/CAM Sculpturing Wax has a melting point of 62°C/144°F.

International Sales Only.



#1910549Green, 70g

#1910546Beige, 70g

#1910552Grey, 70g

Yeti CAD/CAM Scanning Wax

Yeti CAD/CAM Scanning Wax is an economical scanning wax that is a laser and light opaque modeling wax. Specially used for CAD/CAM technology and guaranteed best scan and fitting results. **North American Sales Only.**



#1860360Beige, 45g

Casting Wax

This high-quality Casting Wax makes jobs easier and economical.



Pink

#188027020 gauge.....#1880280

#188029022 gauge.....#1880300

#188031024 gauge.....#1880320

#188033026 gauge.....#1880340

#188036028 gauge.....#1880350

#188037030 gauge.....#1880380

Green

NewCera® Cervical Wax

The NewCera Cervical Wax has high stability, low shrinkage, excellent carving properties, and a low melting point. Melting point: 60°C/140°F. **International Sales Only.**



#1910537Red, 70g

NewCera® ROCK Cervical Wax

This cervical wax has a melting point of 65°C/149°F. **International Sales Only.**



#1911560Black, 70g

LABORATORY

Slaycris Crown & Bridge Wax

Slaycris Wax is medium soft wax prepared for use in all types of crown and bridge work. Available in 3 oz. tin.



#1880795Regular Green

Yeti Creation Wax Set

This versatile Creation Wax Set comes with 12 different colors and allows models for porcelain layer technique. The set contains grey transparent, white transparent, enamel blue transparent, yellow, orange, dentin, light dentin, cervical color, brown, light red brown and dark brown. The wax comes in 8g tins. Refill waxes (6/Pkg) are also sold separately. **North American Sales Only.**



#1860033Creation Set

Creation Individual Refills

- #1860193Enamel, 6/Pkg, 8g each
- #1860186White Transparent, 6/Pkg, 8g each
- #1860187Dentin, 6/Pkg, 8g each
- #1860096Refill Pack I - 8g each, 6/Pkg
Enamel, grey, white, blue, yellow, orange
- #1860065Refill Pack II - 8g each, 6/Pkg
Dentin, light dentin, cervical color, brown, light red-brown and dark brown
- #1860066Refill Pack III - 8g each, 6/Pkg
Transparent, silver, gold, black, 2 x dentin

Dipping Wax

Keystone Dipping Wax is available in Red, Green or Yellow. Available in a 10 oz. jar.



- #1890030Red
- #1890050Green
- #1890040Yellow

NewCera® Dipping Wax

NewCera Dipping Wax comes in hard and soft wax pellets. **International Sales Only.**



- #1910544Red, Hard 300g
- #1910545Yellow, Soft 300g

Yeti Duo Dip Granular Dipping Wax

The Yeti Duo Dip is another alternative wax within Yeti's wide range of dipping waxes. It's the ideal mixture of Elasto Dip and Preci Dip and is very precise with the perfect amount of elastic. The granulated form makes for easier dispensing. It's easy, clean and fast to use. **North American Sales Only.**



#1860104Orange, 80g

Yeti Elasto Dip Dipping Wax

Yeti Elasto Dipping Wax produces almost unbreakable wax copings. It's a natural resin added for more elasticity and stability. The dipping wax is easy to dose, mixable with the harder Preci Dip dipping wax. **North American Sales Only.**



- #1860073Blue, 150g
- #1860041Red, 150g
- #1860035Pink, 150g
- #1860069Brown, 150g

Yeti Preci Dip Dipping Wax

Yeti Preci-Dip Dipping wax offers very precise results and smooth surfaces. The dipping wax creates even wax copings. **North American Sales Only.**



- #1860044 Yellow, 150g
- #1860050 Red, 150g

Corning Inlay Wax

Corning Inlay Wax is developed for the requirements of inlay technique. The wax is form-stable and contraction resistant (low shrinkage) from wax-up to burnout. It will not pull, flare, chip, crack, drag or expand at the margins, even when being cast in extremely thin sections. It will resist deformation due to handling and can be easily carved. Will not become sticky or flaky or gum up instruments and will burn-out cleanly with no residue.




- #1880390Regular Green, 1 lb.
- #1880400Regular Blue, 1 lb.
- #1880410Hard Blue, 1 lb.
- #1880420Hard Green, 1 lb.
- #1880430Special Hard Green, 1 lb.
- #1880440Special Hard Blue, 1 lb.
- #1880450White, 1 lb.

Keystone Inlay Wax

Keystone Inlay Wax provides the highest quality. Perfect for all inlay wax needs. Available in 1-pound box.

- #1880385Regular Blue
- #1880387Regular Green
- #1880450White
- #1880386Hard Blue
- #1880388Hard Green

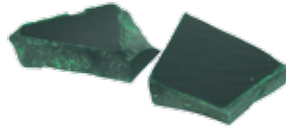

Maves Inlay Wax

Maves Inlay Wax comes in two sizes and styles. We offer #2 and #3 sizes that are in sticks or chunks form.

- #1880460Green, #2 Sticks, 1 lb.
- #1880480Green, #2 Chunks, 1 lb.
- #1880470Green, #3 Sticks, 1 lb.
- #1880490Green, #3 Chunks, 1 lb.



Mizzy Impression Compound Cakes & Sticks

Mizzy Compound Cakes & Sticks are pleasantly scented and flavored compound for taking full and partial impressions, and making preliminary impression trays.

Brown Medium Heat Compound:

- Plastic at 135°F (57.2°C)
- Ideal for snap impressions
- Working Temperature: 125°F - 135°F (52°C - 57°C)

Red Low Heat Compound:

- Maximum visibility of detail
- Working Temperature: 115°F - 125°F (46°C - 52°C)

Green Sticks:

- Has the most diverse working range
- Renders sharp details
- Will not fracture while trimming
- Chills evenly without warping
- Maximum contrast



- #6060700Brown, 15 Sticks, 4.5 oz. (128 g)
- #6060300Red, 15 Sticks, 4.5 oz. (128 g)
- #6060500Green, 15 Sticks, 4.5 oz. (128 g)
- #6060600Brown, 8 Cakes, 8 oz. (227 g)
- #6060100Red, 8 Cakes, 8 oz. (227 g)



Yeti IQ Compact Ash-Free White Wax

Yeti IQ Compact Ash-Free White Wax ideal for waxing up pressable ceramic restorations. It burns out without residue. **North American Sales Only.**



- #1860349Ash-Free White Wax, 45g

Yeti IQ Compact Opaque White Wax

Yeti IQ Compact Opaque has excellent masking properties. Use for temporary covering of framework and wax-ups. **North American Sales Only.**



- #1860350Opaque White Wax, 45g

Yeti IQ Pearl Wax

Yeti IQ Pearl Wax is in the form of wax pearls. This allows for quick taking up of the wax at lower temperatures. The control taking up gives small quantities for occlusal surfaces and larger quantities for basic form of the crown. **North American Sales Only.**



- #1860218Blue, 80g
- #1860244Pearl, 80g
- #1860245Beige, 80g
- #1860246Neon Green, 80g
- #1860247Neon Yellow 80g

NewCera® Milling Wax

The NewCera Milling Wax has a melting point of 60°C/140°F. **International Sales Only.**



- #1910540Green, 70g
- #1910541Gray, 70g

Yeti Nawax Compact

Yeti Nawax Compact is a professional and special abrasive wax for the natural and functional construction of teeth. The wax offers the highest stability even with sculpturing of the finest structures in the construction of morphology. Gives excellent abrasion characteristics for the constructions of functional morphology. Due to an increased hardness grade there's no additional work needed after the modelation modeling and additional polishing needed after casting.

North American Sales Only.



- #1860382Sand, 45g
- #1860138Light Gray, 45g
- #1860217Apricot, 45g

Yeti Sample Set

The Yeti Sample Set has liquids for die preparation and Thowax. The set includes Grey and Beige Sculpturing Wax, Bordeaux Underlining Wax, Red Cervical Wax, Clear Spacer, Yeti Lube, and Blue Marker Indicator.

North American Sales Only.



- #1860126Yeti Sample Set

NewCera® Sculpturing Waxes

Keystone NewCera Sculpturing Wax is a new generation of a perfect sculpting wax that works without limits. It's a wax that does what you want and helps you create wax-ups easily and efficiently. The melting point of the wax is 131°F (55°C). Available in six different colors (Gray, Beige, Blue, Green, Yellow, White) and comes in a 70g package. Sold individually. **International Sales Only.**



Sculpturing Waxes

- #1910501Gray, 70g
- #1910502Beige, 70g
- #1910503Blue, 70g
- #1910504Green, 70g
- #1910505Yellow, 70g
- #1910506White, 70g

Ash-Free Sculpturing Waxes

- #1911501Gray, 70g
- #1911502Beige, 70g
- #1911503Blue, 70g
- #1911504Green, 70g
- #1911505Yellow, 70g
- #1911506White, 70g

NewCera® ROCK Sculpturing Wax

NewCera ROCK Waxes are ideal for waxing up of occlusal surfaces as it is rigid and has excellent flow properties. All the NewCera Rock Waxes are available in ash-free for the ceramic technique as they are free of opaque ingredients. Melting point is 63°C/145°F. **International Sales Only.**



Sculpturing Waxes

- #1910507Gray, 70g
- #1910508Beige, 70g
- #1910509Blue, 70g
- #1910510Green, 70g
- #1910511Yellow, 70g
- #1910512White, 70g

Ash-Free Sculpturing Waxes

- #1911507Gray, 70g
- #1911508Beige, 70g
- #1911509Blue, 70g
- #1911510Green, 70g
- #1911511Yellow, 70g
- #1911512White, 70g

Yeti Monolith Sculpturing Thowax

Yeti sculpturing wax is also available in another smart form. The Monolith Thowax Chip was created for those who prefer a different form to the popular 70g tins. Beside the approved Thowax quality, the Monolith system has several other advantages. A magnet is used for adhesion of the wax chips on the solid safe-standing metal base. There are rubber pads at the bottom of the base avoid slipping. Allows fast and easy changing of the chips. The small 20g chips allows for faster consumption avoids contamination. The Monolith chip has a comfortable and ergonomic taking-up of the wax. It can be completely consumed, no inaccessible wax at in the edges of tins dust and dirt protected by a transparent cap. **North American Sales Only.**



- #1860143Monolith Base w/o Wax Chips
- #1860068Grey Sculpturing Wax Chips 20g, 3/Pkg
- #1860092Bordeaux Underlining Wax Chips 20g, 3/Pkg
- #1860089Beige Sculpturing Wax Chips 20g, 3/Pkg
- #1860099Red Cervical Wax Chips 20g, 3/Pkg
- #1860099Green Milling Wax Chips 20g, 3/Pkg

Yeti IQ Sculpturing Wax

Yeti IQ Sculpturing Wax was created in with the help of well-known dental experts. This produced a high-quality sculpturing wax suitable for all applications. You get a high inner stability that's very hard and not fragile. The higher point of solidification gives very good sculpturing features. **North American Sales Only.**



- #1860133 Neon Yellow, 45g Cylinder
- #1860121 Neon Green, 45g Cylinder
- #1860131 Beige, 45g Cylinder
- #1860128 Gray, 45g Cylinder
- #1860122 Blue, 45g Cylinder
- #1860129 Neon Green, 45g Tin
- #1860132 Beige, 45g Tin
- #1860134 Gray, 45g Tin
- #1860124 Blue, 45g Tin

Yeti Thowax Sculpturing Wax

Yeti Thowax is an all-natural sculpturing wax that's suitable for the crown and bridge and inlay technique. Thowax special waxes are specially blended for each individual field of application. **North American Sales Only.**



- #1860016 Gray, 70g
- #1860017 Blue, 70g
- #1860018 Beige, 70g
- #1860047 Green, 70g
- #1860064 Beige Opaque, 70g
- #1860040 Red, 70g
- #1860051 Brown, 70g
- #1860063 Violet, 70g

Set-Up Wax


The Set-Up Wax has a great tenacity and ideal for partial prosthesis. There's no distortion of the wax, due to small range (<5%) of melting points and there's also no runniness or droplet formation. One sheet is enough for two prostheses. Available in 5 lbs. box.



- #1880155 Pink

Special Formula Wax

The Special Formula Wax is formulated to check for premature occlusion and as an articulating medium. Available sizes 5-7/8" x 2-7/8" and comes in 1-pound box.

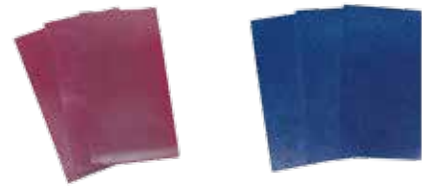

- #6162700 Dr. Schuyler's Articulating Wax
- #6162600 Dr. Cohn's Special Formula Wax



Yeti Thowax Special Waxes

We offer several different types of special Yeti Thowax Waxes. Each is available in 60g tins.

Here are available waxes:

1. Cervical Wax - smooth flowing wax, red color.
2. Underlining Wax - high density enables the carving of sharp edges, Bordeaux color.
3. Milling Wax - can be milled at 5,000 rpm, gray opaque color.
4. Stick-On-Wachs - to place casting channels, sprues and pontics, neon pink color.
5. Sticky Wax - helpful for any repair of broken dentures or to solder metal parts, beige.


North American Sales Only.



- #1860034 Cervical (Red), 60g
- #1860023 Underlining (Bordeaux), 60g
- #1860102 Milling (Gray Opaque), 60g
- #1860080 Stick On (Neon Pink), 60g
- #1860160 Sticky Wax (Beige), 60g

Sprue Wax

Sprue Wax is specifically formulated and designed to be used as sprues. Comes in red color and 6" long rods.

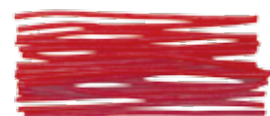

- #1880720 Red 6 gauge, 1 lb.
- #1880730 Red 8 gauge, 1 lb.
- #1880740 Red 10 gauge, 1 lb.
- #1880710 1/4" (6.3 mm) dia., 1 lb.
- #1880680 Wax Cartridge 6" x 1/2" (152.4 x 12.7 mm)

LABORATORY

NewCera® Sticky Wax

NewCera Sticky Wax has a melting point of 75°C/167°F.
International Sales Only.



#1910542Orange Neon, 70g

Sticky Wax

The Sticky Wax is used to hold or weld materials or pieces together. It can also be used to connect or repairs wax patterns. The wax is melted with a gas flame and used as a modeling wax. Available in 1 lb. boxes.



- #1880750Red Lumps
- #1880770Yellow Lumps
- #1880775Corning Yellow Sticks, 120
- #1880760Corning Yellow Lumps

Stippled Wax Sheets

These Stippled Wax Sheets are pressure sensitive. The tried and tested wax is perfect for waxing up the bases of upper partial dentures. The stippled pattern gives the base a natural texture and is easy to adapt and adhere. Available in 15/Pkg.



- #1880790Green, 22 gauge
- #1880785Green, 24 gauge
- #1880791Green, 26 gauge

Yeti Consequent Universal Wax

Yeti Consequent Universal Wax is especially used within the casting technique but also suitable for the waxing up technique. It has good connecting wax, melting point 60F and suitable for crowns and bridges. It cools down slower than Thowax or IQ wax. The softer wax is ideal to transfer the occlusal wax from the master model to the duplicate. Has the same excellent scraping characteristics as Thowax and IQ wax.

North American Sales Only.



- #1860226Green, 60g
- #1860227Red, 60g
- #1860221Blue, 60g

Utility Wax

Utility Wax is especially suited for built up and post-damming purposes. Available in Red Round, White Round, Red Square and White Square. Comes in 1 lb. boxes, 3/16" size.



- #1880180Red Round Strips
- #1880200Red Square Rope
- #1880190White Round Strips
- #1880210White Square Rope

Kewax Wire Wax

Kewax Wire Wax the best wax money can buy. The wax is extruded to a uniform diameter and provides the proper tackiness and adhesion.



Round

- #18805006 gauge
- #18805108 gauge
- #188052010 gauge
- #188053012 gauge
- #188054014 gauge
- #188062016 gauge
- #188063018 gauge
- #188064020 gauge

Half Round

- #18805506 gauge
- #18805608 gauge
- #188057010 gauge
- #188058012 gauge
- #188059014 gauge

Half Pear

- #18806006 gauge

Yeti Deton Wax Wire

Yeti Deton Wax Wire achieves a tension-free wax wire that's not easily produce. A special method and special micro-molecular wax is used for the production. The process yields a medium hardness wax wire with great internal stability. **North American Sales Only.**



- #1860181Roll, 2.5 mm
- #1860058Roll, 3.5 mm
- #1860059Roll, 4.0 mm
- #1860097Roll, 5.0 mm

Yeti VKS Wax

VKS Wax is for full ceramic systems. The wax is filtered several times and free of any opaque particles. **North American Sales Only.**



- #1860060Bordeaux, 60g
- #1860061Beige Transparent, 60g
- #1860062Gray Transparent, 60g

Plastic Hollow Sprues

Keystone is the best source for Plastic Hollow Sprues with or without reservoirs, wax sprues with reservoirs or hollow plastic or wax runner bars. Plastic will not distort during burnout, hollow plastic will not expand and damage the investment mold as solid plastic may.



	250/Box	500/Box	1,000/Box
8 gauge, 1-1/4"	#1290010	#1290060	N/A
10 gauge, 1-1/4"	N/A	#1290070	N/A
12 gauge, 1-1/4"	N/A	#1290080	N/A
14 gauge, 1-1/4"	N/A	#1290090	N/A
16 gauge, 1-1/4"	#1290050	#1290100	N/A
8 gauge, 3/4"	N/A	#1290150	#1290230
10 gauge, 3/4"	N/A	#1290160	#1290240
12 gauge, 3/4"	N/A	#1290170	#1290250
14 gauge, 3/4"	N/A	#1290180	#1290260
8 gauge, 1/2"	N/A	N/A	#1290190
10 gauge, 1/2"	N/A	#1290120	#1290200
12 gauge, 1/2"	N/A	#1290130	#1290210
14 gauge, 1/2"	N/A	N/A	#1290220

Yeti IQ Quadro Sticks

After burnout the Yeti IQ Quadro Sticks leave a square, hollow channel. Molten metal has round cross-section due to its high surface tension and simulates a rolling ball in motion. Displaced air escapes back through the unfilled corners when the molten metal enters the square channels, eliminating blow-back. The resulting castings are visibly smoother and imperfection free while preformed patterns improve quality of your castings.


North American Sales Only.



- #1860137Green 2.5mm, 50g
- #1860138Green 3.0mm, 50g
- #1860217Green 3.5mm, 50g

Yeti IQ Sticks

Yeti IQ Sticks – Round- offers more security when lifting your wax pattern. This item provides stable connections for the whole casting channel system. **North American Sales Only.**

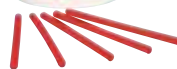

- #1860176Red Hard 2.5mm, 250g
- #1860198Red Hard 3.0mm, 250g
- #1860248Red Hard 3.5mm, 250g
- #1860179Red Hard 4.0mm, 250g
- #1860180Red Hard 5.0mm, 250g

Yeti Sprues

Fluid metal experiences a contraction upon solidification. The resulting partial vacuum can be compensated by drawing molten metal from the "reserve ball". **North American Sales Only.**



- #18600904mm Sprues blue, ball-diameter, 250 pc.
- #18601975mm Sprues blue, ball-diameter, 250 pc.
- #18601116mm Sprues blue, ball-diameter, 250 pc.
- #18601207mm Sprues blue, ball-diameter, 250 pc.

Yeti IQ Triangle

Yeti IQ Triangle has a red casting bar and is extra hard and burns without residue. There's equal cooling of casting object that saves time. The off-centered position of the casting bar due to the angle of inclination of the feeding channels. **North American Sales Only.**



- #1860159Red Hard 2.5mm, 250g
- #1860224Red Hard 3.0mm, 250g

Yeti Wax Pontics

Pontics consist of pure wax and were developed in cooperation with dental master technician Thomas Schmidt. The size offered will fit 90% of all cases and will reduce the number of different packets in stock. If the Pontic does not fit, it can be easily brought to the desired size by waxing it up or cutting it down. **North American Sales Only.**



- #1860037 Blue Pontic Assortment Intro Kit, 6 x 25 pc
- #1860098 Blue Upper Right, 25 pieces
- #1860116 Blue Lower Right, 25 pieces
- #1860114 Blue Upper Left, 25 pieces
- #1860113 Blue Lower Left, 25 pieces
- #1860105 Blue Upper Anterior, 25 pieces
- #1860110 Blue Lower Anterior, 25 pieces

Wax Palate Patterns

This works great for gold, acrylic, partial or full palates. It improves denture quality by obtaining natural living tissue finish, beautiful esthetics and greater comfort.

Reduces labor costs. Less waxing and finishing (saves hand waxing) and is convenient to use.



- #1600010Wax Palate Patterns, 100/Pkg

LABORATORY

Yeti Occlusal Wax Patterns

Yeti Occlusal Wax Patterns have a stable center and more tolerance for jaw movement. The patterns adapt easily in every situation. Comes in blocks of 4 occlusal wax patterns. They are made of 100% wax and burns without residue.

North American Sales Only.



#1860049Blue Assortment, 100/Box

Yeti Occlusal Wax Powder

Yeti Occlusion Powder consists of 100% wax. The wax indicates friction or undesired contact spots when using articulator and serves as a separating powder when applying hot wax. It burns without residue and does not require extra cleaning. **North American Sales Only.**



#1860043White Occlusion Powder, 17g

Yeti Green Occlusal Spray

The Yeti Green Occlusal Spray is for or the recognition of high spots with targeted spray. There are 2 different nozzles enable fine dosing and the solution is 100% water soluble. Environment friendly: pressurized gas not harmful to ozone. **North American Sales Only.**



#1860257Green Occlusal Spray, 75ml

Yeti Lube Insulating Liquid

Yeti Lube Insulating Liquid isolates wax against plaster, metal and wax. It contains no alcohol or solvents, and it is water soluble and will not react with the embedding material. **North American Sales Only.**



#1860008Yeti Lube, 20ml

Yeti Finale Wax Smoothing Liquid

Yeti Finale Wax Smoothing Liquid creates smoother surfaces of wax modelation. Superior adaptation to wax surface for better casting results. It smooths surfaces of wax modelation, scratches and uneven surfaces disappear. There are short drying times – depending on the layer thickness – for a shorter finishing time. Works well for occlusal areas, wax modelation within denture techniques, reinforcement of contact points, casting techniques, sealing of fissures between plaster and wax profiles.

North American Sales Only.



#1860123Finale Wax Smoother, 20ml

Klean All Boil Out Agent

Klean All Boil Out Agent keeps the wax in suspension. The agent does not leave any residue on models. This goes into solution easily and dissolves any grit from the wax. It's odor-free and non-chlorinated.

#101410232 oz.

#10141031 gallon



Ke-Solv

Ke-Solv is a non-toxic solvent for wax removal. Works quickly and is a must-have for any lab.

#14800101 gallon



NewCera® Iso Pen

High quality isolation pen. **International Sales Only.**



#1911562Iso Pen

Yeti Iso-Quick Isolating Pen

Yeti ISO-Quick Isolating Pen is a handy insulating pen filled with Yeti Lube. This allows the exact application without wasting material and easy to refill. Comes with 2 different tips (1 pointed and fine and 1 broad). This item is economical and user-friendly, which allows it to be used in the whole laboratory.

North American Sales Only.



#1860076Iso-quick Isolating Pen

#1860072Refill Bottle, 250 ml

NewCera® Wax Brush

High quality wax brush. **International Sales Only.**



#1911565Wax Brush

Yeti Wax Brush

The Yeti Wax Brush is used to smooth and even out wax surfaces gently after carving. **North American Sales Only.**



#1860038Wax Brush



ARTICULATING 74
CEMENTS & LINERS..... 74
COSMETIC DENTISTRY..... 75
DENTAL DAMS..... 75
ETCHANT GELS 75

IMPRESSION MATERIAL 76
MATRIX MATERIALS 77
MEDICAMENT MATERIALS.... 78
NEEDLESS INJECTOR..... 78
SURGICAL PRODUCTS..... 78

ARTICULATING

Articulating Papers

Our Articulating Paper is soft, smooth and non-sticking. The super sensitive paper is great for marking at every point of articulations. The horseshoe-shaped articulated papers are especially useful for patients who tend to bite unilaterally during the occlusion test due to diminished resilience. Has a high resistance to tearing.



Regular

- #04-00122.....Red/Blue Regular, 101.60 microns
- #04-00522.....Blue Thick, 127 microns
- #04-00622.....Red/Blue Regular, 101.60 microns



Regular

- #04-00222.....Blue Thin, 101.60 microns
- #04-00322.....Blue X-Thin, 76.20 microns
- #04-00422.....Blue XX-Thin, 50.80 microns



Mizzy PIP Paste

PIP is a pure white silicone paste, used for seating crowns, bridges and full, immediate and partial dentures. The brief procedure yields specific results that are easy to read and thoroughly reliable, surpassing results obtained with chemical set pressure indicating materials used to locate areas of denture discomfort. PIP is applied to the dry denture; companion PIP spray is applied to the tissue. The denture is then seated under pressure then removed. Areas where the paste has been displaced show points of excessive pressure.



- #6120900PIP Tube, 1 oz. (.02 L)
- #6120200PIP Jar with Remover, 1-1/4 oz. (.03 L)
- #6120300PIP Jar with Remover, 2-1/4 oz. (.06 L)
- #6120400PIP Jar, 8 oz. (.23 L)
- #6121000PIP Pump, 4 oz. (.11 L)
- #6120550PIP Packet with Remover & Brush, 24/Pkg
- #6120600PIP Brushes, 12/Pkg
- #6000437PIP Remover, 1 oz. (.02 L)
- #6120700PIP Remover, 4 oz. (.11 L)



Mizzy PIP Spray

PIP Spray is the perfect companion product to PIP paste. This mint flavored wetting agent can be sprayed onto teeth and soft tissue to produce smooth, accurate bubble-free surfaces on all types of impression material. Use as a rinse to eliminate ropy saliva.



- #6140100PIP Spray, 4 oz. (.11 L)
- #6140200PIP Spray Concentrate, 16 oz. (.947 L)



CEMENTS & LINERS

Fleck's Zinc Phosphate Cement

Since the advent of zinc phosphate cement, many other types of cement have been introduced. None, however, has demonstrated clinical superior in terms of strength, resistance to dissolution, and overall retention like Fleck's Zinc Phosphate Cement has. And of all zinc phosphate, Fleck's is the leading brand and cement of choice of dentists.



29g

- | | |
|-----------------------------|-----------------------------|
| #6050100Snow White | #6050500Light Gray |
| #6020200Light Yellow | #6050900Golden Brown |
| #6050300Yellow | #6051300Ceramic Blend |
| #6050400Golden Yellow | |

8 oz.

- | | |
|----------------------------|-----------------------------|
| #6055600Snow White | #6055900Golden Yellow |
| #6055700Light Yellow | #6056300Golden Brown |
| #6055800Yellow | #6056700Ceramic Blend |

Ortho Pak: 1/8 oz. Powder & 2x35cc Liquid

- | | |
|----------------------------|-----------------------------|
| #6053600Snow White | #6053900Golden Yellow |
| #6053700Light Yellow | #6054400Golden Brown |
| #6053800Yellow | |

Liquids

- | | |
|---------------------|---------------------|
| #605140015 ml | #605170035 ml |
|---------------------|---------------------|



Mixing Poly Pads

Mixing Poly Pads is extra heavy with a high poly-coated finishes. It can be used for mixing cements, composites, rubber base materials and silicates. Non-skid foam base.



- #15-03022.....1-3/8" x 1-3/8" (34.9 x 34.9 mm)
- #15-03122.....3" x 3" (76.2 x 76.2 mm)
- #15-03022.....3" x 6" (76.2 x 152.4 mm)
- #15-03022.....6" x 6" (152.4 x 152.4 mm)

Orange Solvent with Lanolin

Orange Solvent with Lanolin is perfect for the removal of PIP paste, zinc oxide-eugenol, rubber base and other impression material from dentures, hands, face, mixing slabs and instruments.



- #6120800Orange Solvent, 32 oz. (.95 L)

COSMETIC DENTISTRY

Niu Nait Take-Home Care Whitening Kit

Niu Nait Take-Home Whitening System comes in either 16% or 22% carbamide peroxide concentration. The system was developed by dentists, who aimed to achieve a long-lasting white smile with virtually no sensitivity in the comfort of your own home. The system has no strips, no hassles and no pain. Most importantly there's no sensitivity when using this product and you see results in just two weeks.

Niu Nait is available in two flavors, mint and strawberry, and come in 4, 10 or 50 syringe kits.



16% Concentration

- #08-00777.....Standard 10 Syringe Kit, Mint
- #08-00877.....Standard 10 Syringe Kit, Strawberry
- #08-00977.....Mini 4 Syringe Kit, Mint
- #08-01077.....Mini 4 Syringe Kit, Strawberry
- #08-03577.....Bulk 50 Syringe Kit, Mint
- #08-03677.....Bulk 50 Syringe Kit, Strawberry

22% Concentration

- #08-01177.....Standard 10 Syringe Kit, Mint
- #08-01277.....Standard 10 Syringe Kit, Strawberry
- #08-01377.....Mini 4 Syringe Kit, Mint
- #08-01477.....Mini 4 Syringe Kit, Strawberry
- #08-03777.....Bulk 50 Syringe Kit, Mint
- #08-03877.....Bulk 50 Syringe Kit, Strawberry

Niu Frost Whitening Pen

Niu Frost Whitening is 5% to 8% whitening hydrogen peroxide that performs as advanced, state-of-the-art whitening system. It is a high viscosity, crystal clear gel with a synergetic combination of elements to minimize sensitivity. The gel derives its unique flavor and aroma from all natural ingredients and it has a unique patented blend of soothing agents, conditioners, desensitizers and a high water content to reduce sensitivity.

We offer %5 kits and 8% kits in mint or strawberry flavors. Also, kits come with 1 or 2 whitening pens.



- #08-03177.....5% Standard Kit, 2 Pens, Mint
- #08-03277.....8% Standard Kit, 2 Pens, Strawberry
- #08-03377.....5% Touch Up Kit, 1 Pen, Mint
- #08-03477.....8% Touch Up Kit, 1 Pen, Strawberry

DENTAL DAMS

Dental Dams

These Dental Dams are made of highly flexible material that provide increased stretch and manipulation to reduce cross-contamination. Works well to elimination frequent rinsing and use of multiple cotton rolls during a procedure. The dam minimizes risk of patient aspiration of foreign object. Also, the sheets have an excellent tensile and tear strength. Made with powder-free latex 5 mg per sheet.



(127 x 127 mm)

- #26-00511.....5" x 5" Thin Blue, 52/Box
- #26-00611.....5" x 5" Thin Green, 52/Box
- #26-00111.....5" x 5" Medium Blue, 52/Box
- #26-00211.....5" x 5" Medium Green, 52/Box
- #26-00911.....5" x 5" Heavy Blue, 52/Box
- #26-01011.....5" x 5" Heavy Green, 52/Box



(152.4 x 152.4 mm)

- #26-00711.....6" x 6" Thin Blue, 52/Box
- #26-00811.....6" x 6" Thin Green, 52/Box
- #26-00311.....6" x 6" Medium Blue, 52/Box
- #26-00411.....6" x 6" Medium Green, 52/Box
- #26-01111.....6" x 6" Heavy Blue, 52/Box
- #26-01211.....6" x 6" Heavy Green, 52/Box



ETCHANT GELS

Seity Etchant Gels

Seity Etchant Gel is a superior quality dental etching gel specially formulated for maximum working and handling characteristics. The syringe dispensing gel stays in place and will not run onto exposed tissue. Get precise placement through small-gauge, pre-bent applicator tips. It washes off easily without leaving any residue.



- #07-02777.....37% Etchant Gel, Blue, 12g syringe
- #07-02877.....37% Etchant Gel, Green, 12g syringe

Zinc Oxide Powder USP

Zinc Oxide Powder USP is high in quality and purity. It's very fine, amorphous white or yellowish-white color that's free from gritty particles. Sold individually.



- #11-47577.....Zinc Oxide Powder USP, 1.5 oz. (42 g)
- #11-47578.....Zinc Oxide Powder USP, 1 lb. (453 g)

IMPRESSION MATERIAL

HB Bite Registration

HB Bite Registration provides a tough, precise occlusal record that resists compression and bounce-back during articulation.

The Regular Set has a working time of 2 minutes and a set time of 1 minute, making a total set time of 3 minutes. The Fast Set Fast Set has a working time of 30 seconds and a set time of 1 minute, making a total set time of 1:30. Available in 50 ml cartridges and 2/Pkg.



- #6081105HB Bite Registration, Regular Set
- #6081104HB Bite Registration, Fast Set



HB VPS

HB VPS (polysiloxane) impression material is virtually for any 1- or 2-step impression technique. Available in light, medium and heavy-body viscosities, HB VPS tray materials are highly hydrophilic for precision models. This ensures an accurate impression and makes a pleasant experience for both the dentist and patient. Each type of HB VPS comes in a convenient 50ml cartridge with 4/Pkg.



- #6081707VPS Heavy Fast
- #6081708VPS Heavy Regular
- #6081709VPS Medium Fast
- #6081710VPS Medium Regular
- #6081711VPS Light Fast
- #6081712VPS Light Regular



Consistency Bite Registration

Consistency Bite Registration is a fluffy mousse-like vinyl polysiloxane material. It's used for bite registration, copying transfers, and primary impression with bite trays. Available in 50ml cartridge with 2/Pkg.



- #6081000Bite Registration, Fast Set
- #6081100Bite Registration, Regular Set



Consistency VPS

Consistency VPS is a multipurpose and cost-effective high performance cartridge system available in regular and fast set. All viscosities are available extremely accurate, hydrophilic and strong. Use as a final impression material or an alginate substitute. Each type of Consistency VPS comes in a convenient 50ml cartridge with 4/Pkg.



- #6081200VPS Heavy Body, Fast Set
- #6081300VPS Heavy Body, Regular Set
- #6081400VPS Medium Body, Fast Set
- #6081500VPS Medium Body, Regular Set
- #6081600VPS Light Body, Fast Set
- #6081700VPS Light Body, Regular Set



Dispensing Gun

This Dispensing Gun is used to extrude impression materials and bite registration materials from cartridges. It consists of a release lever and a plunger, which are coordinated simultaneously to load and remove the cartridge. The gun also consists of a trigger to dispense the material. Works with 1-to-1 and 2-to-1 cartridges. Available in two styles for HP cartridges and old style cartridges.



- #6081900HP Dispensing Gun, Split Cartridge

Mixing Tips

These high performance Mixing Tips are for all HP style 50 ml impression material cartridges.

- #6081800HP Mixing Tips, Green, 48/Pkg

Rubber Mixing Bowls

These Rubber Mixing Bowls are manufactured from quality black, flexible rubber.



- #1150010Small, 3" (76.2 mm) dia.
- #1150020Medium, 4" (101.6 mm) dia.
- #1150030Large, 4-3/4" (120.6 mm) dia.
- #1150040X-Large, 6-1/2" (165.1 mm) dia.
- #1150050Jumbo, 9" (228.6 mm) dia.

Disposable Impression Trays

Our Plastic Impression Perforated Trays are disposable. The keyway perforations and side-wall slots ensures enough retention. The posterior rimlock feature minimizes secondary flow or gagging. The finger rest is for easy removal and the smooth edges provide patient comfort. The rigid design captures the best impression without distortion. Available in Blue, 12/bag, 10 bag/box. Comes in 9 sizes (#1-#9) for easy patient anatomy matching.



- #15-00110.....#1, Large Upper
- #15-00210.....#2, Large Lower
- #15-00310.....#3, Medium Upper
- #15-00410.....#4, Medium Lower
- #15-00510.....#5, Small Upper
- #15-00610.....#6, Small Lower
- #15-00710.....#7, Upper Right/Lower Left
- #15-00810.....#8, Upper Left/Lower Right
- #15-00910.....#9, Anterior
- #15-01010.....#10, Upper Anterior



Mizzy Impression Compound Cakes & Sticks

Mizzy Compound Cakes & Sticks are pleasantly scented and flavored compound for taking full and partial impressions, and making preliminary impression trays.

Brown Medium Heat Compound:

- Plastic at 135°F (57.2°C)
- Ideal for snap impressions
- Working Temperature: 125°F - 135°F (52°C - 57°C)

Red Low Heat Compound:

- Maximum visibility of detail
- Working Temperature: 115°F - 125°F (46°C - 52°C)

Green Sticks:

- Has the most diverse working range
- Renders sharp details
- Will not fracture while trimming
- Chills evenly without warping
- Maximum contrast



- #6060700Brown, 15 Sticks, 4.5 oz. (128 g)
- #6060300Red, 15 Sticks, 4.5 oz. (128 g)
- #6060500Green, 15 Sticks, 4.5 oz. (128 g)
- #6060600Brown, 8 Cakes, 8 oz. (227 g)
- #6060100Red, 8 Cakes, 8 oz. (227 g)



MATRIX MATERIALS

Matrix Retainers, Rolls and Bands

Matrix Retainers

Tofflemire Matrix Retainers are made of high quality medical grade stainless steel. Available in universal, adult and child sizes.


- #19-02012.....Matrix Retainer Tofflemire Universal, Each
- #19-02112.....Matrix Retainer Tofflemire CA Adult, Each
- #19-02212.....Matrix Retainer Tofflemire CA Child, Each



Matrix Rolls & Bands

Matrix Rolls & Bands are made of high quality medicals grade stainless steel. These will not tear under stress and has smooth edges for patient comfort. The T-bands are made of brass. Available in 0.0015" thickness and in 0.002 gauge. Comes in 10-foot rolls and 100/Pkg.

- #19-00112.....Matrix Roll, Stainless Steel 3/16", 10 ft. Roll
- #19-00212.....Matrix Roll, Stainless Steel 1/4", 10 ft. Roll

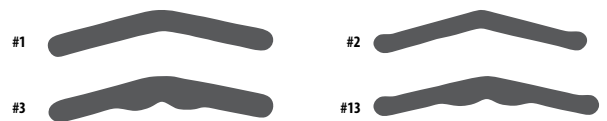

- #19-00512.....Matrix Band Tofflemire #1, 12/Pkg
- #19-00612.....Matrix Band Tofflemire #2, 12/Pkg
- #19-00712.....Matrix Band Tofflemire #3, 12/Pkg
- #19-00812.....Matrix Band Tofflemire #13, 12/Pkg



- #19-00412.....Matrix T-Band Large Curved, 100/Pkg
- #19-00312.....Matrix T-Band Small Curved, 100/Pkg



Mylar Strips

These Mylar Strips are made of genuine DuPont Mylar 0.002 gauge/60mm thick. The strips have smooth finished edges for patient comfort. They resist solvents, are very strong, will not stretch curl or break. Available in 4" x 3/8" (101.6 x 9.5 mm) strips.



- #36-01022.....Mylar Strips, 100/Pkg
- #36-01122.....Mylar Strips, 1000/Pkg



MEDICAMENT MATERIALS

Eugenol USP

Eugenol USP is antiseptic and analgesic. Used with zinc oxide powder for temporary cements and with zinc oxide and resins. It's an ideal antiseptic in root canal therapy.

- #06-00677.....Eugenol USP, 4 oz. (.12 L)
- #06-02077.....Eugenol USP, 1 oz. (.02 L)



Hemox-A

This Hemostatic Agent is a 25% buffered aluminum chloride hemostatic solution.

- #25-03677.....Hemox-A, 1 oz. (.02 L)



NEEDLELESS INJECTOR

Syrijet Needleless Injector



Syrijet Mark II Needleless Injector is a precision instrument designed for simple, reliable operation. Saves time and effort, eliminates unnecessary patient discomfort and needle apprehension. If used as prescribed and cared for routinely, it will give you and your patients years of satisfaction.

Syrijet provides operative & surgical anesthesia for:

- Cavity preparation in upper and lower anterior teeth
- Crown and bridge preparation

Extraction:

- All permanent upper and most lower anterior teeth
- All deciduous teeth
- Deep scaling
- Flap removal
- Gingivectomy
- Excision of soft tissue lesions (fibroma, papilloma, etc.)
- Incision and drainage
- Placement or removal of sutures
- Revision of hypertrophic tissue
- Reduction of tuberosity
- Excision of mucous retention cyst (lip)
- Tissue biopsies
- Curettage
- Copper band impressions
- Cord retractions
- Post-op removal of bone
- Application of arch bars and ligature wires



Syrijet sub-topical anesthesia for painless needle insertions such as:

- Posterior superior alveolar nerve block
- Inferior alveolar nerve block
- Pre-I.V. needle insertion

- #6170000Syrijet

SURGICAL PRODUCTS


Sterile Surgical Blades

Sterile Surgical Blade perfectly fit with scalpel handles. The blades are available in carbon and stainless steel. They come in a sterile package with each blade individually wrapped in a peel pouch.



Carbon Steel

- #27-00110.....#10, 100/Box
- #27-00210.....#11, 100/Box
- #27-00310.....#12, 100/Box
- #27-00510.....#15, 100/Box
- #27-00710.....#20, 100/Box
- #27-00810.....#21, 100/Box
- #27-00910.....#22, 100/Box
- #27-01010.....#23, 100/Box
- #27-01110.....#24, 100/Box
- #27-01310.....#25, 100/Box

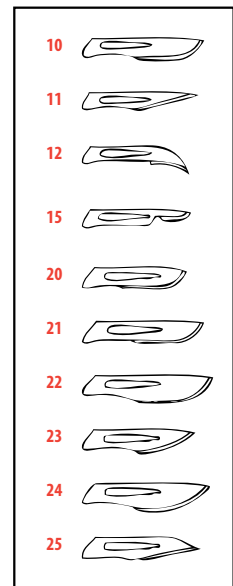

Stainless Steel

- #27-02110.....#10, 100/Box
- #27-02210.....#11, 100/Box
- #27-02310.....#12, 100/Box
- #27-02510.....#15, 100/Box

Surgical Scalpels

These Surgical Scalpels allow the clinicians to focus on the surgical site rather than the instrument. It minimizes the risk of injury during passing and disposing. Made of carbon steel for better sharpness Comes in sterile packaging, with each scalpel individually wrapped in a peel pouch.



- #27-01510.....#10, 10/Box
- #27-01610.....#11, 10/Box
- #27-01710.....#12, 10/Box
- #27-01910.....#15, 10/Box



FLUORIDES80
PREVENTATIVES.....81

PROPHY PRODUCTS81
TOPICAL ANESTHETICS82

FLUORIDES

Gelato APF Fluoride Gel

Gelato APF Gels are economical and quick application fluoride treatment. The gel is a 60-second acidulated phosphate fluoride that contains 1.23% fluoride ion. The low PH provides maximum fluoride uptake in less than 60 seconds and the improved thixotropic properties ensure closer adherence to enamel and interproximal areas. The smooth creamy gel will not run and thickens during treatment to prevent patient gagging. The gel is gluten free and offered in many different flavors for patient satisfaction and there's no bitter aftertaste following application.



16 oz. (.48 L) Flip Top Bottle

- | | |
|-----------------------------|------------------------------|
| #24-08677....Bubble Gum | #24-03377....Strawberry |
| #24-08277....Cherry | #24-10177....Cotton Candy |
| #24-01877....Grape | #24-10677....Mango Smoothie |
| #24-11077....Orange Vanilla | #24-09977....Marshmallow |
| #24-08777....Piña Colada | #24-04477....Mint "Dye Free" |
| #24-00577....Peppermint | |

1 Gallon (3.78 L) Bottle

- | | | |
|-------------------------|---------------------|----|
| #24-14177....Bubble Gum | #24-00677....Cherry | CE |
|-------------------------|---------------------|----|

Gelato Neutral PH Fluoride Gel

Gelato Neutral PH Fluoride Gels are economical and quick application fluoride treatment. The Neutral PH formula provides total treatment for a wide patient profile. It is ideal for patients with cosmetic restorations, porcelain or resin. It provides the perfect non-acidic balance, allowing patients to tolerate and accept fluoride applications with comfort with a 4 minute thixotropic formula. The gel is gluten free and there's no bitter aftertaste following application.



- | |
|---|
| #24-01777.....Bubble Gum, 16 oz. (.48 L) |
| #24-01677.....Strawberry, 16 oz. (.48 L) |
| #24-00777.....Dye-Free Mint, 16 oz. (.48 L) |



Gelato APF Foam Fluoride

Gelato Foam Fluoride is a 1.23% Acidulated Phosphate Fluoride formula that provides excellent, consistent and efficacious coverage. The light foam consistence makes for easy, fast and more pleasant application. The fluoride stays in under bit pressure to eliminate patient gagging.



4.4 oz. (.13 L)

- | | |
|----------------------------|--------------------------|
| #24-08977.....Bubble Gum | #24-04377.....Mint |
| #24-08877.....Cherry | #24-05577.....Strawberry |
| #24-15777.....Cotton Candy | #24-04177.....Grape |

Neutral 4.4 oz. (.13 L)

- | | |
|--------------------------|--------------------|
| #24-25477.....Bubble Gum | #24-26877.....Mint |
|--------------------------|--------------------|

Gelato Home Care Rinse

Gelato Home-Care Fluorides is a great way for patients to protect their teeth affordably and effectively at home.

The 0.63% Stannous Fluoride Perio Rinse has been shown to be an effective therapy to prevent tooth decay while strengthening and protecting roots.

The antimicrobial and alcohol-free Gelato Perio Rinse relieves tooth sensitivity, reduces gingival inflammation, helps inhibit plaque build-up, prevents demineralization and promotes re-mineralization.

The Gelato Home Care 0.40% Stannous Fluoride Brush-on Gel reduces sensitivity and plaque biofilm, while inhibiting the cariogenic micro process.

Gelato Home Care can be easily applied with a tooth brush or a fluoride tray. Contains Xylitol, which is a proven inhibitor of plaque biofilm formation.



10 oz. (.29 L)

- | |
|--|
| #24-11877.....0.63% Stannous Fluoride Rinse, Mint, |
| #24-10377.....0.63% Stannous Fluoride Rinse, Raspberry |

4.3 oz. (.12 L)

- | |
|--|
| #24-10477.....0.40% Stannous Fluoride Gel, Mint |
| #24-10577.....0.40% Stannous Fluoride Gel, Red Berry |
| #24-09177.....1.1% Neutral Sodium Fluoride, Cherry |
| #24-09277.....1.1% Neutral Sodium Fluoride, Mint |



Hinged EZ Fluoride Trays

These Hinged EZ Fluoride Trays are an excellent way to dispense fluoride treatments with upper and lower arches attached together. The tray is made of closed celled, 4 lb. density foam for good rigidity. With excellent anatomical depth for gingival line, the tray promotes full coverage of the fluoride applications.




- #9525720Small - Yellow, 100/Pkg
- #9525890Small/Medium - Yellow/White, 100/Pkg
- #9525390Medium - White, 100/Pkg
- #9525050Large - Blue, 100/Pkg



Mizzy Prevent Trays

Mizzy Prevent Trays include exclusive 'fold-away' hinge that prevents gag reflex and assists in easy separation for single-tray use. The tray also has extra-long locking tabs to ensure cleanliness and ease of insertion and removal. The anatomical design and buccal ridge promotes fluoride coverage of the entire tooth while contoured walls reduce overflow and prevent ingestion of fluoride.

Exclusive 'combo' assortment provides all three sizes in one convenient, re-sealable, freestanding bag. The generous 100-count bag offers a substantial savings over the leading national brands.

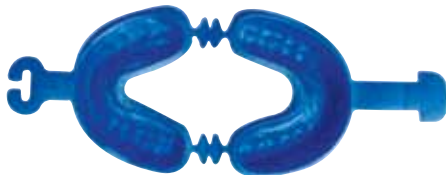


- #7000260Dual, Small, 100/Pkg
- #7000261Dual, Medium, 100/Pkg
- #7000262Dual, Large, 100/Pkg



PREVENTATIVES

Tongue Scrapers

Did you know that more than 85% of bad breath originates in the mouth, not the stomach? Tongue Scrapers provide a handy way to clean the tongue's surface and make sure that bad breath bacteria are scraped away. Cleaning on a regular basis is known to significantly reduce bacteria and food particles on the tongue. Reduces mouth odors, overall bacteria and helps to keep the mouth fresh and clean.

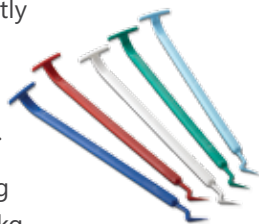

- #9541130Assorted, 100/Pkg
- #9541140Light Blue, 100/Pkg
- #9541160Dark Blue, 100/Pkg
- #9541190Green, 100/Pkg
- #9541220Red, 100/Pkg
- #9541240White, 100/Pkg

PROPHY PRODUCTS

Gelato Prophylaxis Paste



Gelato has been a forerunner of great-tasting prophylaxis paste, bringing on new flavors such as Pina Colada and Orange Sherbet. But what also makes Gelato Prophylaxis Cups great is a non-splatter formula that contains 1.23% APF with a unique blend of cleaning and polishing agents.

Gelato paste comes in boxes of 200 individual disposable cups for convenient use. The disposable cups also eliminate cross contamination and include a prophy ring for ease of application on the patient's teeth. For more options, the paste also comes in 6 ounce and 12 ounce jars.

Gelato Prophylaxis Paste is offered in 4 different grits: fine, medium, coarse and x-coarse. The fine grit is best for light stain removal and amalgam polishing. It's recommended to use the fine grit on children. For normal cleaning and polishing the medium grit is an ideal choice. The coarse grit is used in medium to heavy stain and plaque removal.



12 oz. Jar (.35 L)

- #24-10877.....Fine Bubble Gum
- #24-09077.....Medium Cherry
- #24-08177.....Medium Mint
- #24-08077.....Medium Piña Colada
- #24-03977.....Coarse Cherry
- #24-04077.....Coarse Mint

200 Cups

- #24-02877.....Fine Bubble Gum
- #24-02377.....Medium Bubble Gum
- #24-02477.....Medium Cherry
- #24-02277.....Medium Mint
- #24-02577.....Medium Piña Colada
- #24-11677.....Medium Orange Sherbert
- #24-15877.....Medium Raspberry
- #24-02677.....Medium Assorted
- #24-07577.....Coarse Bubble Gum
- #24-02977.....Coarse Cherry
- #24-03077.....Coarse Mint
- #24-03177.....Coarse Piña Colada
- #24-11777.....Coarse Orange Sherbert
- #24-15977.....Coarse Raspberry
- #24-02777.....Coarse Assorted
- #24-05077.....X-Coarse Cherry
- #24-05177.....X-Coarse Mint
- #24-15177.....Prophy Ring



ORAL HEALTH

Gelato Primal Prophylaxis Cups

Gelato Primal is the only prophylaxis paste formulated with casein – a phosphor-amino acid derivative – and tricalcium phosphate. The unique combination stimulates the re-mineralization of tooth enamel while cleaning and polishing agents maximize stain removal and minimize enamel loss.

Gelato Primal comes in a wealth of flavors to satisfy even the pickiest patients. The paste comes in boxes of 200 individual disposable cups for convenient use. The disposable cups also eliminate cross contamination and include a prophy ring for ease of application on the patient's teeth.



200 Cups

- #24-31077.....Fine Bubble Gum
- #24-30477.....Medium Assorted
- #24-30277.....Medium Bubble Gum
- #24-30177.....Medium Cherry
- #24-30077.....Medium Mint
- #24-30377.....Medium Tropical Punch
- #24-30977.....Coarse Assorted
- #24-30777.....Coarse Bubble Gum
- #24-30677.....Coarse Cherry
- #24-30577.....Coarse Mint
- #24-30877.....Coarse Tropical Punch

Fortis Disposable Prophyl Angle

The Fortis Prophyl Angle has been engineered to run longer and smoother with less chatter, providing superior comfort with less hand fatigue than traditional angles. Independent testing has confirmed that the design of the Fortis angle out-performed all other disposable prophyl angles with respect to ease of use.

These prophyl angles have a second generation gear design with ergonomic grips that reduce the potential fatigue through the hand and arm. The cups are 100% latex free and are available in soft cup, firm cup, pedo cup and tapered brush.



- #24-13077.....Firm Cup, 100/Pkg
- #24-13177.....Soft Cup, 100/Pkg
- #24-13277.....Pedo Cup, 100/Pkg
- #24-13377.....Tapered Brush, 100/Pkg

Prophylaxis Polishing Brushes

These Prophyl Brushes are used for polishing and removal of heavy stains. The brushes are made of fine nylon bristles assembled to prevent the possibility of pulling out bristles.



- #24-40014.....Prophy Brush RA, Flat, 144/Jar

Prophylaxis Polishing Cups

These Prophy Cups are used for polishing and removal of heavy stains. The cups are 100% latex free and made of FDA-listed synthetic material. These premium quality cups minimize chatter and heat build-up. The cups are webbed to conform to tooth surface and retain polishing material. Available in snap-on, latch type and screw-type.



- #24-40514.....Prophy Brush RA, Latch Type, 144/Jar
- #24-40614.....Prophy Cups Screw Type, 144/Jar
- #24-40714.....Prophy Cups Snap-on, 144/Jar

TOPICAL ANESTHETICS

Gelato Topical Anesthetic Gel

The Gelato 20% Benzocaine Anesthetic Gels are fast-acting with no systemic absorption. The 20% benzocaine provides temporary relief of pain during procedures, including local anesthetic injections, periodontal curettage, impression taking, scaling, intra-oral radiographs, root planning and prophylaxis. The gel comforts patients with ulcers, wounds or other minor mouth irritations. Also, Gelato has no bitter aftertaste.



1 oz. Jar (.03 L)

- #03-02419....Bubble Gum
- #03-02319....Cherry
- #03-02019....Raspberry
- #03-02519....Piña Colada
- #03-02819....Mint
- #03-01719....Piña Colada "Dye Free"
- #03-02219....Strawberry
- #03-03519....Banana
- #03-01319....Grape
- #03-02119....Mango

Gelato Plus Anesthetic Gel

The Gelato "PLUS" Anesthetic Gel is fast acting and long lasting! This gel contains 18% benzocaine and 2% tetracaine. The thick, smooth gel reduces pain immediately. The 18% benzocaine produces topical anesthesia within 20 seconds of application while the 2% tetracaine ensures topical anesthesia for 15-20 minutes.



1 oz. Jar (.03 L)

- #03-01019....Raspberry
- #03-00919....Cherry
- #03-01119....Mint

Gelato Benzocaine Anesthetic Spray

The Gelato Benzocaine Anesthetic Spray is great for use on a large area that requires anesthesia. Containing 20% benzocaine the spray works quickly and effectively – approximately 15 to 20 seconds. The spray also contains 25 dispensing tips. Replacement tips are sold separately.



- #03-01519....Metered Spray
- #03-01619....Replacement Tips



THERMAL PLASTICS..... 84
VACUUM FORMERS..... 87
VACUUM FORMING ACCESSORIES..... 81

PRO-FORM/VACUUM FORMING

THERMAL PLASTICS

Pro-form Glitter Guards

Pro-form athletic mouthguard line got a little more flash with Glitter Guards. The Glitter Guard material adds a bit of sparkle to the Pro-form lauded line, which delivers protection, affordability and style. Available in packages of 6.



- | | |
|----------------------|------------------------|
| #7965020Orange | #7965060Pink |
| #7965010Blue | #7965040Green |
| #7965030Yellow | #7965070Assorted |
| #7965050Purple | |



Pro-form Tie-Dye Mouthguards

The Pro-form line now boasts Tie Dye designs to the extensive line. The Tie Dye Laminates are made the same way all Pro-form laminates to ensure the long-proven quality and protection athletes of all levels need. Available in packages of 6.



- | | |
|--------------------------|------------------------|
| #7911111Rainbow | #7965000Shamrock |
| #7954000Patriot | #7934000Flamingo |
| #7924000Wilderness | #7974000All in |
| #7944000Midnight | |



Fun Mouthguards

The Pro-form line now carries Fun Guard designs to add even more options to the already extensive line of styles and colors. The Fun Guard laminates are made the same way all Pro-form laminates to ensure the long-proven quality and protection athletes of all levels need. The laminates come in packages of 6.



- | |
|---|
| #9600006Diagonal (red/green/blue/yellow) |
| #9600929Polka Dots (red background) |
| #9598079Polka Dots (blue background) |
| #9601039Polka Dots (white background) |
| #9599075The Fangs (black with white fangs) |
| #9599077Dripping Blood (black with red drops) |



Pro-form Mouthguard Laminates

Pro-form manufactures the only pre-made laminated mouthguard material in the industry. This material has long been considered the leader in custom-made mouthguards. By laminating two layers of soft EVA together, the laminate has excellent tensile strength. All of these features make the Pro-form mouthguard laminate the material of choice for custom-made mouthguards.



- | | |
|------------------------------------|----------------------------------|
| #9596920Clear | #9598730Black |
| #9600850Orange | #9600910Red |
| #9601040Yellow | #9598060Blue |
| #9600820Maroon | #9600870Purple |
| #9601020White | #9600180Assorted |
| #9600800Green | #9600888Breast Cancer Pink |
| #9600886Pink | |
| | |
| #9598360Blue/Green | #9598280Orange/White |
| #9598240Black/White | #9598260Blue/White |
| #9598210Blue/Red | #9598400Orange/Blue |
| #9598110Black/Yellow | #9598190Green/Yellow |
| #9598090Black/Red | #9598320Black/Orange |
| #9598340Purple/Yellow | #9598450Purple/White |
| #9598300Maroon/Yellow | #9598380Green/White |
| #9598150Blue/Yellow | #9598420Red/Yellow |
| #9598130Red/White | #9598710Assorted |
| | |
| #9598840Red/White/Blue | #9600170Assorted |
| #9598900Canada Red/White/Red | |



Pro-form Sample Pack

Keystone Industries has put together a sample of various thermoplastics. The package contains all of the thermoplastics presently used in the dental industry and comes with technical booklet that details better-known procedures. It contains all of the Pro-form laminated materials, base plates, custom trays, EVA for bleaching and model duplication, retainers, and much more. Each material is labeled with the name of the product and the various applications for which the material can be used. Can easily be cross-referenced to this catalog and the technical booklet.



- | |
|------------------------------------|
| #7001211Pro-form Sample Pack |
|------------------------------------|



Mouthguard Resin Sheets

The .160 thick Pro-form Mouthguard Resin Sheets are soft for energy absorption, adding more protection. The resin is perfect for mouthguards and niteguards. Available in sheets of 5" x 5" (127 x 127 mm).



25/Pkg		300/Pkg
#9597940 Clear.....	#9598010
#9597740 Red.....	#9597910
#9597680 Blue	#9597880
#9597700 Green.....	#9597890
#9597760 Yellow	#9597930
#9597660 Black.....	#9597870
#9597720 Orange.....	#9597900
#9597750 White.....	#9597920

Youth Mouthguard

The .120 thick Youth Mouthguard Laminates are great for little kids who are playing sports. Made of soft EVA, for energy absorption and is thin enough to fit the smaller mouth of a child. Available in sheets of 5" x 5" (127 x 127 mm).



25/Pkg		300/Pkg
#9611240 Clear.....	#9611280
#9611241 Red.....	#9611281
#9611242 Blue	#9611282
#9611243 Green.....	#9611283
#9611244 Yellow	#9611284
#9611245 Black.....	#9611285

Extra-Pro Laminates

Keystone Industries now manufactures a triple layered, extra thick (.200) laminated material for mouthguards. Made for optimum protection for athletes like pros, colleges and varsity high school players. Available in solid colors in packages of 12. If you need extra bulk and protection, you might want to try our triple-layered Extra-Pro Laminate.



#9601215Clear	#9601225Blue
#9601485Green	#9601275Black
#9601515Red	#9601350Assorted
#9601535Yellow		

Mouthguard Straps

Keystone Industries offers easy-to-attach helmet straps to be used with Pro-form mouthguards. The strap works great for any sport that has a helmet, such as football, hockey and lacrosse. The strap is especially helpful due to its safety release feature. This allows the mouthguard to break away under less than 12 pounds of pressure.



#9597420Clear, 25 straps
#9597400Clear, 12 straps
#9597360Black, 25 straps
#9597340Black, 12 straps

Pro-Flex & Dual Combo


The Pro-flex and Dual Combo is an excellent way to make niteguards. First, vacuum form a dual laminate onto the patient's cast. This delivers a hard bite plate with a built-in soft liner. Then, modify the occlusal surface by adding the semi-flexible Pro-flex NFC acrylic resin. The final result is a hard bite plate modified to the patient's needs that is slightly flexible with a soft inner liner for patient comfort. This is two great products combined into one terrific final appliance. Kits contain 160g powder with 4 oz. liquid.



#9640880Pro-flex Clear NFC Kit
#9603580Dual Laminate Kit

Bleaching Laminates

It's become an acceptable dental technique to bleach teeth using carbamide peroxide and a bleaching tray. The Bleaching Laminate is ideally suited for making the bleaching tray and withstanding bleaching products. It is thin, only .020 inches thick once it's formed over the model. It has a foam liner to absorb the bleach, thus decreasing the bleaching time for the patient. The Bleaching Laminate is our best-selling item for bleaching processes.

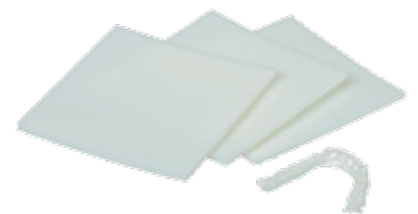

#9605780Bleaching Laminates, 50/Pkg
----------	----------------------------------

PRO-FORM/VACUUM FORMING

Dual Laminates/E-Gasket



The Pro-form Dual Laminate (E-gasket) is a unique combination of two materials formed into a single laminated sheet. It was made with bruxing cases in mind. The soft liner assures patient comfort and the hard surface assures long-lasting wear. Also, acrylic will bond to the hard surface to allow the dental technician to modify the occlusal surface. Available in 3 different thicknesses: 2, 3 and 5mm.



- #96047002mm E-Gasket, 12 sheets/Pkg
- #96035803mm Dual Laminates, 12 sheets/Pkg
- #96036055mm Anti Snore Device, 12 sheets/Pkg

Niteguards



This laminate or layered soft/soft material was made specifically for the making of niteguards. The layering gives the thermoplastic a more abrasion-resistant characteristic that will withstand grinding better than traditional soft sheet materials.



- #9602550Niteguards, 12 sheets/box

Base Plate Material



This Base Plate Material is used for denture base or wax bite plates. Made of pure high-impact styrene, the material melts at a low temperature and is easy to work with. Available in 5" x 5" (127 x 127 mm) sheets and also in round.



- | | | |
|------------------------------------|--|----------------|
| 100/Pkg | | 625/Pkg |
| #96161200.060 thickness..... | | #9616180 |
| #96161300.080 thickness..... | | #9616190 |
| #96161400.100 thickness..... | | #9616200 |

Coping Material



These cloudy, stiff polypropylene sheets are excellent for fabricating temporary crowns, bridges and copings. The material gives good definition and will not adhere to acrylics. Available in 5" x 5" (127 x 127 mm) sheets.



- | | | |
|------------------------------------|--|----------------|
| 50/Pkg | | 625/Pkg |
| #96134700.020 thickness..... | | #9613540 |
| #96134800.030 thickness..... | | #9613550 |
| #96135100.040 thickness..... | | #9613530 |

Crown & Bridge Material



Our Crown & Bridge Material is a thin, clear rigid material that works great for temporary bridges. The material is easy to work with while it does not bond to acrylic. Available in 5" x 5" (127 x 127 mm) sheets.



- #96136800.020 thickness, 50/Pkg
- #96137000.020 thickness, 625/Pkg

Retainer Material



This stiff but clear material is ideal for orthodontic retainer. The long-lasting material enables acrylics to stick, but also provides the long-lasting form that retainers need. Available in 5" x 5" (127 x 127 mm) sheets.



- #9614810Clear 0.030 thickness, 50/Pkg
- #9614870Clear 0.030 thickness, 625/Pkg
- #9615025Super Clear 0.030 thickness, 25/Pkg
- #9614816Super Clear 0.030 thickness, 50/Pkg
- #9614875Super Clear 0.030 thickness, 625/Pkg

Splint Material



This stiff clear material is ideal for hard splints, thin splints, surgical trays, orthodontic retainers and bleaching trays. The material sticks to acrylics for various dental appliances. The versatile splint material will surely be used in any lab or office. Available in 5" x 5" (127 x 127 mm) sheets.



- | | | |
|------------------------------------|--|----------------|
| 50/Pkg | | 625/Pkg |
| #96148000.020 thickness..... | | #9614860 |
| #96148300.040 thickness..... | | #9614880 |
| #96147800.060 thickness..... | | #9614840 |
| #96147900.080 thickness..... | | #9614850 |
| #96148200.100 thickness..... | | #9614950 |

Tray Material

The Tray Material is great for custom impression trays. Made of pure high-impact styrene, the material comes in two different thicknesses to provide versatility. Spacer foam is also included with the tray material. Available in 5" x 5" (127 x 127 mm) sheets and also in round.



- #96173200.125 thickness, 100/Pkg
- #96173500.125 thickness, 625/Pkg
- #96173300.150 thickness, 100/Pkg
- #96173600.150 thickness, 625/Pkg

Soft EVA

Keystone Industries' Soft EVA is great for the fabrication of customer fluoride trays, bleaching trays and model duplication. The material is soft, clear and easily forms. Also the material can be trimmed effortlessly. The material is ideal for soft brushing patients. Available in 5" x 5" (127 x 127 mm) sheets.



- | | | |
|------------------------------------|--|----------------|
| 25/Pkg | | 300/Pkg |
| #95969800.040 thickness..... | | #9597040 |
| #95970900.060 thickness..... | | #9597130 |
| #95971500.080 thickness..... | | #9597240 |
| #96112400.120 thickness..... | | #9611280 |

Spacer Foam

Reusable spacer foam, one size fits all. Available in sheets of 5" x 5" (127 x 127 mm).



- #9617430Spacer Foam, 25/Pkg

CE

CE

CE

VACUUM FORMERS

Machine IV

The Keystone Dental Vacuum Forming Machine is the most popular vacuum forming machine in the dental industry. The price, features and reliability make it the choice dental forming machines out there. The Machine IV is the newest of Keystone Industries' long-standing machines and boasts new and improved features that make your vacuum forming work easier than ever.

The base of the machine is made from super strong, lightweight space-age thermoplastic material that is designed for high-temperature application. The easy-to-clean polymer base has no paint or powder coating that will chip or scratch off, so the machine will look brand-new for years to come. The plastic housing dampens vibrations better than traditional cast aluminum, which results in a quieter unit.

The system has a simple reliable touch pad control with several safety features such as:

- Vacuum control turns off the heater and the heater control turns off the vacuum. This ensures that the heater will never be left on continuously for many hours. This protects the heater and ensures safety in the workplace. Whenever there is a need to run many cycles in succession simply turn the vacuum off using the heater control. This will turn off the vacuum and turn the heat back on. The heater will still be sufficiently warm to start your next model.
- Built-in safety timer that shuts down the heater if not used for 15 minutes.
- Built-in over temperature sensor on the circuit board powers down the entire unit in the unlikely event timer fails and the unit is left on overnight.

Overall, the Machine IV's compact and lightweight design takes less space than other vacuum formers and is can be easily picked up and stowed away when not in use. The vented heated cover allows heat to quickly dissipate, allowing the machine to cool quicker. Also, a locking handle allows varying the height of the material play during the heating process.

The Machine IV is great for processing bleaching trays, sports mouthguards, implant stints, temporary splints, customer impression trays, denture bases, copings, temporary crowns, bridges and orthodontic appliances.



- #7000338110 Volt
- #7000339220 Volt



48 - 72 hour Service

We guarantee the quickest handling of your repairs. Therefore we have two repair centers:

52 West King Street
Myerstown, PA 17067
USA
Phone: (800) 333-3131

Werner-von-Siemens-Str. 14a
78224 Singen
Germany
Phone: (+49) 7731 912101
Fax: (+49) 7731 912102

PRO-FORM/VACUUM FORMING

Machine III

The Machine III has been a staple in Keystone Industries' long-lasting line of highly-recommended vacuum former machines. The price, features and reliability make it the first choice in dental vacuum formers. The machine has more power than any other dental vacuum former, which results in a more accurate and definite dental thermoplastic appliance.

The aluminum construction is long-lasting, eliminating the possibility of rust. The heating element is symmetrically engineered for even heat distribution throughout the thermoplastic material.



#7000330110 Volt
#7000332220 Volt

Single Chambered Vacuum Former

The Single Chambered Vacuum Former is an economical answer that can still excellently fabricate most thermo-plastic appliances. The all-aluminum construction eliminates rusting and the heating element is symmetrically engineered for uniform heat distribution. The hexagonal post keeps the frame from rotating or sliding. This vacuum former uses 5" x 5" (127 x 127 mm) resin sheets.



#9594470110 Volt
#9594480220 Volt

Proline Air Vac 2000

Don't let the size of this machine misguide you. The powerful pressure unit provides superior adaptation for dental plastics using laboratory or operatory internal air compressors. Because the unit has no motor, it is compact and very quiet. Also, the vacuum pressure from an office compressor is usually greater than the pressure generated from smaller vacuum units. The greater the pressure, the better definition and adherence the dental appliances will have. The Air vac uses 5" x 5" (127 x 127 mm) resin sheets or round resin sheets.



#7000470110 Volt
#7000471220 Volt

Pro-form Comfort

The Pro-form Comfort is manufactured from high quality materials for a light and compact feel. The energy efficient machine has a vacuum that's powerful and quiet to help speed up the thermoforming process while delivering utmost precision.

The machine raises and lowers the thermo-forming material by an electric motor, eliminated error and maximizing accuracy. Compatible with round and square vacuum forming materials.

International Sales Only.



#9617502220 Volt

Pro-form Deluxe

The Pro-form Deluxe is manufactured from high quality materials for a light and compact feel. The energy efficient machine has a vacuum that's powerful and quiet to help speed up the thermoforming process while delivering utmost precision.

The programmable machine raises and lowers the material by an electric motor, eliminating error and maximizing accuracy. Pro-form Deluxe's program ability also ensures that each thermoforming sheet is heated to its precise temperature. Compatible with round and square vacuum forming materials.

International Sales Only.



#9617504220 Volt

Pro-form Silent

The Pro-form Silent is the most efficient machine in the Pro-form series. With the same pump as the other machines, the Pro-form Silent manually lowers the thermoplastic laminates. The machine is still made of the same high-quality materials as the Comfort and Deluxe models, such as the sturdy base and the same powerful vacuum. The Pro-form Silent is a basic machine that will get you started in making high quality Pro-form products. Compatible with round and square vacuum forming materials.

International Sales Only.



#9617500220 Volt

VACUUM FORMING ACCESSORIES

Pressure Dome

This new attachment works in conjunction with Keystone Industries' vacuum formers to create a better fitting dental appliance. The dome uses positive pressure during vacuuming allowing for a form-fitting mold of thermoplastics. The Pressure Dome hooks into any existing air compressor lines (needs 80 PSI line pressure with 3/8 line) to provide extra pressure when adapting thermoplastics over an existing model. It will fit over any vacuum unit that makes 5" x 5" (127 x 127 mm) plastics. Works great for making laminated mouthguards, heavy-duty splints, hard/soft dual laminated splints, Proflex base plates, model duplication, orthodontic retainers and more.



#9594050Pressure Dome

Pro-form centri-fuse

The demand of athletic mouthguard customization has grown significantly during the past few years. It's not uncommon to see professional athletes showing off their custom mouthguards with personal numbers, team logos or phrases on it. That's why Keystone Industries is thrilled to launch the centri-fuse kit, an innovated product that makes the application of decals to mouthguards quicker, easier and more cost effective.

The typical lamination process for customize mouthguards can become expensive and time consuming, however with centri-fuse, customization has never been easier or more affordable. The centri-fuse system gives the advantage of applying the decal while eliminating the necessity of a second forming.

The adhesive material is applied by using a flameless torch to create superb clarity and excellent adhesion while eliminating bubbles. The material is user friendly and adjustments are made in a matter of a few minutes with conventional tools.

The centri-fuse kit includes everything needed to start up an easy mouthguard customization system.

Kit includes: 20 centri-fuse strips, Flameless Torch, Primer, Brush, Scissors, Trimming and Polishing Brush, Pre-printed Decals.

With this kit, you're able to meet the growing demand of custom mouthguards. The centri-fuse system will let you print anything you want and fuse it to a vacuum formed mouthguard, making it truly a custom mouthguard.



#1009100centri-fuse Kit

#1009101centri-fuse Replacement Sheets

Pro-form Electric Knife

This is a great little electric knife useful in trimming and cutting a variety of thermoplastic dental materials. The unit has solid state circuitry, an interchangeable handpiece with up to four other replacement handpieces available, 2 outlet stations to accommodate more than one person, all stainless steel construction to resist rusting, and 10 amps of power.

The 10 amps allow you to cut most materials easily and smoothly by providing extra heat at the tip. Excellent for bleaching trays, impression trays, splints, mouthguards and more.



#9144730Electric Knife

Microtorch

This Microtorch is a 2-in-1 open flame torch and flameless heat tool with a heat shrink attachment. The torch is excellent use to heat materials, ignite materials, pinpoint torch, shrink tubing, solder and desolder, terminate specialty connectors and more.



#1820005Microtorch

Microtorch

A hand held, self-igniting, butane powered microtorch creates a precise flame emitting from a nozzle head, allowing direct heat application to an object.



#9139680Microtorch

Insta Cool

Use Insta Cool to immediately chill dental appliances for better adaptation and faster production. Using Insta Cool assures fast, excellent fitting results.



#7000333Insta Cool, 10 oz. can (283 g)

PRO-FORM/VACUUM FORMING

Mold Release Spray

This Mold Release Spray is an extremely versatile release agent and lubricant. The silicone spray produces a dry anti-stick surface coating, which is particularly suited as a mold spray prior to vacuum forming mouthguard or other thermoplastic materials.



#5940310Mold Release Spray, 11 oz. can (311 g)

Block Out Compound

This Block Out Compound is used to block out areas of models or can be rolled around a model base for easy periphery trimming. The reusable material will not melt under heat or stick to models, plastics or vacuum formers.



#1570210Block Out Compound, Black, 1 lb.

Z-Bur

The special hollowed, fluted design of Keystone's Z-Bur Diamond Bur is perfect for easy removal of sticky tissue conditioners and pastes commonly used to take denture impressions. The bur also works well in removing most soft liners. Messy removal of materials is eliminated with the Z-Bur action set at a medium speed (15,000-25,000 rpm). The slower cutting speed reduces burning and melting of the liner. The hollowed design also reduces the clogging of carbide and minimizes flying debris. Once the liner is removed the final preparation of the hard surface can be performed with carbide of choice.



#1201995Z-Bur

Utility Scissors

These All-Purpose Utility Scissors are great for bulk trimming of thermoplastics, especially Pro-form mouthguards. The easy grip handle and stainless steel construction is very handy throughout the lab or office.



#9139720Utility Scissors

Hatho Miniature Handpiece Scotch Brite™ Brushes

Mounted on mandrel. Same application as Polishing Discs with Scotch Brite™. Available in packs of 12.



#1670016Coarse, 5,000 rpm
 #1670092Medium, 5,000 rpm
 #1670091Fine, 5,000 rpm

Heavy Duty Strap Boxes

This is an ideal box for a mouthguard with an attached helmet strap. The cases are made of high-density polypropylene with a hinged lid. A slit on the front of the box allows for the mouthguard strap to come out of the box. We provide personal imprinting for this product as well.



#9575615Black, 12/Pkg
 #9575635Gray, 12/Pkg

High Gloss Ortho Boxes

High Gloss Ortho Boxes are perfect for storing mouthguards, retainers, bleaching trays and splints. Made of flexible plastic, the box has a hinge design and secure latching system to ensure maximum protection. The box is 3/4-inch deep. Available in packages of 12. Imprinting available, see page 7.

#9575185Assorted Marble
 #9575128Marble Blue/White
 #9575129Marble Pink/White
 #9575127Marble Black/White
 #9575115Light Blue
 #9575135Dark Blue
 #9575155White
 #9575165Yellow
 #9575125Green
 #9575190Assorted
 #9574460Beige
 #9575105Red

New Age Colors

#9575191Lime
 #9575195Pink
 #9575192Orange
 #9575196Purple
 #9575193Aqua
 #9575194Lavender
 #9575197Assorted



PAYMENT OPTIONS:

Open accounts may be established following approval of a completed Credit Application.

Open account invoices are payable 30 days from date of invoice. Past due accounts are subject to finance charge of 1-1/2% per month. Payment may be made by check or C.O.D. Prepayment accounts and accounts awaiting open account status may pay using Visa, Mastercard and American Express. All orders are payable in U.S. dollars. All freight charges are subject to a \$25 fee.

ORDERING:

Phone Orders: Monday - Friday, 8:30 AM to 5:30 PM, Eastern Standard Time.

Placing Orders via Fax: 856-663-0381

Minimum Order: \$100.00 for Domestic and \$250.00 for International. Orders under minimum amount are subject to a \$10.00 handling charge.

Drop Ship Orders: Are subject to a \$5.00 fee.

DELIVERY:

Keystone delivers domestic orders via insured ground service carrier. Next Day Air, Second Day Air, and Third Day Select Delivery are available at an additional charge upon request. Hazardous materials are subject to the carrier's handling charge and some items may not be shipped by air.

NOTE: For orders outside the continental U.S., call for shipping rates. Claims for damages or shortages must be made within 7 days of receipt. Damaged shipping cartons must be noted as such when signed for. All carton counts must be checked and signed for by the receiver in order for any discrepancies to be reconciled with the shipper.

RETURN POLICY:

All returns require prior return authorization. Contact our customer service department or your sales representative for return authorization.

The following conditions must be complied with:

- All products must be returned complete with all accessories and instruction sheets, if applicable, in original packaging and in saleable condition.
- A copy of the invoice and reason for the return must accompany all returns.
- Returned products not purchased within the previous sixty (60) days are subject to a minimum 15% restocking charge.

RETURN POLICY cont'd

- No merchandise or equipment may be returned for credit after 120 days of purchase.
- No merchandise or equipment may be returned for credit, regardless of its purchase date, if it is within fifteen (15) months of its expiration date.
- Used handpieces and equipment may not be returned for credit. Please verify that you have received exactly what you ordered before you use it.
- Defective merchandise will be repaired or replaced in accordance with manufacturer warranties.
- Returns must be sent by a traceable carrier (e.g. UPS).
- Return freight costs are the responsibility of the customer unless the return is due to a Keystone error.
- Returns not following the above policies may be refused or subject to a restocking charge.

CANADIAN RETURN POLICY:

In the event Keystone makes an error filling your order, please contact our customer service department or your sales representative immediately upon receipt of your order to request a pick up for the incorrect items. Keystone will contact our Canadian shipper, UPS to request a pick up and UPS will coordinate the item's return with our Credit>Returns Department. We will issue all credits upon receipt of the parcel according to the return freight charges that result from company errors, so if it is determined that it was not our error, a fee for the return freight will be deducted from the credit amount due the customer.

Items not returnable:

- Hazardous items
- Items that cannot be returned to the vendor (including expired items)
- Special Order Items: Products that we do not ordinarily stock are not returnable.

ALL RETURNS SHOULD BE SENT TO:

KEYSTONE INDUSTRIES
52 West King Street
Myerstown, PA 17067

INDEX

20 Minute Monomer.....	28
30B Blowpipe.....	35

A

Abbot Robinson Brushes.....	60
Abrasive Rolls.....	23

Abrasives..... 16-25

Arbor Band Mandrels.....	23
Arbor Bands.....	23
Black Utility Grinding Wheel.....	20
Blue Mounted Points.....	16
Brown High Speed Disc.....	22
Carbo Model Trimmer Wheel.....	24
Clasp Polisher.....	18
Cut-Off Wheels.....	21
Deluxe Silicone Points & Wheels.....	18
Dura-Thin Disc.....	21
Econo Cutter Mandrel.....	24
Econo Cutters.....	23
Field Abrasive Rolls.....	23
Field Roll Mandrels.....	23
Flexible Diamond Discs.....	25
Green Engine Wheels.....	19
Green Mounted Stones.....	16
Grinding Wheel Dresser.....	19
Grinding Wheels.....	20
Heavy Bonded Arbor Band.....	23
Interproximal Rubber Wheels.....	18
Joe Dandy Type Discs.....	22
Key-flex Discs.....	25
Keystone Cut-off Wheels.....	22
Knife Edge Pumice Wheels.....	19
Knife Edge Rubber Wheels.....	18
Mizzy Heatless Wheels.....	24
Pink Mounted Stones.....	16
Point & Wheel Truer.....	19
Porcelain Finishing Discs.....	22
Pure Blast White Aluminum Oxide.....	20
Red Engine Wheels.....	19
Red Flash Disc.....	21
Red Flash Finishing Wheels.....	19
Red Heatless Wheels.....	24
Red Mounted Stones.....	16
Roughing Disc.....	19,22
Rubber Acrylic & Baseplate Wheel.....	20
Superior Rubber Clasp Polishers.....	17
Superior Rubber Wheels.....	17

Ti-Coated Diamond Discs.....	25
Ultra-Flex Disc.....	21
Ultra-Thin Disc.....	21
Veri-Thin Disc.....	21
Waterproof Model Trimmer Discs.....	24
White Acrylic Knock-Down Wheels.....	20
White Mounted Stones.....	16
Acculab Model V-400 Scale.....	47
Acid Neutralizer.....	52

Acrylics..... 26-30

All Purpose Duplicating Material.....	29
Dappen Dishes.....	30
Diamond D Acrylic.....	26
Diamond D Ultra Wash.....	26
Hatho Thermoplastic Polishing Kit.....	29
Itsoclear.....	30
Millennium Acrylic.....	27
Millennium Pour Acrylic.....	28
Pro-flex NFC Clear.....	29
Pro-flex NFC Resin.....	29
ProCure Custom Tray Wafers.....	29
ProCure Hard Reline.....	29
ProCure Soft Reline.....	29
Reline Bond.....	30
Sledgehammer 20 Minute Monomer.....	28
Sledgehammer Heat Cure Acrylic.....	26,27
Sledgehammer Self Cure Acrylic.....	27
Versacryl Thermo-soft Adjustable Reline.....	28
Adhesive-Coated Wax.....	64
Adhesives.....	45
Adjustable Saw Frame.....	43, 49
AFG Aesthetic Translucent Wax.....	64
Air Gun.....	46
Air Vac 2000.....	88
Air/Water Syringe Tips.....	13
Akers Carver.....	50
Alcohol Torch.....	35
Alert Pouch Bag.....	6
All Purpose Duplicating Material.....	29
Aluminum Oxide.....	20
Anesthetic Gel & Spray.....	82
Apex #2 Articulators.....	30
APF Fluorides.....	80
Arbor Band Chucks.....	63
Arbor Band Mandrels.....	23
Arbor Bands.....	23
Articulating..... 74	
Articulating Papers.....	74
Mizzy PIP Paste.....	74

Mizzy PIP Spray 74

Articulating Papers 74

Articulator Boxes 6

Articulators 30-32

 Apex #2 Bronze Articulator 30

 Apex #2 Chrome Articulator 30

 Crown & Bridge Articulator 30

 Deluxe Magnetic Articulator with Pin..... 30

 Foster Articulators 31

 Galetti Articulator..... 31

 High Arch Articulator..... 31

 K-Brass Denture Articulator..... 31

 K-Chrome Denture Articulator 31

 Keystone Deluxe Articulator..... 32

 Lab-O-Mate Articulator 32

 Luxury Articulator 32

 Magnet X..... 32

 Plasterless Articulator..... 32

 Plastic C&B Disposable Articulator 32

 Precision Articulator..... 32

 Spring Chrome Articulator..... 32

B

Baby Muslin Buffs..... 34

Bag Sealer..... 42

Ball Clasps 40

Bar Bending Pliers..... 50

Bard-Parker Blades..... 49

Barrier Film..... 10

Barrier Products 10-11

 Barrier Film..... 10

 Chair Sleeve Covers 10

 Digital X-Ray Sensor Sleeves..... 10

 Handpiece Sleeves 10

 Headrest Covers..... 10

 Light Handle Sleeves 10

 Syringe Sleeve Covers..... 10

 Tray Sleeves..... 10

 X-Ray Sleeves 11

Base Plate Material 38,86

Base Plate Wax 64

Beakers 48

Benzocaine Anesthetic Spray..... 82

Bibs 13,14

Biohazardous Pouch Bag..... 6

Bite Blocks..... 65

Bite Registration..... 76

Black Utility Grinding Wheel 20

Blade Handles..... 49

Blades..... 49,78

Bleaching Laminates 85

Block Out Compound..... 90

Block Out Material..... 44

Block Out Wax..... 64,65

Blowpipe..... 35,36

Blue Safe Acid Neutralizer 52

Bobbing Compound 53

Boil Out Agent..... 72

Boley Gauge..... 43,51

Boxing Strip Wax..... 65

Brown High Speed Disc..... 22

Brushes & Buffs 33-35

 Baby Muslin Buffs..... 34

 Chamois Buffs 35

 Diamond D Sep Brushes 33

 Felt Cylinders..... 34

 Felt Wheels 34

 Flannel Buffs..... 35

 Keystone Kolinsky Ceramist Brushes 33

 Kolinsky Brushes 33

 Laboratory Hand Brush..... 34

 Lead Center Buffs 34

 Leather Center Buffs..... 34

 Mounted Felt Cones Kit 34

 Muslin Buffs 35

 Red Sable Brushes 33

 Yeti Dia Quick..... 35

 Buffs..... 34,35

Bunsen Burner 35

Bur Chucks 63

Burners & Torches 35-36

 30B Blowpipe 35

 Bunsen Burner 35

 Casting Torch Accessories..... 36

 Deluxe Bunsen Burner 35

 Hanau Alcohol Torch..... 35

 Harris Automatic Casting Torch 36

 Harris Manual Torch 36

 Microtorch..... 36

 National Blowpipe..... 36

 Plastic Alcohol Torch 35

Burs..... 61-62

INDEX

C

CAD/CAM Discs	65
CAD/CAM Scanning Wax	65
CAD/CAM Sculpturing Wax.....	65
Calico Interleaved Lath Brush	60
Calipers.....	43,51
Carbide Burs.....	61,62
Carbide Twist Drills	63
Carbo Model Trimmer Wheel	24
Carbon Insert Ceramic Tray	57
Carvers.....	50
Casting.....	36-38
Casting Ring Liner	37
Casting Rings.....	36
Centrifugal Type Crucibles	37
Crucible Formers.....	37
Hi-Heat Split Casting Rings	36
High Heat Crucibles.....	38
Hooded Crucibles.....	37
Induction Crucibles.....	37
Slotted Crucibles.....	37,38
Wesgo Crucibles	38
Casting Ring Liner	37
Casting Rings	36
Casting Torch	36
Casting Torch Accessories	36
Casting Wax.....	65
Cements & Liners.....	74
Fleck's Zinc Phosphate Cement.....	74
Mixing Poly Pads.....	74
Orange Solvent with Lanolin.....	74
centri-fuse	89
Ceramic Sculpting Set.....	51
Ceramic Tray.....	57
Cervical Wax.....	65
Chair Sleeve Covers	10
Chamois Buffs.....	35
Chisels.....	49
Claps Bending Pliers.....	50
Clasp Kit	40
Clasp Polisher.....	18
Clasp Wire	40
College Type Pliers.....	50
Composite Polish.....	53
Consistency Bite Registration.....	76
Consistency VPS.....	76
Convolute Fillers.....	6
Coping Material	86
Corning Inlay Wax.....	66

Cosmetic Dentistry..... 75

Niu Nate Take-Home Whitening Kit	75
Niu Frost Whitening Pen	75
Cotton Rolls	11
Cotton Tipped Applicators	11
Crown & Bridge Articulator.....	30
Crown & Bridge Material.....	86
Crown & Bridge Scissors	50
Crown & Bridge Wax.....	66
Crown Holders.....	50
Crucible Formers	37
Crucibles	37,38
Curing Unit	45
Cut-Off Wheels.....	21,22

D

Dappen Dishes	30
Debubblers.....	38
Klean Cast Debubbler.....	38
Mizzy Blue Debubbler	38
Sure-Flo.....	38
Yeti Wax Debubbler.....	38
Deluxe Articulator	32
Deluxe Bunsen Burner	35
Deluxe Magnetic Articulator	30
Deluxe Magnetic Articulator with Pin	30
Deluxe Silicone Points & Wheels	18
Dental Dams.....	75
DENTAL PACKAGING	5-8
Dental Packaging Accessories	6
Alert Pouch Bag	6
Biohazardous Pouch Bag.....	6
Convolute Fillers.....	6
Mailer Labels	6
Occlusal Surface Protection.....	6
Denture Accessories.....	38-42
3 Post Reline Jig	39
Clasp Kit	40
Denture Flask	40
Diamond D Sep Brushes	39
Diamond D Ultra Sep.....	38
Diamond D Wax Gloss.....	39
Duplicating Flasks.....	40
Flask Compress.....	40
Frego Strengtheners.....	42
Grid Strengtheners.....	42
Heat Sealing Bags.....	42

Impulse Bag Sealer.....	42	Yeti Dia Glaze	44
Kefoil	39	Yeti Die Bloc.....	44
Lingual Bars	41	Yeti Die Hardener	44
Palatinal Brush	39	Yeti Die Spacer.....	43
Plastic Sheets.....	39	Yeti Light Spacer	44
ProCure Base Plate Material	38	Yeti Sakura Pencil.....	44
Relief Metal.....	39	Yeti Steam Off Spacer.....	44
Reline Jig.....	39	Die Bloc.....	44
Stainless Steel Ball Clasps.....	40	Die Hardener.....	44
Stainless Steel Clasp Wire	40	Die Lube.....	44
Stainless Steel Wire Clasps.....	40,41	Die Spacer	43
Tray Handles.....	40	Digital X-Ray Sensor Sleeves.....	10
Wire Mesh	42	Dipping Wax.....	66
Denture Curing Unit.....	45	Dispensing Gun.....	76
Denture Flask	40	Dispos-A-Trap.....	46
Denture Storage Boxes.....	8	Disposable Impressions Trays.....	77
Dial Caliper	43, 51	Disposables	11
Diamond D Acrylic.....	26	Cotton Rolls	11
Diamond D Sep Brushes	33,39	Cotton Tipped Applicators.....	11
Diamond D Ultra Sep.....	38	Non-Woven Sponges.....	11
Diamond D Ultra Wash	26	Plastic Cups.....	11
Diamond D Wax Gloss.....	39	DISPOSABLES/INFECTION CONTROL.....	9-14
Diamond Discs.....	25	Double Dowel Pins.....	42
Die & Model Fabrication.....	42-45	Double Sided Caliper.....	43,51
Adjustment Saw Frame	43	Dowel Pins	42
Boley Gauge	43	Dual Laminates/E-Gasket	86
Dial Calipers.....	43	Duplicating Flasks	40
Die Lube.....	44	Duplicating Material.....	29
Die Spacer	43	Dura-Thin Disc.....	21
Double Dowel Pins.....	42		
Double Sided Caliper	43	E	
Iwansson Calipers.....	43	E-Gasket.....	86
J-Pin.....	42	Ear Loop Masks	12
Keybond.....	45	Econo Cutter Mandrel	24
NewArea Die Spacer.....	43	Econo Cutters.....	23
NewArea Special Liquids	44	Electric Knife.....	89
Offset Pin Saw Frame	43	Engine Belts	48
Permabond 910.....	45	Engine Wheels.....	19
Pin Saw Frame	43	Epoxy.....	45
Plug-It.....	45	Equipment.....	45-48
ProCure Block Out Material.....	44	Acculab Model V-400 Scale	47
Reverse Dowel Pins	42	Air Gun	46
Saw Blades	43	Denture Curing Unit.....	45
Slaycris Slik Die Lubricant.....	44	Dispos-A-Trap.....	46
Stick Dowel Pins.....	42	Engine Belts	48
Swiss Cut Offset Saw Frame.....	43	Harris Oxygen Regulator	46
Tri-Dynamics Tri-Epoxy.....	45	Heat Regulated Pressure Pot.....	45
Tunnelmate	42		
Yeti Blue Marker.....	44		

INDEX

Hydro-air Carver.....	47
Keystone Three-Well Wax Pot.....	47
Mini Scale.....	47
Modeling Tips for Yeti Waxers.....	48
Optimist Ceramic Spray System.....	47
Plastic Beaker.....	48
Plastic Liquid Dispensers.....	48
Pro-form Vibrator.....	46
Pro-form Pressure Pail.....	45
ProCure 300 Light Oven.....	46
The Trap.....	46
Ultra Sonic 2300 General Purpose Cleaner.....	46
Ultra Sonic 2400 Plaster & Stone Remover.....	46
Yeti Dippy NT Light.....	47
Yeti Dippy Pro Waxer Digital.....	48
Yeti Pro Waxer Duo.....	48
Etchant Gels.....	75
Seity Etchant Gel.....	75
Zinc Oxide Powder USP.....	75
Eugenol USP.....	78
Evacuation Products.....	11-12
High Volume Evacuator Tips.....	11
Saliva Ejectors.....	12
Scented High Volume Evacuator Tips.....	11
Surgical Aspirator Tips.....	12
Extra-Pro Laminates.....	85
Eyewear.....	14,51

F

Face Masks.....	51
Face Masks & Shields.....	12-13, 51
Ear Loop Masks.....	12
HB Anti-Fog Face Shield.....	13
Optima Procedural Face Mask.....	12
Procedural Ear Loop with Visor Masks.....	12
Surgical Molded Cone Masks.....	12
Surgical Tie-On Masks.....	12
Face Shield.....	13
Felt Cones.....	34
Felt Cylinders.....	34
Felt Wheels.....	34
Field Abrasive Rolls.....	23
Field Roll Mandrels.....	23
Flannel Buffs.....	35
Flask Compress.....	40
Flasks.....	40

Fleck's Zinc Phosphate Cement.....	74
Flexible C&B Boxes.....	7
Flexible Diamond Discs.....	25
Flexis Diamond Discs.....	60
Fluoride Trays.....	81
Fluorides.....	80-81
Gelato APF Fluoride Gel.....	80
Gelato APF Foam Fluoride.....	80
Gelato Home Care Rinse.....	80
Gelato Neutral PH Fluoride Gel.....	80
Hinged EZ Fluoride Trays.....	81
Mizzy Prevent Trays.....	81
Foam Fillers.....	7,8,87
Fortis Disposable Prophylaxis Angles.....	82
Foster Articulators.....	31
Frego Strengtheners.....	42
Fun Mouthguards.....	84
Furnace Tongs.....	54

G

Galetti Articulator.....	31
Gauges.....	43,51
Gelato APF Fluoride Gel.....	80
Gelato APF Foam Fluoride.....	80
Gelato Benzocaine Anesthetic Spray.....	82
Gelato Home Care Rinse.....	80
Gelato Neutral PH Fluoride Gel.....	80
Gelato Plus Anesthetic Gel.....	82
Gelato Primal Prophylaxis Cups.....	82
Gelato Prophylaxis Paste.....	81
Gelato Topical Anesthetic Gel.....	82
General Purpose Cleaner.....	46
Glitter Guards.....	84
Goggles.....	51
Green Plate Polish.....	53
Grid Strengtheners.....	42
Grifold Knife.....	49
Grinding Wheel Dresser.....	19
Grinding Wheels.....	20

H

H-Base Pegs.....	57
H.P. Sintered Diamond Kit.....	61
H.P. Sintered Diamonds.....	61
Hanau Alcohol Torch.....	35

Handpiece Sleeves	10
Handy Knife.....	49
Hard Reline	29
Harris Automatic Casting Torch	36
Harris Manual Torch	36
Harris Oxygen Regulator	46
Hatho Acrypol Polishing Kit	54
Hatho Geza Brushes	58
Hatho Habras Discs	58
Hatho Habras Miniature Brushes	58
Hatho Miniature Brush with Polishing Disc	59
Hatho Miniature Handpiece Scotch Brite Brushes	59,90
Hatho Miniature Polishing Brush	59
Hatho Polishing Compounds.....	53
Hatho Polishing Discs	59
Hatho Polistar Polishing Kit	54
Hatho Poly-Buff Chung King Lathe Brushes ..	59
Hatho Special Lathe Brushes.....	59
Hatho Thermoplastic Polishing Kit	29,54
Hatho Toro Miniature Brushes.....	59
HB Anti-Fog Face Shield.....	13
HB Bite Registration	76
HB VPS.....	76
Head Mounted Loupe	52
Headband Magnifier.....	52
Headrest Covers.....	10
Heat Regulated Pressure Pot.....	45
Heat Sealing Bags	42
Heat-Sink Sagger Trays.....	57
Heatless Wheels.....	24
Heavy Bonded Arbor Band.....	23
Heavy Duty Strap Boxes.....	8,90
Hemostatic Solution.....	78
Hemostats	50
Hemox-A	78
Hi-Heat Split Casting Rings	36
High Arch Articulator.....	31
High Gloss Ortho Boxes.....	8
High Volume Evacuator Tips.....	11
Hinged EZ Fluoride Trays.....	81
Hollow Sprues.....	71
Home Care Rinses	80
Honey-Comb Pegs.....	55
Honey-Comb Trays.....	56
Hydro-air Carver.....	47

I	
Impression Compound.....	67
Impressions Material	76-77
Consistency Bite Registration.....	76
Consistency VPS.....	76
Dispensing Gun.....	76
Disposable Impressions Trays.....	77
HB Bite Registration.....	76
HB VPS.....	76
Mixing Tips.....	76
Mizzy Impression Compound Cakes & Sticks	77
Rubber Mixing Bowls	76
Impressions Trays.....	77
Impulse Bag Sealer	42
Inlay Wax	66,67
Insta Cool	89
Insta-Frac.....	54
Instruments.....	49-51
Adjustable Saw Frame	49
Akers Carver	50
Bar Bending Pliers	50
Bard-Parker Blades.....	49
Blade Handles.....	49
Boley Gauge	51
Carvers.....	50
Ceramic Sculpting Set.....	51
Claps Bending Pliers.....	50
College Type Pliers.....	50
Crown & Bridge Scissors	50
Dial Caliper	51
Double Sided Calipers	51
Grifold Knife	49
Handy Knife	49
Hemostats	50
Iwansson Calipers.....	51
Magna Bar.....	51
Plaster Knives	49
Plaster Nippers	49
Scrapers.....	49
Spatulas.....	50
Straight Handle Chisel	49
Swiss Cut Offset Saw Frame.....	49
Tweezer Type Crown Holder	50
Utility Scissors	50
Wheel Saw.....	49
Interproximal Rubber Wheels.....	18
Isolating Pen.....	72
Itsoclear.....	30

INDEX

Iwansson Calipers 43,51

J

J-Pin 42

Joe Dandy Type Discs 22

K

K-Brass Denture Articulator 31

Ke-Solv 72

Kefoil 39

Kewax Wire Wax 70

Key-flex Discs 25

Keybond 45

Keygold 62

Keystone Cut-off Wheels 22

Keystone Deluxe Articulator 32

Keystone Inlay Wax 67

Keystone Kolinsky Ceramist Brushes 33

Keystone Spiral Cut Carbide Burs 61,62

Keystone Three-Well Wax Pot 47

Klean All Boil Out Agent 72

Klean Cast Debubbler 38

Knife Edge Pumice Wheels 19

Knife Edge Rubber Wheels 18

Knives 49

Knock-Down Wheels 20

Kolinsky Brushes 33

L

Lab Safety 51-52

 Eyewear 51

 Face Masks 51

 Goggles 51

 Overgloves 52

 Respirators 51

Lab Work Pans 8

Lab-O-Mate Articulator 32

LABORATORY 15-72

Laboratory Hand Brush 34

Laboratory Pumice Powder 54

Lamp/Magnifiers 52

 Head Mounted Loupe 52

 Headband Magnifier 52

 Megaview 52

Lathe Brushes 59,60

Lead Center Buffs 34

Leather Center Buffs 34

Light Handle Sleeves 10

Light Oven 46

Lingual Bars 41

Liquid Dispensers 48

Loupe 52

Luxury Articulator 32

M

Machine III 88

Machine IV 87

Magna Bar 51

Magnet X 32

Magnifiers 52

Mailer Labels 6

Mandrels 23,24,63

Matrix Materials 77

 Matrix Retainers, Rolls and Bands 77

 Mylar Strips 77

Matrix Retainers, Rolls and Bands 77

Maves Inlay Wax 67

Medicament Materials 78

 Eugenol USP 78

 Hemox-A 78

Megaview 52

Metal Pins 55

Micro Applicators 13

Microtorch 36,89

Millennium Acrylic 27

Millennium Pour Acrylic 28

Milling Wax 67

Mini Scale 47

Mixing Bowls 76

Mixing Pallet 56

Mixing Poly Pads 74

Mixing Tips 76

Mizzy Allcezon Base Plate Wax 64

Mizzy Blue Debubbler 38

Mizzy Byte Ryte Wax 65

Mizzy Heatless Wheels 24

Mizzy Impression Compound Cakes & Sticks

..... 67,77

Mizzy Mint Flavored Diamond Polish 54

Mizzy PIP Paste 74

Mizzy PIP Spray 74
 Mizzy Prevent Trays 81
 Model Storage Boxes 6
 Model Trimmer Wheels 24
 Modeling tips for Yeti Waxers 48
 Mold Release Spray 90
 Mounted Felt Cones Kit 34
 Mounted Stones 16
 Mounted Wire Brushes 60
 Mouthguard Laminates 84
 Mouthguard Straps 85
 Mouthguards Resin Sheets 85
 Muslin Buffs 34,35
 Mylar Strips 77

N

National Blowpipe 36
 Needleless Injector 78
 Needles 13
 Neutral PH Fluoride Gel 80
 NewArea Die Spacer 43
 NewArea Special Liquids 44
 NewCera Block Out Wax 64
 NewCera CAD/CAM Discs 65
 NewCera CAD/CAM Sculpturing Wax 65
 NewCera Cervical Wax 65
 NewCera Dipping Wax 66
 NewCera Iso Pen 72
 NewCera Milling Wax 67
 NewCera ROCK Sculpturing Wax 68
 NewCera Sculpturing Wax 68
 NewCera Sticky Wax 70
 Nippers 49
 Niteguards 86
 Nitric Acid 52
 Niu Frost Whitening Pen 75
 Niu Nate Take-Home Whitening Kit 75
 Non-Woven Sponges 11

O

Occlusal Spray 72
 Occlusal Surface Protection 6
 Offset Pin Saw Frame 43
OPERATORY 73-78
 Optimist Ceramic Spray System 47

ORAL HEALTH 79-82

Orange Solvent with Lanolin 74
 Ortho Boxes 8, 90
 Overgloves 14,52
 Oxygen Regulator 46

P

P.D. Attachments 53
 Palatinal Brush 39
 Pattern Adhesive 52
 Pegs 55,57
 Perio Maintenance 80
 Permabond 910 45
 Pin Saw Frame 43
 Pins 55
 PIP Paste 74
 PIP Spray 74
 Plaster & Stone Remover 46
 Plaster Knives 49
 Plaster Nippers 49
 Plaster Trap 46
 Plasterless Articulator 32
 Plastic Alcohol Torch 35
 Plastic C&B Disposable Articulator 32
 Plastic Cups 11
 Plastic Mailer Boxes 6,7
 Plastic Mailer Replacement Parts 7
 Plastic Sheets 39
 Pliers 50
 Plug-It 45
 Point & Wheel Truer 19

Polishing 53-54

Hatho Acrypol Polishing Kit 54
 Hatho Polishing Compounds 53
 Hatho Polistar Polishing Kit 54
 Hatho Thermoplastic Polishing Kit 54
 Laboratory Pumice Powder 54
 Mizzy Mint Flavored Diamond Polish 54
 Polishing Compounds 53
 Porte Polishers 54
 Polishing Compounds 53
 Polishing Kits 29,54

Porcelain Accessories 54-57

Carbon Insert Ceramic Tray 57
 Ceramic Tray 57
 Furnace Tongs 54
 H-Base Pegs 57

INDEX

Heat-Sink Sagger Trays.....	57
Honey-Comb Pegs.....	55
Honey-Comb Trays.....	56
Insta-Frac.....	54
Metal Pins.....	55
Mixing Pallet.....	56
Porcelain Stacking Trays.....	55
Porcelain Tongs.....	55
Roundhouse Trays.....	57
Slant Tile.....	56
Soft Pillows.....	57
Spider Tray.....	57
Staining Pallet.....	56
Uni-Pegs.....	55
Universal Pegs.....	55
V Pegs.....	55
Wire Pins.....	55
Porcelain Finishing Discs.....	22
Porcelain Stacking Trays.....	55
Porcelain Tongs.....	55
Porte Polishers.....	54
Power Wash.....	46
Pre-Bent Dispensing Tips.....	13
Precision Articulator.....	32
Pressure Dome.....	89
Pressure Pail.....	45
Pressure Pot.....	45
Preventatives.....	81
Tongue Scrapers.....	81
Pro-Flex & Dual Combo.....	85
Pro-flex NFC Clear.....	29
Pro-flex NFC Resin.....	29
Pro-form Base Plate Material.....	86
Pro-form centri-fuse.....	89
Pro-form Comfort.....	88
Pro-form Deluxe.....	88
Pro-form Dual Laminates/E-Gasket.....	86
Pro-form Electric Knife.....	89
Pro-form Fun Mouthguards.....	84
Pro-form Glitter Guards.....	84
Pro-form Machine IV.....	87
Pro-form Mouthguard Laminates.....	84
Pro-form Niteguards.....	86
Pro-form Pressure Pail.....	45
Pro-form Sample Pack.....	84
Pro-form Silent.....	88
Pro-form Tie-Dye Mouthguards.....	84
Pro-form Vibrator.....	46
PRO-FORM/VACUUM FORMING.....	83-90
Procedural Ear Loop with Visor Masks.....	12
Procedure Accessories.....	13
Air/Water Syringe Tips.....	13
Micro Applicators.....	13
Overgloves.....	14
Pre-Bent Dispensing Tips.....	13
Pro-Guard I Bibs.....	13
Pro-Guard II Bibs.....	14
Protective Eyewear.....	14
ProCure 300 Light Oven.....	46
ProCure Base Plate Material.....	38
ProCure Block Out Material.....	44
ProCure Custom Tray Wafers.....	29
ProCure Hard Reline.....	29
ProCure Soft Reline.....	29
Proline Air Vac 2000.....	88
Prophy Angles.....	82
Prophy Brushes.....	82
Prophy Products.....	81-82
Fortis Disposable Prophy Angles.....	82
Gelato Primal Prophylaxis Cups.....	82
Gelato Prophylaxis Paste.....	81
Prophylaxis Polishing Brushes.....	82
Prophylaxis Polishing Cups.....	82
Prophylaxis Polishing Brushes.....	82
Prophylaxis Polishing Cups.....	82
Protective Eyewear.....	14
Pumice.....	54
Pure Blast White Aluminum Oxide.....	20
R	
Red Flash Disc.....	21
Red Flash Finishing Wheels.....	19
Red Heatless Wheels.....	24
Red Rouge.....	53
Red Sable Brushes.....	33
Relief Metal.....	39,52
Reline Bond.....	30
Reline Jig.....	39
Resin Sheets.....	85
Respirators.....	51
Retainer Material.....	86
Reverse Dowel Pins.....	42
Rigid C&B Boxes.....	8
Rotary.....	58-63
Abbott Robinson Brushes.....	60
Arbor Band Chucks.....	63

Bur Chucks	63	Sep Brushes	33,39
Calico Interleaved Lathe Brush	60	Separator	38,39
Carbide Twist Drills	63	Set-Up Wax	69
Flexis Diamond Discs	60	Shipping Boxes	6-7
H.P. Sintered Diamond Kit	61	Articulator Boxes	6
H.P. Sintered Diamonds	61	Model Storage Boxes	6
Hatho Geza Brushes	58	Plastic Mailer Boxes	6,7
Hatho Habras Discs	58	Plastic Mailer Replacement Parts	7
Hatho Habras Miniature Brushes	58	Shipping Containers	7
Hatho Miniature Brush w/ Polishing Disc ..	59	Shipping Containers	7
Hatho Miniature Handpiece Scotch Brite		Shure Shine Plate Polish	53
Brushes	59	Single Chambered Vacuum Former	88
Hatho Miniature Polishing Brush	59	Slant Tile	56
Hatho Polishing Discs	59	Slaycris Crown & Bridge Wax	66
Hatho Poly-Buff Chung King Lathe Brushes		Slaycris Slik Die Lubricant	44
.....	59	Sledgehammer 20 Minute Monomer	28
Hatho Special Lathe Brushes	59	Sledgehammer Heat Cure Acrylic	26,27
Hatho Toro Miniature Brushes	59	Sledgehammer Self Cure Acrylic	27
Keygold	62	Soft EVA	87
Keystone Spiral Cut Carbide Burs	61,62	Soft Pillows	57
Lathe Brushes	60	Soft Reline	29
Mandrels	63	Spacer Foam	87
Mounted Wire Brushes	60	Spatulas	50
Stone Chucks	63	Special Formula Wax	69
Tapered Spindles	63	Spider Tray	57
Z-Bur	62	Splint Material	86
Roughing Disc	19,22	Spring Chrome Articulator	32
Roundhouse Trays	57	Sprue Wax	69
Rubber Acrylic & Baseplate Wheel	20	Sprues	71
Rubber Mixing Bowls	76	Staining Pallet	56
		Stainless Steel Ball Clasps	40
		Stainless Steel Clasp Wire	40
		Stainless Steel Wire Clasps	40,41
		Stannous Fluoride	80
		Sterilization Products	14
		Dual Sterilization Pouches	14
		Sterilization Pouches	14
		Stick Dowel Pins	42
		Sticky Wax	70
		Stippled Wax Sheets	70
		Stone Chucks	63
		Storage Boxes	7-8
		Denture Storage Boxes	8
		Flexible C&B Boxes	7
		Heavy Duty Strap Boxes	8
		High Gloss Ortho Boxes	8
		Lab Work Pans	8
		Rigid C&B Boxes	8
		Straight Handle Chisel	49

S

Safety Glasses	14,51
Sagger Trays	57
Sakura Pencil	44
Saliva Ejectors	12
Saw Blades	43
Saw Frames	43,49
Scales	47
Scalpels	78
Scented High Volume Evacuator Tips	11
Scissor Type Crown Holder	50
Scissors	50,90
Scotch Brite Brushes	59,90
Scrapers	49
Sculpturing Waxes	64,65,68,69
Seity Etchant Gel	75

INDEX

Strap Boxes	8, 90
Strengtheners.....	42
Strip Wax.....	65
Stripit.....	52
Superior Rubber Clasp Polishers.....	17
Superior Rubber Wheels.....	17
Sure-Flo.....	38
Surgical Aspirator Tips.....	12
Surgical Blades	78
Surgical Molded Cone Masks	12
Surgical Products	78
Surgical Blades.....	78
Surgical Scalpels.....	78
Surgical Scalpels.....	78
Surgical Tie-On Masks.....	12
Swiss Cut Offset Saw Frame.....	43,49
Swiss Dowel Pins.....	42
Synthetic Ceramist Brushes.....	33
Syrijet Needleless Injector.....	78
Syringe Sleeve Covers.....	10

T

Tapered Spindles.....	63
The Trap	46

Thermal Plastics

Base Plate Material	86
Bleaching Laminates.....	85
Coping Material	86
Crown & Bridge Material.....	86
Dual Laminates/E-Gasket	86
Extra-Pro Laminates.....	85
Mouthguard Straps.....	85
Mouthguards Resin Sheets	85
Niteguards	86
Pro-Flex & Dual Combo.....	85
Pro-form Fun Mouthguards.....	84
Pro-form Glitter Guards.....	84
Pro-form Mouthguard Laminates.....	84
Pro-form Sample Pack.....	84
Pro-form Tie-Dye Mouthguards.....	84
Retainer Material	86
Soft EVA	87
Spacer Foam.....	87
Splint Material	86
Tray Material	87
Youth Mouthguard	85
Ti-Coated Diamond Discs.....	25

Tie-Dye Mouthguards	84
Tin Foil.....	39
Tongs.....	54,55
Tongue Scrapers.....	81
Topical Anesthetics	82
Gelato Benzocaine Anesthetic Spray	82
Gelato Plus Anesthetic Gel.....	82
Gelato Topical Anesthetic Gel.....	82
Tray Handles.....	40
Tray Material	87
Tray Sleeves.....	10
Tri-Dynamics Tri-Epoxy.....	45
Tripoli	53
Truer	19
Tunnelmate.....	42
Tweezer Type Crown Holder	50

U

Ultra Sep.....	38
Ultra Sonic 2300 General Purpose Cleaner.....	46
Ultra Sonic 2400 Plaster & Stone Remover.....	46
Ultra Wash.....	26
Ultra-Flex Disc.....	21
Ultra-Thin Disc.....	21
Uni-Pegs.....	55
Universal Pegs.....	55
Utility Scissors	50,90
Utility Wax.....	70

V

V Pegs.....	55
-------------	----

Vacuum Formers.....

Machine III	88
Machine IV	87
Pro-form Comfort	88
Pro-form Deluxe	88
Pro-form Silent.....	88
Proline Air Vac 2000	88
Single Chambered Vacuum Former.....	88

Vacuum Forming Accessories

Block Out Compound.....	90
Hatho Miniature Handpiece Scotch Brite ...	
Brushes.....	90
Heavy Duty Strap Boxes.....	90
High Gloss Ortho Boxes.....	90

Insta Cool.....	89
Microtorch.....	89
Mold Release Spray	90
Pressure Dome	89
Pro-form centri-fuse.....	89
Pro-form Electric Knife	89
Utility Scissors	90
Z-Bur	90
Veri-Thin Disc.....	21
Versacryl Thermo-soft Adjustable Reline	28
Vibrator	46
VPS.....	76

W

Waterproof Model Trimmer Discs	24
Wax Brush	72
Wax Gloss	39
Wax Palate Patterns.....	71
Wax Pontics	71
Wax Pot.....	47,48
Wax Solvent.....	72
Waxers	48

Waxes 64-72

Adhesive-Coated Wax	64
AFG Aesthetic Translucent Waxes	64
Base Plate Wax	64
Boxing Strip Wax.....	65
Casting Wax.....	65
Cervical Wax.....	65
Crown & Bridge Wax.....	66
Dipping Wax.....	66
Impression Compound.....	67
Inlay Wax	66,67
Isolating Pen.....	72
Ke-Solv.....	72
Kewax Wire Wax	70
Milling Wax.....	67
Mizzy Allcezon Base Plate Wax	64
Mizzy Byte Ryte Wax.....	65
Mizzy Impression Compound	
Cakes & Sticks	67
NewCera Block Out Wax	64
NewCera CAD/CAM Discs.....	65
NewCera CAD/CAM Sculpturing Wax	65
NewCera Cervical Wax	65
NewCera Milling Wax	67
NewCera ROCK Cervical Wax	65

NewCera ROCK Sculpturing Wax	68
NewCera Sculpturing Waxes.....	68
NewCera Sticky Wax.....	70
Plastic Hollow Sprues.....	71
Sculpturing Waxes.....	68,69
Set-Up Wax	69
Special Formula Wax.....	69
Sprue Wax	69
Sticky Wax	70
Stippled Wax Sheets	70
Utility Wax.....	70
Wax Brush.....	72
Wax Palate Patterns	71
Yeti Block Out Wax.....	64
Yeti CAD/CAM Scanning Wax.....	65
Yeti Consequent Universal Wax.....	70
Yeti Creation Wax Set.....	66
Yeti Cutex Block Out Wax.....	65
Yeti Deton Wax Wire	70
Yeti Duo Dip Granular Dipping Wax	66
Yeti Elasto Dip Dipping Wax.....	66
Yeti Finale Wax Smoothing Liquid	72
Yeti Green Occlusal Spray.....	72
Yeti IQ Compact White Wax.....	67
Yeti IQ Pearl Wax.....	67
Yeti IQ Quadro Sticks.....	71
Yeti IQ Sculpturing Wax	69
Yeti IQ Triangle	71
Yeti Iso-Quick.....	72
Yeti Lube Insulating Liquid.....	72
Yeti Monolith Sculpturing Thowax.....	68
Yeti Nawax Compact	68
Yeti Occlusal Wax Patterns	72
Yeti Occlusal Wax Powder	72
Yeti Preci Dip Dipping Wax.....	66
Yeti Sample Set.....	68
Yeti Solidus Base Plate Wax	64
Yeti Special Thowax Waxes	69
Yeti Sprues.....	71
Yeti Thowax.....	69
Yeti VKS Wax	70
Yeti Wax Pontics	71
Wheel Dresser.....	19
Wheel Saw.....	49
White Acrylic Knock-Down Wheels	20
White Diamond Compound.....	53
Whitening Kit.....	75
Whitening Pen.....	75
Wire Clasps	40,41

INDEX

Wire Mesh.....	42
Wire Pins.....	55

X

X-Ray Sleeves.....	11
--------------------	----

Y

Yellow High Shine.....	53
Yeti Block Out Wax	64
Yeti Blue Marker.....	44
Yeti CAD/CAM Scanning Wax.....	65
Yeti Consequent Universal Wax	70
Yeti Creation Wax Set.....	66
Yeti Cutex Block Out Wax.....	65
Yeti Deton Wax Wire	70
Yeti Dia Glaze	44
Yeti Dia Quick.....	35
Yeti Die Bloc.....	44
Yeti Die Hardener	44
Yeti Die Spacer.....	43
Yeti Dippy NT Light.....	47
Yeti Dippy Pro Waxer Digital.....	48
Yeti Duo Dip Granular Dipping Wax	66
Yeti Elasto Dip Dipping Wax.....	66
Yeti Finale Wax Smoothing Liquid	72
Yeti Green Occlusal Spray.....	72
Yeti IQ Compact White Wax.....	67
Yeti IQ Pearl Wax	67
Yeti IQ Quadro Sticks	71
Yeti IQ Sculpturing Wax	69
Yeti IQ Sticks	71
Yeti IQ Triangle	71
Yeti Light Spacer	44
Yeti Lube Insulating Liquid.....	72
Yeti Monolith Sculpturing Thowax.....	68
Yeti Nawax Compact.....	68
Yeti Occlusal Wax Patterns.....	72
Yeti Occlusal Wax Powder	72
Yeti Preci Dip Dipping Wax.....	66
Yeti Pro Waxer Duo.....	48
Yeti Sakura Pencil.....	44
Yeti Sample Set.....	68
Yeti Solidus Base Plate Wax	64
Yeti Special Thowax Waxes	69
Yeti Sprues.....	71

Yeti Steam Off Spacer	44
Yeti Thowax.....	69
Yeti VKS Wax	70
Yeti Wax Debubblizer.....	38
Yeti Wax Pontics.....	71
Youth Mouthguard	85

Z

Z-Bur	62,90
Zinc Oxide Powder USP	75
Zinc Phosphate Cement.....	74

Proform Vacuum Formed Baseplate and Tray

By Tom Zaleske

Thermal plastic, vacuum formable materials are one of the most under used products in the dental laboratory. Given the low tech nature, most technicians look right past many unique and potentially time savings applications vacuum forming offers.

Thermal formable baseplate and custom tray material (styrene) are examples that on the surface appear to be rather one dimensional, but possess properties and characteristics underused by most. Besides providing well adapted baseplates, many technicians are unaware of the fact that acrylic sticks to styrene. This property allows for an acrylic tray handle, or chair-side border molding extensions to be easily and securely adapted without a secondary forming. Another unique property is the ability to form over itself, without adhering one layer to another. This lends itself well to providing a fast, non-melting spacer with tissue stops for vacuum formed trays.

For light cured trays, which typically produce a great deal of heat during the cure in a VLC box, wax spacers typically melt and adhere to the VLC material. Vacuum formable tray material can provide a clean, uniformly thick, easy to remove, post-cure spacer, which eliminates the time consuming task of cleaning melted wax off the tray. Melted wax not only has poor esthetic appeal, but also has the potential to interfere with adhesives used to secure impression material into the tray which may create inaccuracies by allowing the material to pull from the tray.

When fabricating implant borne prosthetic try-ins, a vacuum formed base provides the stability and support needed for wax and teeth. Coupled with the acrylic adhesion properties of the styrene, I have found it to be ideal for affixing baseplates to implant cylinders and bars.

So the next time someone mentions vacuum forming, realize it has more potential than making athletic mouthguards and bleach trays.

Available Baseplate (pink) thicknesses: .060, .080, .100


Available Tray (blue) thicknesses: .125, .150

Photo descriptions:

1. Comes in pink and blue
2. Provides uniform spacer thickness. Tissue stops are round rather than square to prevent corner compression.
3. Tray handle made with GC pattern resin.
4. Spacer and tray both vacuum formed.
5. Lincoln VLC tray and styrene spacer.
6. Warm to remove spacer from VLC tray.
7. Lincoln VLC tray complete.
8. Bar attached to baseplate.



Thomas Zaleske is the owner of Matrix Dental Laboratory and Consulting, specializing in removable prosthetics, and is the technical products coordinator for Keystone Industries. Join me on Facebook @ Zaleske Removable Prosthetics. More techniques of Tom Zaleskes can be found on his YouTube channel TomZale56.

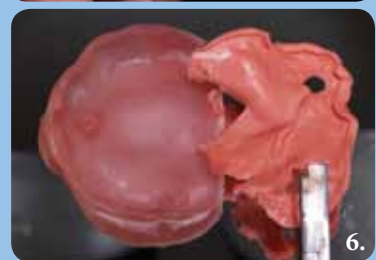



616 Hollywood Avenue
Cherry Hill, NJ 08002
USA



Werner-von-Siemens-Str. 14a
78224 Singen
Germany